


Articulating Legrest Support Assembly

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

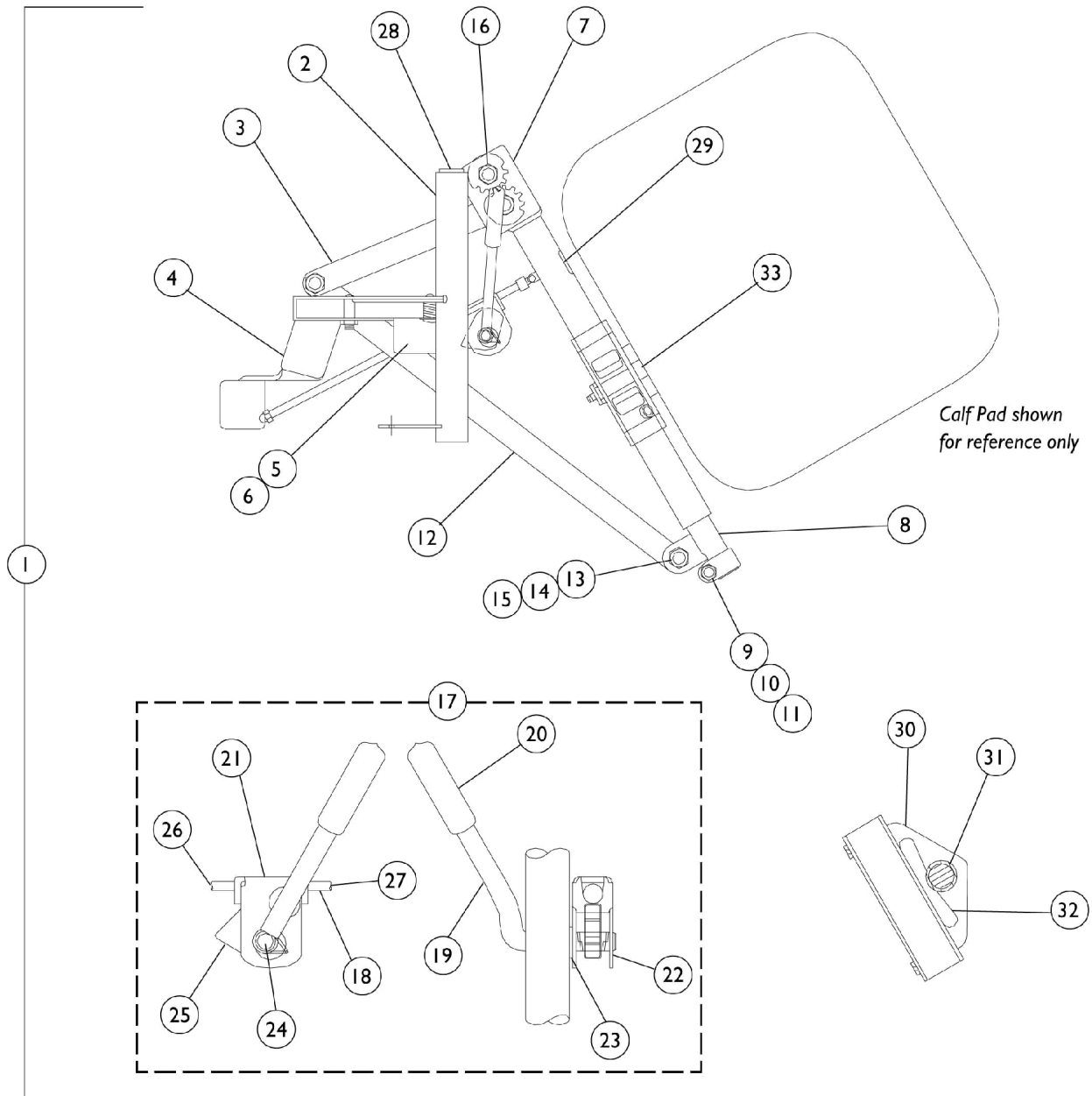

Articulating Legrest Support Assembly

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Part Number	Description	Package Quantity
1072298-NLA	Legrests with Footplates, Chrome (16")	2
1129608-NLA	Legrests with Footplates, Grey (16")	2
1072307-NLA	Legrests with Footplates, Chrome (18")	2
1129607-NLA	Legrests with Footplates, Grey (18")	2

Articulating Legrest Support Assembly

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,E	1041797-NLA	NLA - Legrest Support Assembly, Articulating, Adult, Chrome - Right	1	1
1	A	1149647	Legrest Support Assembly, Articulating, Adult, Chrome - Right	1	1
1	A,F	1041798-NLA	NLA - Legrest Support Assembly, Articulating, Adult, Chrome - Left	1	1
1	A	1149648	Legrest Support Assembly, Articulating, Adult, Chrome - Left	1	1
2	B	1022871-NLA	NLA - Legrest Support Tube Assembly, Adult, Chrome - Right	1	1
2	B	1022872-NLA	NLA - Legrest Support Tube Assembly, Adult, Chrome - Left	1	1
3		1022862	Tube, Legrest Support, Adult, Chrome - Right	1	1
3		1022863	Tube, Legrest Support, Adult, Chrome - Left	1	1
4		1031235-NLA	NLA - Handle, Release, Chrome - Right	1	1
4		1031236-NLA	NLA - Handle, Release, Chrome - Left	1	1
5		1022874	Bumper	1	1
6		1030979	Screw, Phillips Pan Head Tap (#10-32 x 3/4")	1	1
7		1022877-NLA	NLA - Tube, Support and Drive Link, Chrome - Right	1	1
7		1022878-NLA	NLA - Tube, Support and Drive Link, Chrome - Left	1	1
8		1022848	Tube, Slide, Articulating, Chrome - Right	1	1
8		1022849	Tube, Slide, Articulating, Chrome - Left	1	1
9		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
10		1027377	Washer (#20)	1	1
11		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	1
12		1022846-NLA	NLA - Bar, Link, Black	1	1
13		1022866-NLA	NLA - Screw, Socket Button Head (3/8-16 x 3/4")	1	2
14		1049527	Washer, Nylon (3/8 x 3/4 x 1/16")	1	2
15		40363X000	Hex Nut (3/8"-16)	1	3
16		1022854-NLA	NLA - Screw, Shoulder Slotted Head (1/2 x 1-1/8", 3/8-16)	1	1
17	C	1046329	Kit, Elevating Legrest Ratchet Hardware (Pair)	2	1
18		42326X004	Bar, Ratchet - Right/Left	1	1
19		1037126	Handle, Release	1	1
20		1040060	Tip, Plastic, Black	1	1
21		04528X002	Bracket, Release, Black	1	1
22		1040831	Ring, Snap	1	1
23		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
24		1037485	Spring, Release	1	1
25		1037125	Cam, Legrest	1	1
26		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
27		47330X003	Spacer (5/16 x 1/2 x 3/8")	1	1
28		1027097	Plug Button, Black (3/4")	1	1
29		1024911-NLA	NLA - Decal, Smartleg	1	1
30		1025104	Bracket, Legrest, Chrome	1	1
31		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
32		48823X008	Spring, Leaf	1	1

Articulating Legrest Support Assembly

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
33	D	1025133 See Chart	Spacer, Legrest Pad, Black Articulating Legrest Assembly	1 1	1 1
NOTE: A - Includes items 3-33. B - Includes items 3-6. C - Includes items 18-27. D - Not Shown. E - Item no longer available, refer to part number 1149647 for replacement. F - Item no longer available, refer to part number 1149648 for replacement.					