

**A-4®
Wheelchair**

(Chairs Built Before 9/16/2002)



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 94-162. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2022 Invacare Corporation. Elyria, Ohio. Invacare, Yes, you can., A-4, and the Medallion Design are registered trademarks of Invacare Corporation. KIK is a registered trademark of KIK Technology, Inc.

TABLE of CONTENTS

Description	Page Number
Accessories, Miscellaneous	68
Arm - "T" & Hardware	25
Arms - Half & Hardware	29
Axles & Camber Bars	37
Casters - Forks	31
Casters - Front and Mounting Hardware (After 6/29/11)	35
Casters - Front and Mounting Hardware (Before 7/1/11)	33
Frame Finishes & Touch-Up Paint	4
Frames	8
Front Riggings - Adjustable Angle Footplate Assemblies - AT5543	58
Front Riggings - Footplate - One Piece Adjustable Angle	62
Front Riggings - Footplates - Rigid Tubular	60
Packaging - Literature - Decals	6
Push Handles	66
Rear Upright Tube Mounting Hardware & Back Canes	12
Rear Upright Tubes & Backrest Cross Members	15
Rigid Side Guards	64
Seat Frames & Frame Hardware	10
Upholstery - Back - Adjustable Tension	21
Upholstery - Seat & Back	18
Wheel Locks - High Mount Push to Lock	49

TABLE of CONTENTS

Description	Page Number
Wheel Locks - Hill Holder & Attaching Hardware	54
Wheel Locks - Pull to Lock	52
Wheel Locks - Push/Pull to Lock (Replacements for High Mount Push to Lock)	51
Wheel Locks - Undermount	56
Wheels - Handrims	47
Wheels - Hardware - Handrims	48
Wheels - Rear - Composite w/o Handrims	41
Wheels - Rear - Spoke w/ Handrims	45
Wheels - Rear - Spoke w/o Handrims	42

Frame Finishes & Touch-Up Paint

Anodized frame finishes Pacific Blue (57A), Island Green (58A), Majestic Purple (59A), and Super Black (78A) are NLA through production or specials.

Color Number	Touch Up Paint	Color
22P	1033376-NLA	Anthracite
102P	1101327-NLA	Denim Blue
62P	1046320-NLA	Electric Blue
61P	1046321-NLA	Electric Red
104P	1101316-NLA	Electric Teal
80P	1073892-NLA	Evergreen with Navy
101P	1101325-NLA	Jeweled Green
84P	1073896-NLA	Lavender
105P	1101326-NLA	Midnight Blue
103P	1101324-NLA	Mocha Frost
66P	1046322-NLA	Electric Purple
33P	1031455-NLA	Red
83P	1073895-NLA	Royal Blue with Grape
30P	1031448-NLA	Sunny Yellow
24P	1031450-NLA	Wet Black
29P	1031447-NLA	White


Frame Finishes & Touch-Up Paint

Anodized frame finishes Pacific Blue (57A), Island Green (58A), Majestic Purple (59A), and Super Black (78A) are NLA through production or specials.

Sparkle Frame Finishes

Part Number	Color
72P	Black w/ Twilight Sparkle
73P	Black w/ Gold Sparkle
74P	Black w/ Green Sparkle
75P	Black w/ Blue Sparkle
76P	Black w/ Red Sparkle
88P	Wet Black w/ Twilight Sparkle


Packaging - Literature - Decals

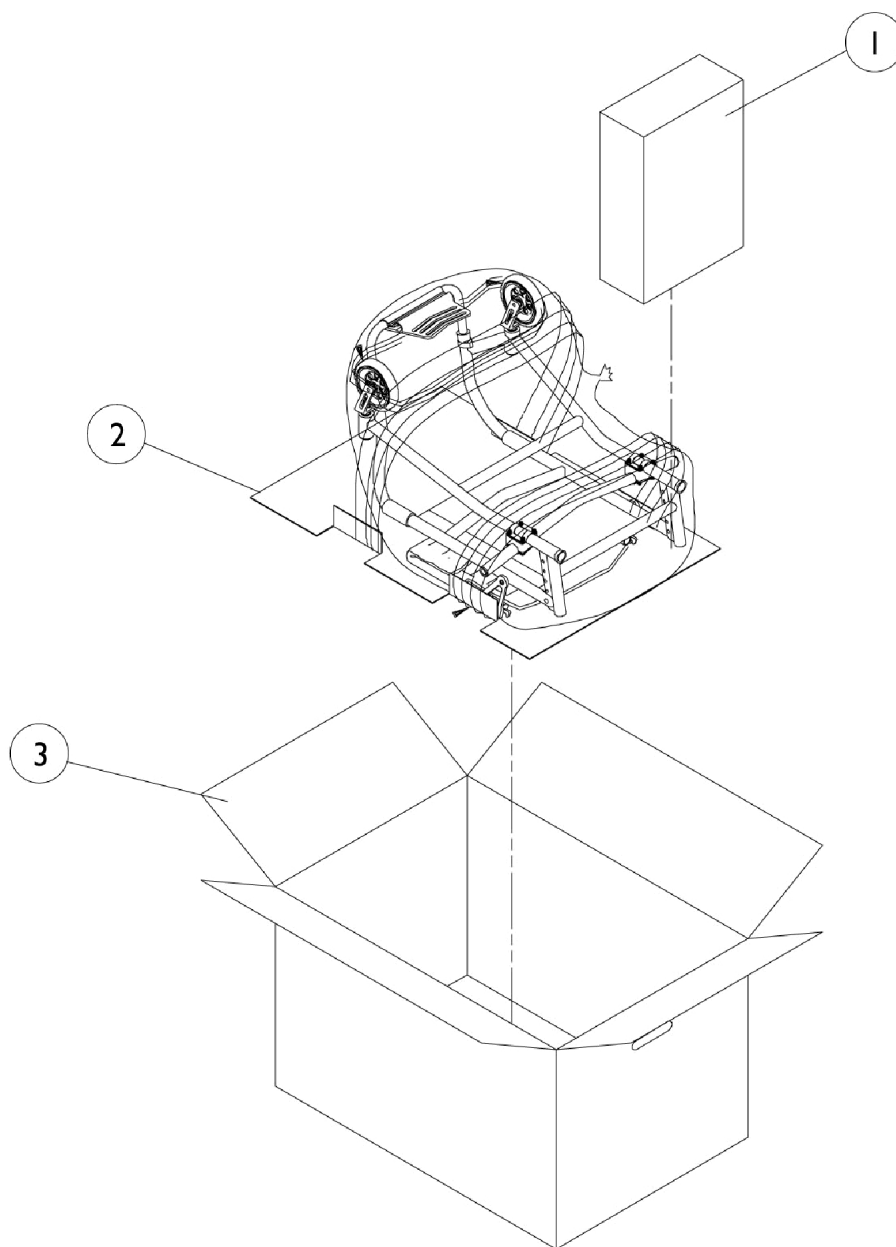


Packaging - Literature - Decals


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		40131X011-NLA	NLA - Carton, Front Riggings	1	1
2		1049222-NLA	NLA - Insert, Bottom Pad	1	1
3		1045583-NLA	NLA - Carton, Shipper	1	1
	A	1062605-NLA	NLA - Decal, Black	1	1
	A	1046498	Decal, White	1	1
	A,B	1110545-NLA	NLA - Owner's Manual, A-4	1	1
NOTE: A - Not Shown B - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					
When adjusting or replacing parts please refer to your Owner's Manual for assistance.					

Frames

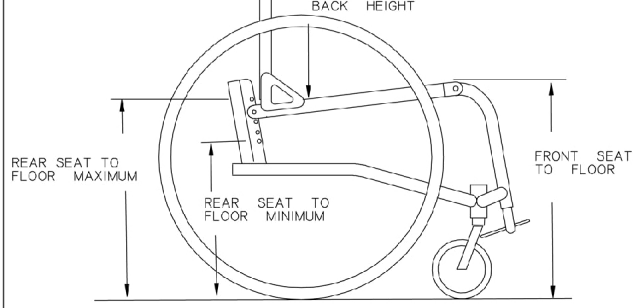
	A4 Frame Orders
 NOTE: When replacing rigid frames for chairs made Before 8/95, a new seat frame is required.	Orders for an A4 frame must be handled as a custom order. When placing the order, the following information is required. 1. Tapered or "V" front 2. Width and Depth. 3. Front seat-to-floor. 4. Rear seat-to-floor. 5. Caster size. 6. Rear wheel size. 7. For fixed back specify degree of angle. 8. For fixed back specify height range. 9. Frame color.

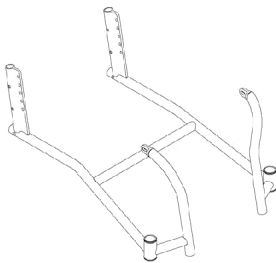


Tapered Front & Adj. RSTF

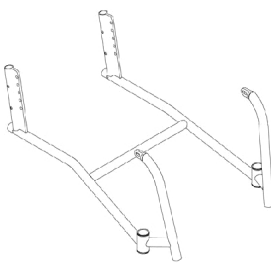


"V" Front & Adj. RSTF

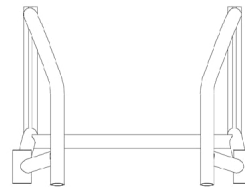

Tapered Front View

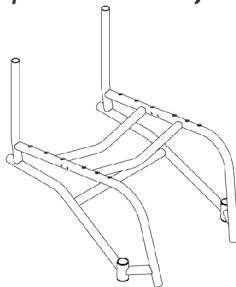

Fixed Back & Fixed RSTF
(Available w/ Tapered or "V" Front)

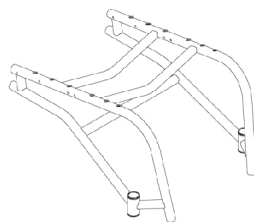

Fold Down Back & Fixed RSTF
(Available w/ Tapered or "V" Front)



"V" Front View

Frames


Frames - Chairs Built Before 11/26/96

Frame Front	Caster Size	Width	Front Seat to Floor	Adjustable RSTF	Fold Down Back Fixed RSTF	Fixed Back Fixed RSTF
Tapered	3"/5"	12-20"	17"	8881049328-NL A	8881060060-NL A	8881060069-NL A
Tapered	3"/5"	12-20"	19"	8881049329-NL A	8881060061-NL A	8881060070-NL A
Tapered	3"/5"	12-20"	21"	8881049330-NL A	8881060062-NL A	8881060071-NL A
"V"	3"/5"	12-20"	17"	8881056318-NL A	8881060063-NL A	8881060072-NL A
"V"	3"/5"	12-20"	19"	8881056319-NL A	8881060064-NL A	8881060073-NL A
"V"	3"/5"	12-20"	21"	8881056320-NL A	8881060065-NL A	8881060074-NL A
"V"	6"	12-20"	17"	8881057105-NL A	8881060066-NL A	8881060075-NL A
"V"	6"	12-20"	19"	8881057106-NL A	8881060067-NL A	8881060076-NL A
"V"	6"	12-20"	21"	8881057107-NL A	8881060068-NL A	8881060077-NL A

Frames - Chairs Built After 11/25/96

Adjustable RSTF	Fold Down Back Fixed RSTF	Fixed Back Fixed RSTF
8881070121	8881070122	8881070123

Seat Frames & Frame Hardware

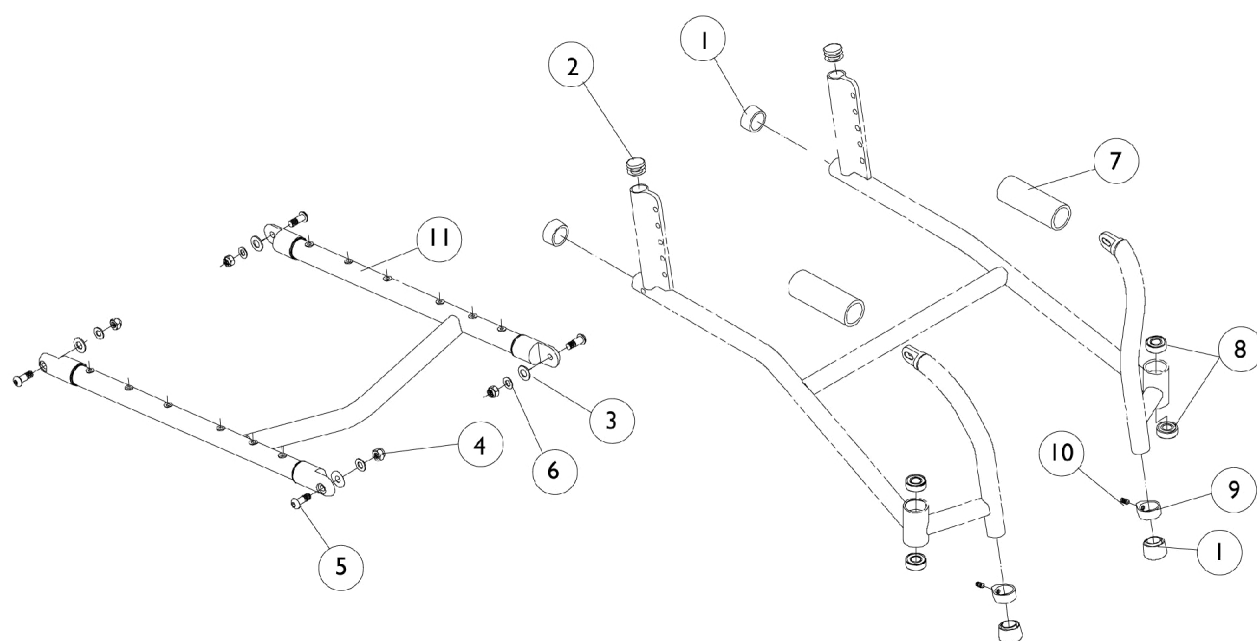

Seat Frames & Frame Hardware




Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		15680X013	Anti-Rattle (1")	1	4
2		1029501	Plug Button, Black (1")	1	2
3		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	4
4		1028769	Locknut (5/16-18)	1	4
5		1046494	Screw, Socket Button Head (5/16-18 x 7/8")	1	4
6		1046497-NLA	NLA - Washer, Nylon (5/16 x 7/8 x 1/64")	1	4
7		1046496-NLA	NLA - Pad, Foam	1	2
8		40117X021	Bearing (1/2" I.D. x 1-1/8" O.D.)	1	4
9		1024749	Clamp	1	2
10		1026911-NLA	NLA - Screw, Socket Set Cone Point (1/4-20 x 1/2")	1	2
11		See Chart	#11 Seat Frames and Frame Hardware	1	1

#11 Seat Frames


			Seat Depth			
Seat Width	14"	15"	16"	17"	18"	19"
12"	8881048722-NL A	--	--	--	--	--
13"	8881048723-NL A	--	--	--	--	--
14"	8881048724-NL A	8881048731-NL A	8881048739-NL A	8881048748-NL A	8881048755-NL A	--
15"	8881048725-NL A	8881048732-NL A	8881048740-NL A	8881048749-NL A	8881048756-NL A	8881048762-NL A
16"	8881048726-NL A	8881048733-NL A	8881048741-NL A	8881048750-NL A	8881048757-NL A	8881048763-NL A
17"	8881048727-NL A	8881048734-NL A	8881048742-NL A	8881048751-NL A	8881048758-NL A	8881048764-NL A
18"	8881048728-NL A	8881048735-NL A	8881048743-NL A	8881048752-NL A	8881048759-NL A	8881048765-NL A
19"	--	--	8881048744-NL A	8881048753-NL A	8881048760-NL A	8881048766-NL A
20"	--	--	8881048745-NL A	8881048754-NL A	8881048761-NL A	8881048767-NL A
21"	--	--	--	--	8881082651-NL A	--
22"	--	--	--	--	8881082652-NL A	--

Specify frame color when ordering.

Rear Upright Tube Mounting Hardware & Back Canes

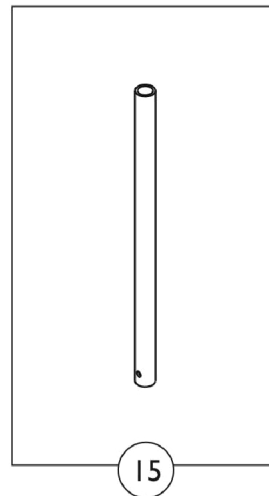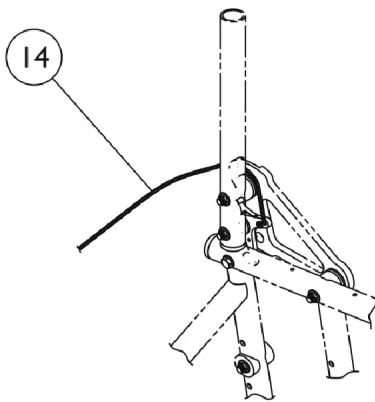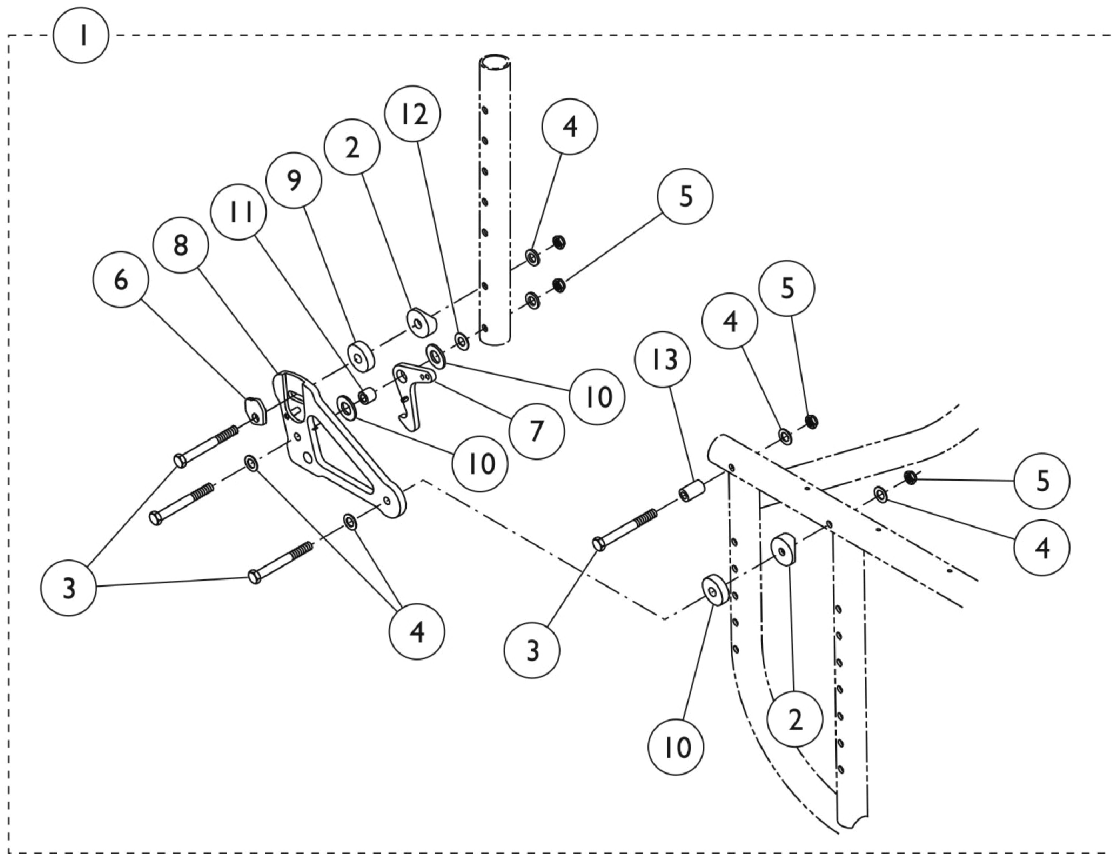

**Adjustable Back
Cane w/o
Push Handle**

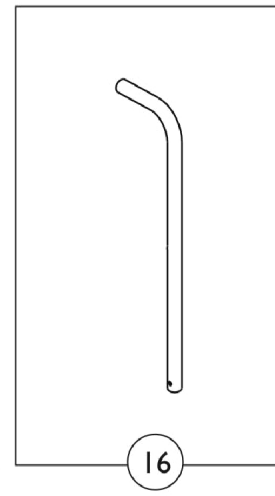

**Straight Cane
w/ Push
Handle**


Rear Upright Tube Mounting Hardware & Back Canes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1041149-NLA	NLA - Kit, Backrest Hardware (Pair)	1	1
2		40157X020	Spacer, Coved (1/4 x 1 x 9/16")	1	2
3		1055349	Screw, Hex Head (1/4-28 x 2")	1	4
4		1026907	Washer (1/4 x 1/2 x 1/16")	1	16
5		1025765	Locknut (1/4-28)	1	4
6		1039232-NLA	NLA - Cam, Adjustment	1	1
7		1081354-NLA	NLA - Latch, Backrest - Right	1	1
7		1081355-NLA	NLA - Latch, Backrest - Left	1	1
8		1079920-NLA	NLA - Plate, Backrest Support -Right	1	1
8		1079921-NLA	NLA - Plate, Backrest Support -Left	1	1
9		1040776-NLA	NLA - Spacer, Nylon (1/4 x 1 x 1/4")	1	2
10		1012371	Washer, Nylon (3/8 x 7/8 x 1/16")	1	1
11		1026162	Spacer (1/4 x 3/8 x 11/32")	1	1
12		1025728	Washer (1/4 x 1/2 x 1/32")	1	1
13		1039233	Spacer (1/4 x 7/16 x 5/8")	1	1
14		1141633	Cord, Nylon (6-1/2 ft.)	1	1
15		1026971	Back Canes, Adjustable w/o Push Handle (8-11" H)	2	1
			Plug Button, Black (3/4")	1	2
			NLA - Tube, Insert, Adjustable Back (8 - 11" H)	1	2
			Spring, Torsion w/ Button	1	2
			Bushing, Pivot	1	2
15		1026972	Back Canes, Adjustable w/o Push Handle (10-14"/16-20" H)	2	1
			Plug Button, Black (3/4")	1	2
			NLA - Tube, Insert, Adjustable Back (10 - 20" H)	1	2
			Spring, Torsion w/ Button	1	2
			Bushing, Pivot	1	2
16		1054773-NLA	NLA - Cane, Back, Straight w/ Push Handle and Hardware (10-14" H)	1	1
			Grip, Foam (8")	1	1
			Plug Button, Black (3/4")	1	1
			NLA - Back Cane w/ Push Handle, Straight (10-14" H)	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			NLA - Locknut (1/4-20)	1	1
16		1054774-NLA	NLA - Cane, Back, Straight w/ Push Handle and Hardware (12-16" H)	1	1
			Grip, Foam (8")	1	1
			Plug Button, Black (3/4")	1	1
			NLA - Back Cane w/ Push Handle, Straight (12-16" H)	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			NLA - Locknut (1/4-20)	1	1
16		1027850-NLA	NLA - Cane, Back, Straight w/ Push Handle and Hardware (12-16" and 14-18" H)	1	1
			Grip, Foam (8")	1	1

Rear Upright Tube Mounting Hardware & Back Canes

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
16		1044031-NLA	NLA - Cane, Back, Straight, Black - Right/Left	1	1
			Screw w/ Washer (#10 x 7/8")	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			NLA - Locknut (1/4-20)	1	1
			NLA - Back Cane w/ Hardware, Straight w/ Push Handle (14-18" and 16-20" H)	1	1
			NLA - Back Cane, Straight w/ Push Handle	1	1
			Grip, Foam (8")	1	1
			Screw, Socket Button Head (1/4-20 x 1-1/4")	1	1
			NLA - Locknut (1/4-20)	1	1
	B,C	1038784-NLA	NLA - Pin, Roll (3/16 x 1/2")	1	1
	B,C	1038785-NLA	NLA - Pin, Roll (1/8 x 3/8")	1	1
	B,C	1039231-NLA	NLA - Latch, Black	1	1
	B,C	1039234-NLA	NLA - Spring, Leaf	1	1
	B	1039235-NLA	NLA - Plate, Backrest Support, Right	1	1
	B	1039236-NLA	NLA - Plate, Backrest Support, Left	1	1
NOTE: A - Includes Items 2 thru 13. B - Not Shown. C - Before 5/7/98 only.					

Rear Upright Tubes & Backrest Cross Members

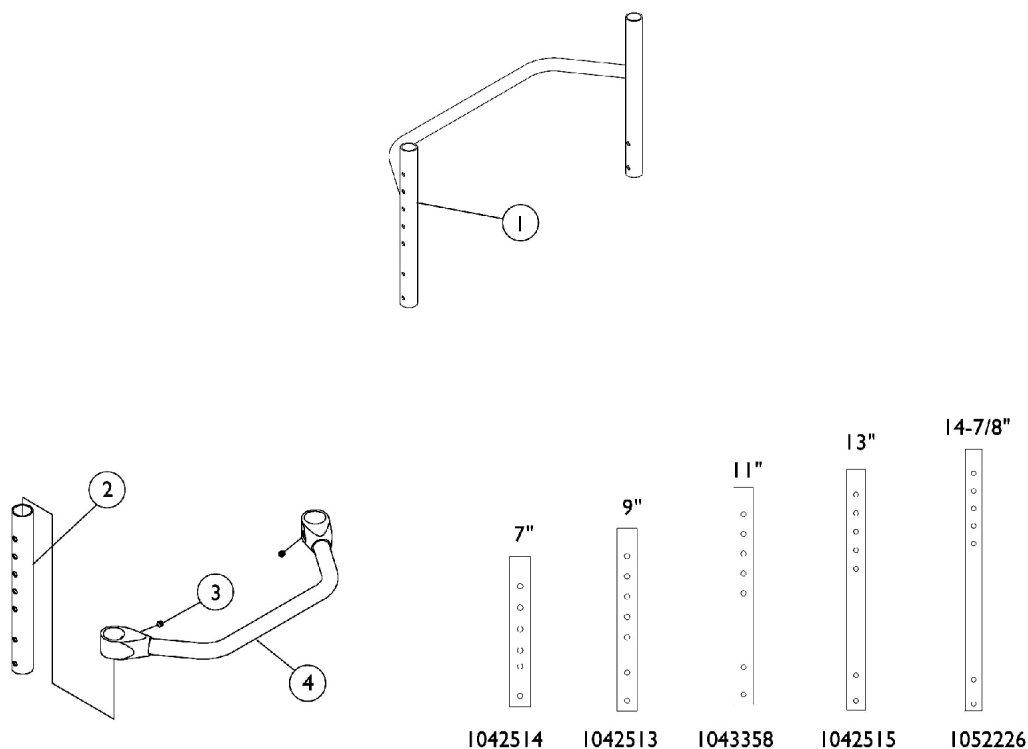

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		See Chart	#1 One Piece Backrest Frame	1	1
2		1042514-NLA	NLA - Tube, Rear Upright, Black (8 - 11" H)	1	2
2		1042513-NLA	NLA - Tube, Rear Upright, Black (10 - 14" H)	1	2
2		1043358	Tube, Rear Upright, Black (12 - 16" H)	1	2
2		1042515-NLA	NLA - Tube, Rear Upright, Black (14 - 18" H)	1	2
2		1052226-NLA	NLA - Tube, Rear Upright, Black (16 - 20" H)	1	2
3		1045495-NLA	NLA - Screw, Set Socket (1/4-20 x 1/4")	1	2
4		See Chart	#4 Backrest Frame and Rear Upright Tubes (12-18" Wide)	1	1
4		See Chart	#4 Backrest Frame and Rear Upright Tubes (19-22" Wide)	1	1

Rear Upright Tubes & Backrest Cross Members

#1 One Piece Backrest Frame

Chair Width	8-11" High	10-14" High	12-16" High	14-18" High	16-20" High
12"	1071788-NLA	1071789-NLA	1071790-NLA	1071791-NLA	1071792-NLA
13"	1071793-NLA	1071794-NLA	1071795-NLA	1071796-NLA	1071797-NLA
14"	1024706-NLA	1026188-NLA	1042982-NLA	1026757-NLA	1026764-NLA
15"	1024707-NLA	1026189-NLA	1042986-NLA	1026758-NLA	1026765-NLA
16"	1024708-NLA	1026190-NLA	1042987-NLA	1026759-NLA	1026766-NLA
17"	1024709-NLA	1026191-NLA	1042988-NLA	1026760-NLA	1026767-NLA
18"	1024710-NLA	1026192-NLA	1042989-NLA	1026761-NLA	1026768-NLA
19"	1024711-NLA	1026193-NLA	1042990-NLA	1026762-NLA	1026769-NLA
20"	1024712-NLA	1026194-NLA	1042991	1026763-NLA	1026770-NLA
21"	1082654-NLA	1082952-NLA	1082953-NLA	1082954-NLA	1082955-NLA
22"	1082956-NLA	1082957-NLA	1082958-NLA	1082959-NLA	1082960-NLA

Rear Upright Tubes & Backrest Cross Members

#4 Backrest Crossmember

(12-18" Wide Chairs)


12" Wide	13" Wide	14" Wide	15" Wide	16" Wide	17" Wide	18" Wide
1042508-NLA	1042509-NLA	1042510	1042511	1042512	1050083	1050080

#4 Backrest Crossmember


(19-22" Wide Chairs)

19" Wide	20" Wide	21" Wide	22" Wide
1050077	1050074	1085292-NLA	1085293-NLA

Seat & Back Upholstery

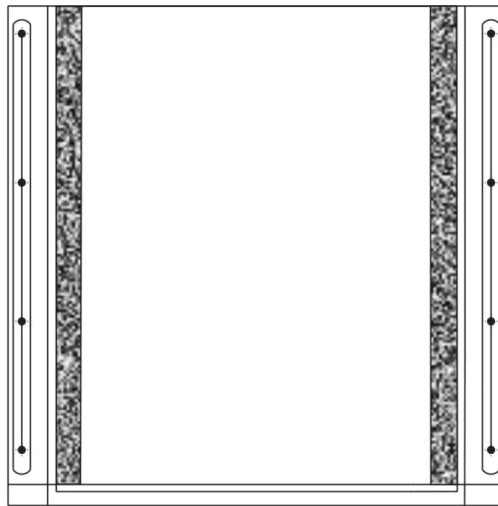

Seat Upholstery
- Sail Cloth

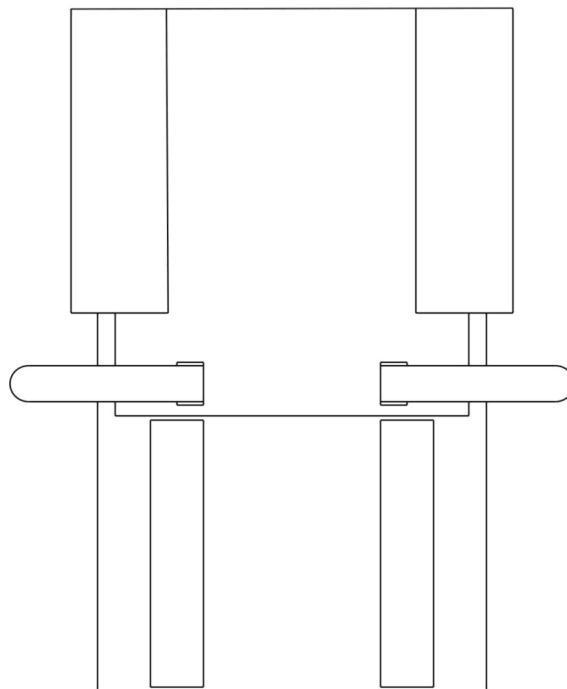

Back Upholstery
- Nylon

Seat & Back Upholstery

Seat Upholstery

Seat Width	14" Seat Depth	15" Seat Depth	16" Seat Depth	17" Seat Depth	18" Seat Depth	19" Seat Depth
12"	1032174-NLA	1065142-NLA	--	--	--	--
13"	1032179-NLA	1064706-NLA	1065143-NLA	--	--	--
14"	1032080	1032081	1032082	1032083	1032084	--
15"	1032085	1032086	1032087	1032088	1032089	--
16"	1032090	1032091	1032092	1032093	1032094	1046478-NLA
17"	1032095	1032096	1032097	1032098	1032099	1046479-NLA
18"	1032100	1032101	1032102	1032103	1032104	1046480-NLA
19"	--	--	1032105	1032106	1032107	1046481-NLA
20"	--	--	1032108	1032109	1032110	1046482-NLA
21"	--	--	--	--	1083022-NLA	1114115-NLA
22"	--	--	--	--	1083028-NLA	1114116-NLA

Seat Upholstery Hardware Kits

Seat Depth	Hardware Kit	Screw w/ Washer Quantity/Size per Kit
14-15"	1026931-NLA	10 ea. (10-32 x 3/4")
16-17"	1026932-NLA	12 ea. (10-32 x 3/4")
18-19"	1026933-NLA	14 ea. (10-32 x 3/4")

Seat & Back Upholstery


Back Upholstery

Chair Width	8-11" Back Height	10-14" Back Height	12-16" Back Height	14-18" Back Height	16-20" Back Height
12"	1043361-NLA	1042521-NLA	1042525-NLA	1042529-NLA	1042519-NLA
13"	1043362-NLA	1042522-NLA	1042526-NLA	1042530-NLA	1042520-NLA
14"	1026795-NLA	1026802	1043539-NLA	1026809-NLA	1026816-NLA
15"	1026796-NLA	1026803-NLA	1043540-NLA	1026810-NLA	1026817-NLA
16"	1026797-NLA	1026804-NLA	1043541-NLA	1026811-NLA	1026818-NLA
17"	1026798-NLA	1026805-NLA	1043542-NLA	1026812-NLA	1026819-NLA
18"	1026799-NLA	1026806-NLA	1042983-NLA	1026813-NLA	1026820-NLA
19"	1026800-NLA	1026807-NLA	1043543-NLA	1026814-NLA	1026821-NLA
20"	1026801-NLA	1026808-NLA	1043544-NLA	1026815-NLA	1026822-NLA
21"					1083021-NLA
22"					1083027

Back Upholstery Hardware

Kit Part Number	Kit Includes:
1029850	2 ea. Screws (#10 x 7/8") 2ea. Cable Ties

Adjustable Tension Back Upholstery

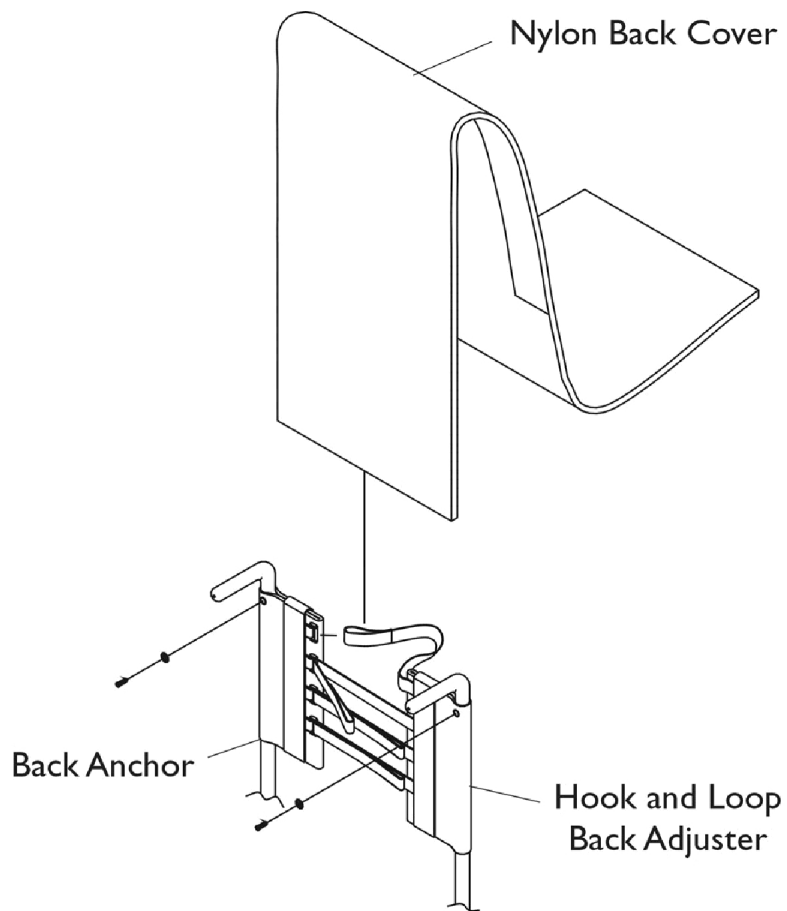

Adjustable Tension Back Upholstery

Back Height	Seat Width	Back Anchor	Hook & Loop Adjuster	Back Cover	Back Assembly See Note(s) A
8-11"	12"	1046405-NLA	1046401-NLA	1046242-NLA	1079092-NLA
8-11"	13"	1046405-NLA	1046401-NLA	1046243	1079093-NLA
8-11"	14"	1046405-NLA	1046402-NLA	1046244	1079094-NLA
8-11"	15"	1046405-NLA	1046402-NLA	1046245	1079095-NLA
8-11"	16"	1046405-NLA	1046402-NLA	1046246	1079096
8-11"	17"	1046405-NLA	1046403-NLA	1046247	1079097-NLA
8-11"	18"	1046405-NLA	1046403-NLA	1046248-NLA	1079098-NLA
8-11"	19"	1046405-NLA	1046404-NLA	1046249	1079099-NLA
8-11"	20"	1046405-NLA	1046404-NLA	1046250-NLA	1079100-NLA
10-14"	12"	1046287-NLA	1046283-NLA	1050685-NLA	1079101-NLA
10-14"	13"	1046287-NLA	1046283-NLA	1050688	1079102-NLA
10-14"	14"	1046287-NLA	1046284-NLA	1037929	1079103-NLA
10-14"	15"	1046287-NLA	1046284-NLA	1037930	1079104-NLA
10-14"	16"	1046287-NLA	1046284-NLA	1037931	1079105-NLA
10-14"	17"	1046287-NLA	1046285-NLA	1037932	1079106-NLA
10-14"	18"	1046287-NLA	1046285-NLA	1037933	1079107
10-14"	19"	1046287-NLA	1046286-NLA	1037934	1079108-NLA
10-14"	20"	1046287-NLA	1046286-NLA	1037935	1079109-NLA
12-16"	12"	1037916	1052740-NLA	1050686-NLA	1052745-NLA
12-16"	13"	1037916	1052740-NLA	1050689	1052746-NLA
12-16"	14"	1037916	1037925	1037918	1039478
12-16"	15"	1037916	1037925	1037919	1039479
12-16"	16"	1037916	1037925	1037920	1039480
12-16"	17"	1037916	1037917	1037921	1039481
12-16"	18"	1037916	1037917	1037922	1039482
12-16"	19"	1037916	1037926	1037923	1039483
12-16"	20"	1037916	1037926	1037924	1039484
14-18"	12"	1037720-NLA	1052741-NLA	1050687-NLA	1052747-NLA
14-18"	13"	1037720-NLA	1052741-NLA	1050690	1052748-NLA
14-18"	14"	1037720-NLA	1037736	1037922	1039485
14-18"	15"	1037720-NLA	1037736	1037923	1039486
14-18"	16"	1037720-NLA	1037736	1037924	1039487
14-18"	17"	1037720-NLA	1037721	1037925	1039488-NLA
14-18"	18"	1037720-NLA	1037721	1037926	1039489-NLA
14-18"	19"	1037720-NLA	1037737	1037927	1039490-NLA
14-18"	20"	1037720-NLA	1037737	1037928	1039491
16-20"	12"	1046731	1046732	1050687-NLA	1052747-NLA
16-20"	13"	1046731	1046732	1050690	1052748-NLA
16-20"	14"	1046731	1046732	1046735-NLA	1046742
16-20"	15"	1046731	1046732	1046736	1046743
16-20"	16"	1046731	1046732	1046737	1046744

Adjustable Tension Back Upholstery


Back Height	Seat Width	Back Anchor	Hook & Loop Adjuster	Back Cover	Back Assembly See Note(s) A
16-20"	17"	1046731	1046733	1046738	1046745
16-20"	18"	1046731	1046733	1046739	1046746
16-20"	19"	1046731	1046734	1046740	1046747
16-20"	20"	1046731	1046734	1046741	1046748
NOTE: A - Back Assembly includes Back Anchor, Hook and Loop Adjuster, Back Cover and Instruction Sheet.					

Adjustable Tension Back Upholstery

Back Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1026298	Package, Screw with Washer (#10 x 7/8") (Pair)	2	1
	A,B	1039470-NLA	NLA - Instruction Sheet, Adjustable Tension Back Upholstery	1	1
	A	See Chart	Back Upholstery -Adjustable Tension	1	1
NOTE: A - Not Shown B - Item is no longer available from the Invacare Service Parts Department. Refer to the Invacare Website under "Product Literature Search". Enter the part number in the "Keywords" field and download a free PDF version copy.					

"T" Arm & Hardware

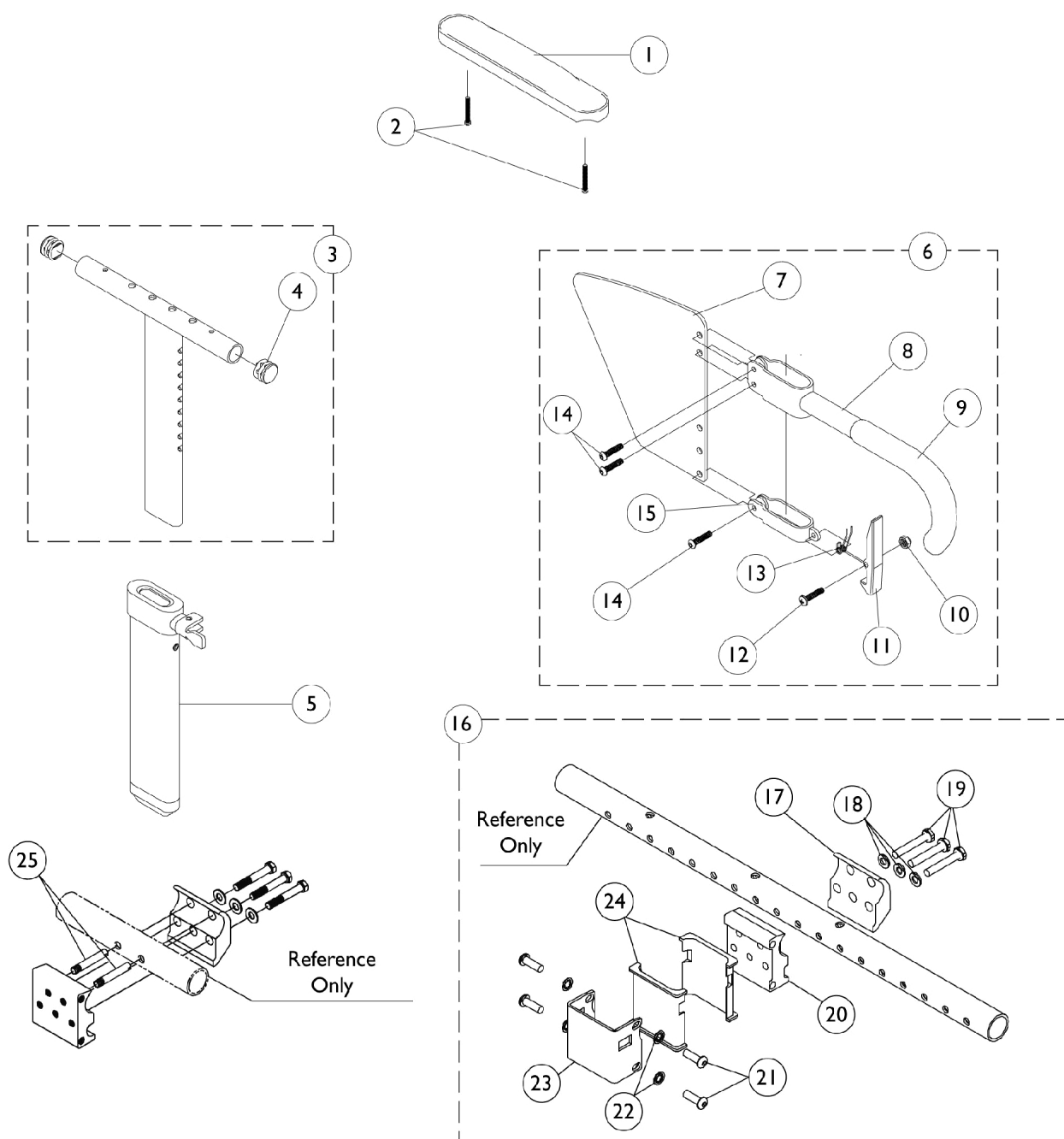

"T" Arm & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881053570	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
1	A	8881053571	Kit, Arm Pad w/ Screws, Full Length (Pair)	2	1
2		1055205	Package, Screw Phillips Pan Head (#10-32 x 1-3/8")	2	1
3		1069639	T-Arm Assembly w/ Plug Buttons, Desk Length, Low Range, Black	1	1
3		1069643	T-Arm Assembly w/ Plug Buttons, Full Length, Low Range, Black	1	1
3		1069641	T-Arm Assembly w/ Plug Buttons, Desk Length, High Range, Black	1	1
3		1069645	T-Arm Assembly w/ Plug Buttons, Full Length, High Range, Black	1	1
4		1027095	Plug Button, Black (7/8")	1	2
5		1069635	Post, Arm, Low Range	1	1
5		1069637	Post, Arm, High Range	1	1
6		1073424	Kit, Clothing Guard, Low Range - Right	1	1
6		1073425	Kit, Clothing Guard, Low Range - Left	1	1
6		1073422	Kit, Clothing Guard, High Range - Right	1	1
6		1073423	Kit, Clothing Guard, High Range - Left	1	1
7		1069652-NLA	NLA - Guard, Clothing, Low Range	1	1
7		1069651-NLA	NLA - Guard, Clothing, High Range	1	1
8		1069634	Tube, Support, Low Range, Black - Right	1	1
8		1073247-NLA	NLA - Tube, Support, Low Range, Black - Left	1	1
8		1069650-NLA	NLA - Tube, Support, High Range, Black - Right	1	1
8		1073246	Tube, Support, High Range, Black - Left	1	1
9		1028427	Grip, Foam	1	1
10		1025731	Locknut (#10-32)	1	1
11		1076839-NLA	NLA - Lever, Locking	1	1
12		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
13		1073196-NLA	NLA - Spring, Torsion	1	1
14		1006773-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1/2")	1	3
15		1075069-NLA	NLA - Clamp, Clothing Guard - Right	1	1
15		1075070-NLA	NLA - Clamp, Clothing Guard - Left	1	1
16		1069627-NLA	NLA - Arm Socket Assembly, Clamp On	1	1
17		1069605	Clamp, "T" Arm Socket - Inside	1	1
18		1026907	Washer (1/4 x 1/2 x 1/16")	1	3
19		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	3
20		1075162-NLA	NLA - Clamp, "T" Arm Socket - Outside	1	1
21		1055346-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	4
22		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	4
23		1069620-NLA	NLA - Bracket, Socket, Black	1	1
24		1069609	Insert, Arm Socket, Black	1	2
25		1058545	Lever, Actuator	1	1
	B	See Chart	"T" Arm Kits	1	1

"T" Arm & Hardware

NOTE: A - Specify U67 color when ordering.
B - Includes items 1-3, 5-6.


"T" Arm & Hardware

"T" Arm Kits

Kits Include Items 1-3,5-6

Part Number	Arm Style
8881074078	Desk Length, Pediatric (1 Pair)
8881074079	Desk Length, Adult (1 Pair)
8881074080	Full Length, Pediatric (1 Pair)
8881074081-NLA	Full Length, Adult (1 Pair)
Specify U67 upholstery color when ordering.	



Half Arms & Hardware

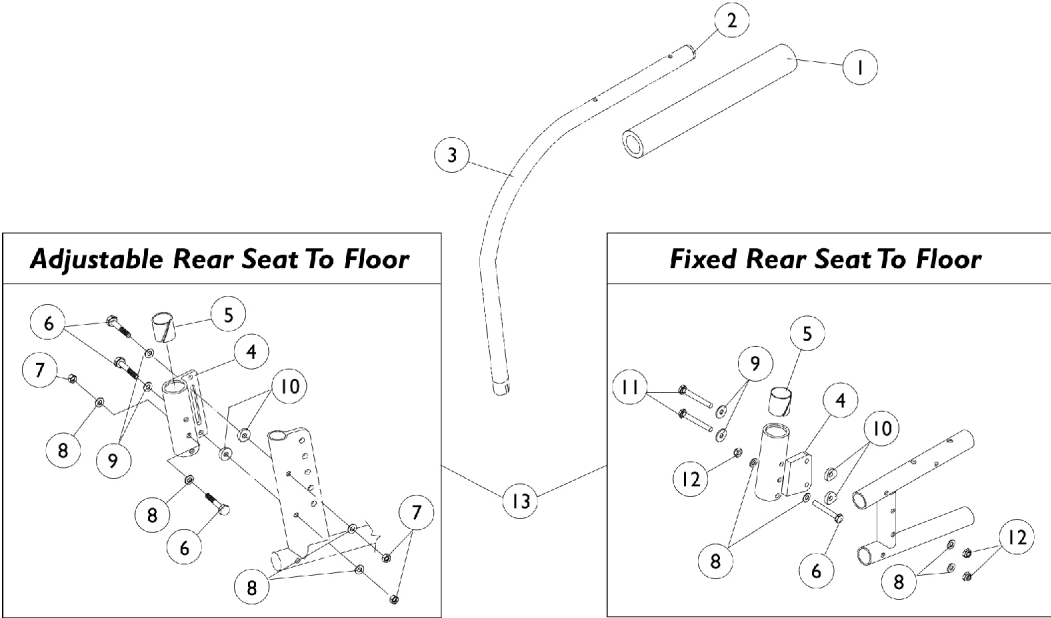

Half Arms & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1064064	Grip, Foam - Full Length	1	1
2		1056328	Plug Button, Black (1")	1	1
3		1056644-NLA	NLA - Half Arm Tube Assembly - Right	1	1
3		1056643-NLA	NLA - Half Arm Tube Assembly - Left	1	1
4	A,B	1032206-NLA	NLA - Socket, Arm, Rear - Right	1	1
4	A,B	1032207-NLA	NLA - Socket, Arm, Rear - Left	1	1
4	C	1079287-NLA	NLA - Socket, Arm - Right	1	1
4	C	1079288-NLA	NLA - Socket, Arm - Left	1	1
5		1018687	Sleeve, Bearing	1	1
6		1055348	Screw, Hex Head (1/4-28 x 1-3/4")	1	1
7		1065362	Locknut (1/4-28)	1	3
8		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
9		1026900	Washer (1/4 x 23/32 x 1/16")	1	2
10		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
11		1055349	Screw, Hex Head (1/4-28 x 2")	1	2
12		1025765	Locknut (1/4-28)	1	3
13	B	1056651-NLA	NLA - Hardware, Half Arm Mounting (Pair)	2	1
13	C	1083029-NLA	NLA - Hardware, Half Arm Mounting (Pair)	2	1
NOTE: A - After 12/00; For chairs built Before 1/01 order item number 13 (p/n 1056651) (1 Pair) B - Adjustable Rear Seat to Floor C - Fixed Rear Seat to Floor					


Caster Forks

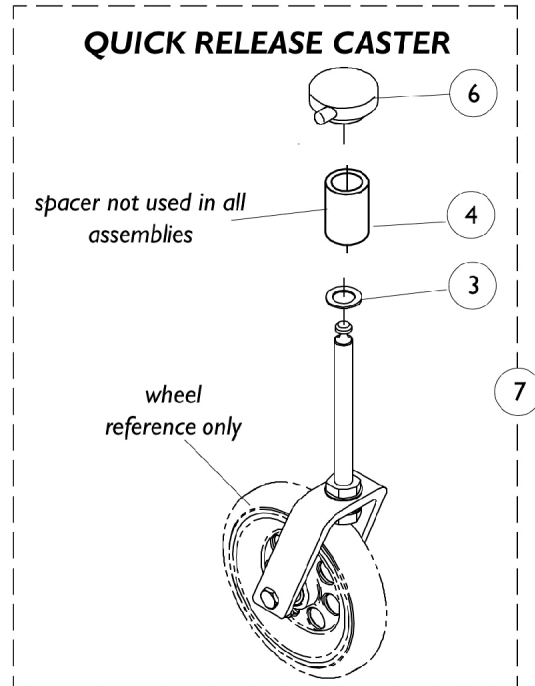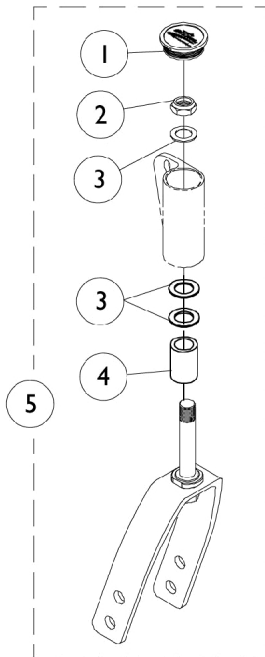

Caster Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1038353	Plug Button, Headtube, Black	1	1
2		1025737	Locknut (1/2-20)	1	1
3		1026174	Washer (1/2 x 7/8 x 3/64")	1	3
4		1046495	Spacer (1/2 x 3/4 x 1")	1	1
4		1026852	Spacer (1/2 x 3/4 x 2")	1	1
5		1065623	Kit, Fork w/ Hardware (3" Caster)	1	1
5		1065624	Kit, Fork w/ Hardware, 1" Extension (3 and 5" Caster)	1	1
5		1065625	Kit, Fork with Hardware and 2" Spacer (3" Caster)	1	1
5		1065626	Kit, Fork w/ Hardware (5" Caster)	1	1
5		1065627	Kit, Fork w/ Hardware, 1" Extension (5 and 6" Caster)	1	1
5		1065628	Kit, Fork w/ Hardware (6 and 6 x 1" Casters)	1	1
6		1042044-NLA	NLA - Cap, Head Tube, Quick Release Caster, Black	1	1
7		1066391-NLA	NLA - Quick Release Caster Fork Assembly, 3" Quick Release Fork, 2-1/2" Caster Headtube	1	1
7		1066392-NLA	NLA - Quick Release Caster Fork Assembly, 3" Quick Release Fork with 1" Extension, 2-1/2" Caster Headtube,	1	1
7		1066393-NLA	NLA - Quick Release Caster Fork Assembly, 3" Quick Release Fork with 2" Extension, 2-1/2" Caster Headtube	1	1
7		1066394-NLA	NLA - Quick Release Caster Fork Assembly, 5" Quick Release Fork, 2-1/2" Caster Headtube	1	1
7		1066395-NLA	NLA - Quick Release Caster Fork Assembly, 5" Quick Release Fork with 1" Extension, 2-1/2" Caster Headtube	1	1
7		1066396-NLA	NLA - Quick Release Caster Fork Assembly, 5" Quick Release Fork with 2" Extension, 2-1/2" Caster Headtube	1	1
7		1069162-NLA	NLA - Quick Release Caster Fork Assembly, 6" Quick Release Fork, 2-1/2" Caster Headtube	1	1

Caster Wheels & Mounting Hardware (Before 7/1/11)

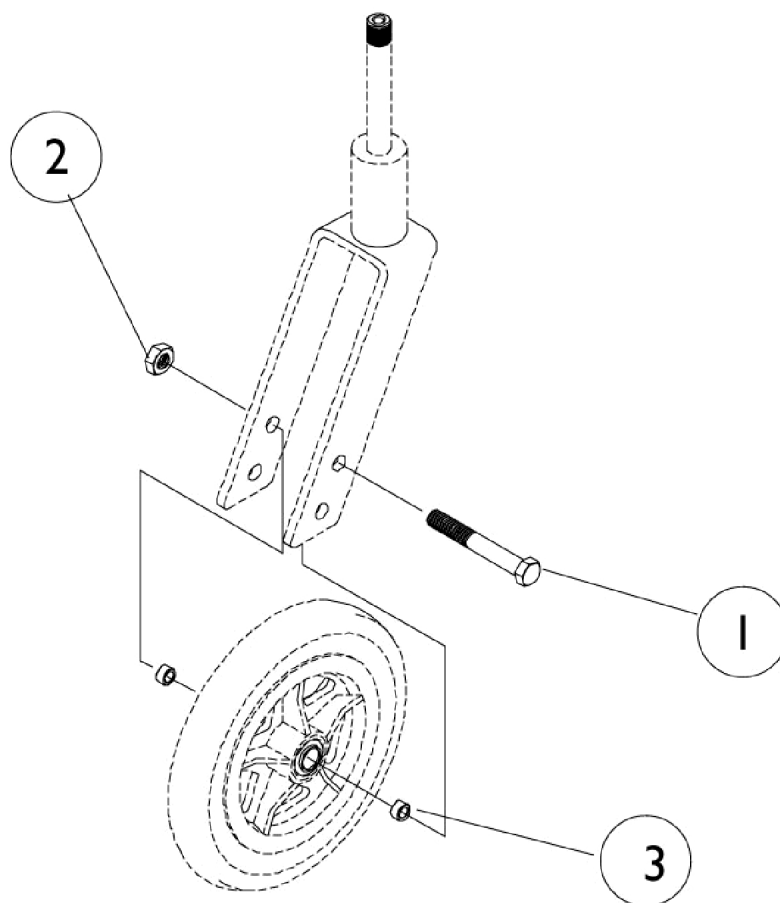

Caster Hardware

Wheel Style	Wheel Size	Tire Type	#1 Axle	#2 Axle Nut	#3 Axle Spacer
Composite	3"/5"	Urethane	1026908	7930330-NLA	1026102
Composite	6"	Urethane	1025764	7930330-NLA	1026102
Composite	6"	Semi-Pneumatic	1025764	7930330-NLA	1026102

Caster Wheels & Mounting Hardware (Before 7/1/11)

Caster Wheels


Wheel Style	Wheel Size	Tire Type	Bearing	Bearing Spacer	Caster Single # See Note(s) A	Tire	Tube/ Insert
AT900	3"	Urethane	40117X026	1026172	1032190-NLA	N/A	N/A
5ALUM	5"	Urethane	40117X026	1026172	1026952	N/A	N/A
AT901	5"	Urethane	40117X026	1026172	1029496	N/A	N/A
AT902	6"	Urethane	N/A	N/A	1034352 See Note(s) B	N/A	N/A
6SKYWAY	6"	Urethane	40117X026	1026172	1026956	N/A	N/A
6PNEU	6"	Pneumatic	40117X026	1026172	1044040	1026010-NLA	1026011-NLA

NOTE: A - Casters include Tire, Tube (if applicable), and Bearings.

B - Consists of (2) Casters.

6" Pneumatic Wheels with Flat-Free Inserts are No Longer Available (NLA). 6" Pneumatic Wheels are suggested as a replacement. It is recommended that they are ordered in pairs to match.

Caster Wheels & Mounting Hardware (After 6/29/11)

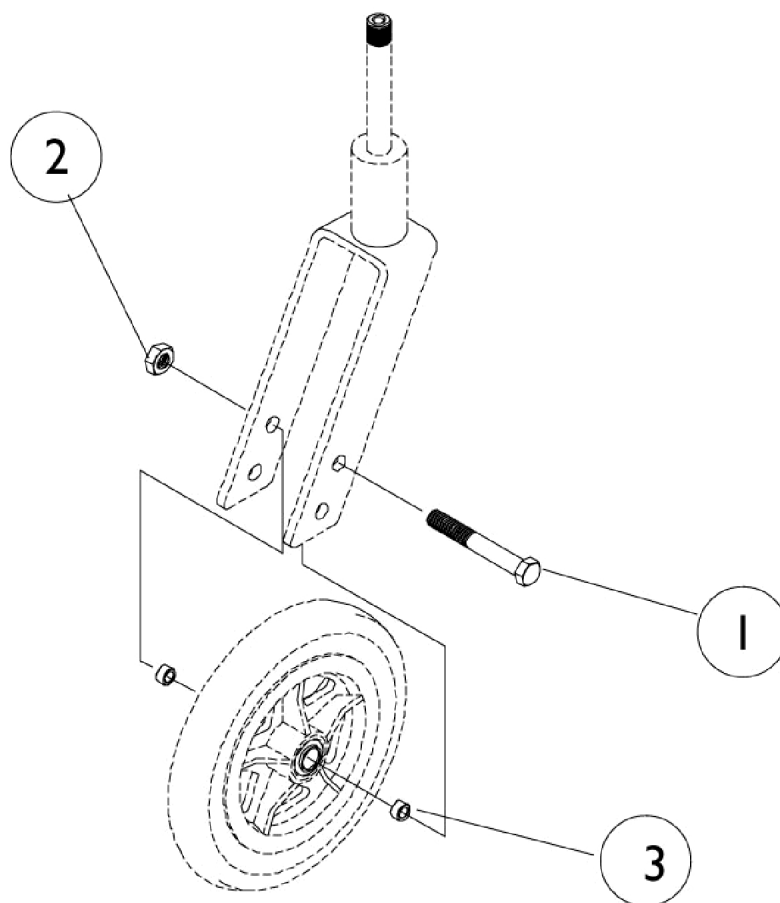

Caster Hardware

Wheel Style	Wheel Size	Tire Type	#1 Axle	#2 Axle Nut	#3 Axle Spacer
Composite	3"/5"	Urethane	1026908	7930330-NLA	1026102
Composite	6"	Urethane	1025764	7930330-NLA	1026102
Composite	6"	Semi-Pneumatic	1025764	7930330-NLA	1026102

Caster Wheels & Mounting Hardware (After 6/29/11)

Caster Wheels


Wheel Style	Wheel Size	Tire Type	Bearing	Bearing Spacer	Caster Assembly Single # See Note(s) A	Tire	Tube/ Insert
AT900	3"	Urethane	40117X026	1026172	1032190-NLA	N/A	N/A
41DC	4"	Urethane	N/A	N/A	1164317-NLA	N/A	N/A
51DC	5"	Urethane	N/A	N/A	1169260	N/A	N/A
6SKYWAY	6"	Urethane	40117X026	1026172	1026956	N/A	N/A
61DC	6 x 1"	Urethane	N/A	N/A	1164314	N/A	N/A
615DC	6 x 1-1/2"	Urethane	N/A	N/A	1164313	N/A	N/A

NOTE: A - Casters include Tire, Tube (if applicable), and Bearings.


6" Pneumatic Wheels with Flat-Free Inserts are No Longer Available (NLA). 6" Pneumatic Wheels are suggested as a replacement. It is recommended that they are ordered in pairs to match.

Axles & Camber Bars

Before 11/25/96

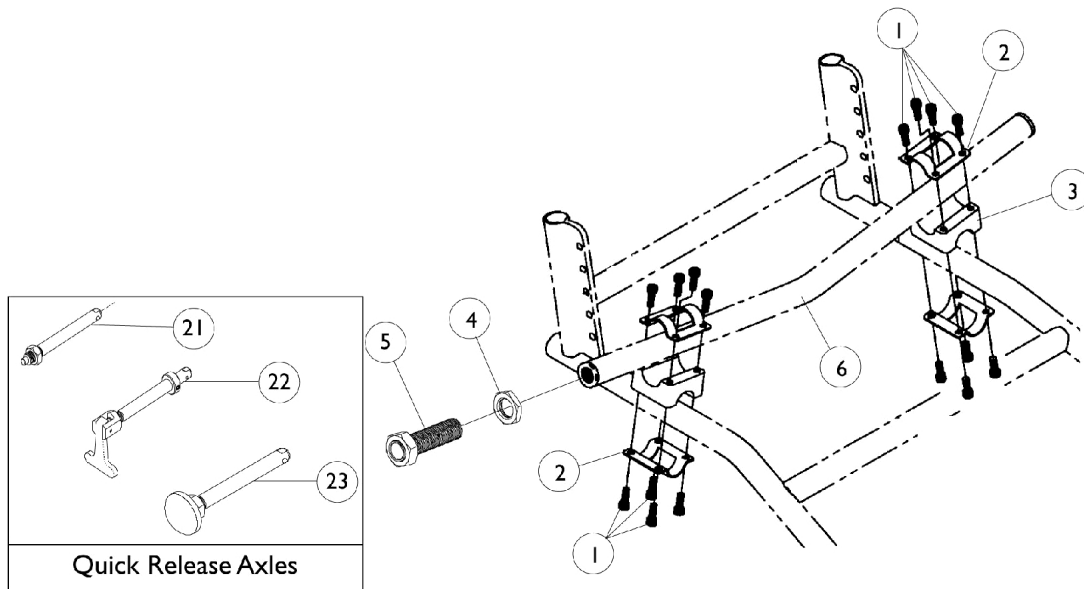

After 11/24/96

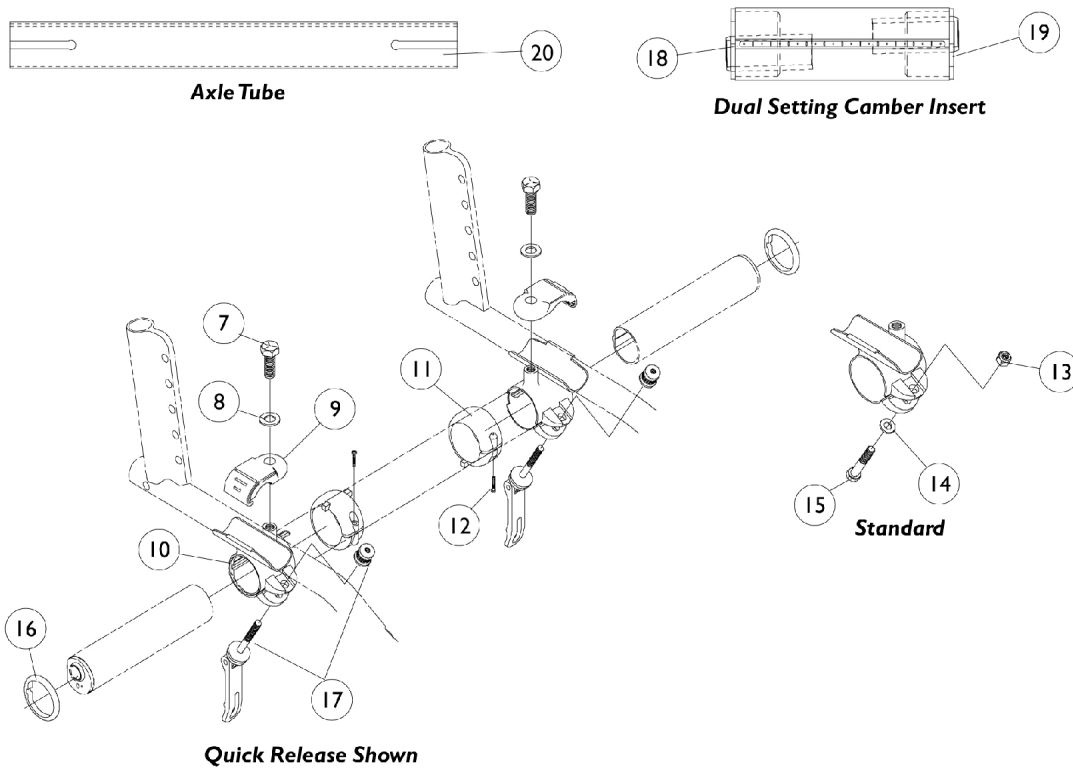

Axles & Camber Bars

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028481-NLA	NLA - Screw, Socket Head (1/4-20 x 1/2")	1	16
2		1046298-NLA	NLA - Half Clamp	1	4
3		1046235-NLA	NLA - Spacer, Camber Tube	1	2
4		1046399-NLA	NLA - Hex Jam Nut (5/8-24)	1	2
5		1048768-NLA	NLA - Bushing, Axle, Quick Release	1	2
5		1049335-NLA	NLA - Bushing, Axle, Quad Release	1	2
6		See Chart	Camber Bars	1	1
7		1065939-NLA	NLA - Screw, Hex Head (3/8-16 x 1")	1	2
8		1065940-NLA	NLA - Washer (3/8 x 3/4 x 5/64")	1	2
9		1062890-NLA	NLA - Frame, Clamp, Black	1	2
10		1062875-NLA	NLA - Clamp, Axle Tube - Right	1	1
10		1062876-NLA	NLA - Clamp, Axle Tube - Left	1	1
11		1065886	Ring, Toe-In/Toe-Out Adjusting	1	2
12		1066489-NLA	NLA - Screw, Thread Forming (M4 x 16mm)	1	2
13		1065362	Locknut (1/4-28)	1	2
14		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
15		1055347-NLA	NLA - Screw, Hex Head (1/4-28 x 1-1/2")	1	2
16		1065887-NLA	NLA - Ring, Indexing	1	2
17		1064265-NLA	NLA - Pin, Quick Release	1	2
18		1062949-NLA	NLA - Decal, Indexing	1	2
19		See Chart	Dual Setting Camber Insert	1	1
20		See Chart	Axle Tubes	1	1
21		1040361	Axle, Quick Release w/ Locknut (1/2" x 5")	1	1
22		1043487-NLA	NLA - Axle, Quad Release	1	1
23	A	1068038	NLA - Axle, Quick Release (Dome)	1	1
23	B	1113049	Axle, Quick Release w/ Dome Button, Black	1	1
23	C	1065303	Axle, Quick Release w/ Flat Button, Black	1	1
NOTE: A - Q.R. Axle (1068038) had an Anodized Dome finish. Q.R. Axle part no. 1113049, w/ Black dome button, can be used as a replacement but may want to replace in pairs due to aesthetics. B - Used on all Spoke Rear Wheels except SPIN24, SPIN25, and SPIN26 C - Used on SPIN24, SPIN25, and SPIN26 Spoke Rear Wheels					

Axles & Camber Bars

Camber Tube Hardware Kits

Kit Part Number	Description
1067258 See Note(s) A	Standard Hardware Kit
1067259 See Note(s) B	Quick Release Hardware Kit
NOTE: A - Includes items 7-16 B - Includes items 7-12,16,17	

#6 Camber Bar Before 11/25/96

Camber Angle	Part Number
0 degrees	1046237-NLA
3 degrees	1046238-NLA
6 degrees	1046239-NLA
9 degrees	1046240-NLA
12 degrees	1046241-NLA

Axles & Camber Bars

#19 Dual Setting Camber Insert

Camber Angle Dual Settings	Right	Left
0 - 3 degrees	1066219-NLA	1066229-NLA
0 - 6 degrees	1066220-NLA	1066230-NLA
0 - 9 degrees	1066221-NLA	1066231-NLA
0 - 12 degrees	1066222-NLA	1066232-NLA
3 - 6 degrees	1066223-NLA	1066233-NLA
3 - 9 degrees	1066224-NLA	1066234-NLA
3 - 12 degrees	1066225-NLA	1066235-NLA
6 - 9 degrees	1066226-NLA	1066236-NLA
6 - 12 degrees	1066227-NLA	1066237-NLA
9 - 12 degrees	1066228-NLA	1066238-NLA

#20 Axle Tubes

Chair Width					
12"	13"	14"	15"	16"	17"
1065990-NLA	1065991-NLA	1065992-NLA	1065993-NLA	1065994-NLA	1065995-NLA
18"	19"	20"	21"	22"	
1065996-NLA	1065997-NLA	1065998-NLA	1083015-NLA	1083016-NLA	

Composite Rear Wheels (without Handrims)

Wheel Style	Wheel Size	Tire Type	Wheel Single #	Tire	Tube/ Insert	Bearing	Hub Spacer
22COM	22 x 1-1/4"	Urethane	1032116	1028250	N/A	40117X022	1015756
22COM	22 x 1-3/8"	Pneumatic	1026420	1018544	1018545	40117X022	1015756
22COM	22 x 7/8"	Pneumatic w/ Flat Free	1026421	1018544	1023323	40117X022	1015756
24COM7	24 x 1-1/4"	Urethane	1032118	1026904	N/A	40117X022	1015756
24COM7	24 x 1-1/4"	KIK®	1091330	1062852	N/A	40117X022	1015756
24COM7	24 x 1-3/8"	Pneumatic	1026362-NLA	40176X060	40176X061	40117X022	1015756
24COM7	24 x 1-3/8"	Pneumatic w/ Flat Free	1026363	40176X060	1010906	40117X022	1015756
3SCF	25"	559 Primo	1064552-NLA See Note(s) A	1044947 See Note(s) A	1029471 See Note(s) A	N/A	N/A

NOTE: A - The 3 Spoke Carbon Fiber Wheel comes with the Handrim already attached.

Spoke Rear Wheels (without Handrims)

Wheel Style	Wheel Size	Tire Type	Wheel Assembly	Tire	Tube Insert	Bearing	Hub Spacer
22SP	22 x 1-3/8"	Pneumatic	1115656-NLA See Note(s) C	1018544	1018545	40117X021	1015756
22SP	22 x 1-3/8"	Pneumatic w/ Flat Free	1115658-NLA See Note(s) A, C	1018544	1023323	40117X021	1015756
24SP	24"	Continental	1116705-NLA See Note(s) C	1026139-NLA	N/A	40117X021	1015756
24SP	24"	Pneumatic	1115660 See Note(s) C	40176X060	40176X061	40117X021	1015756
24SP	24"	Pneumatic w/ Flat Free	1116707 See Note(s) A, C	40176X060	1010906	40117X021	1015756
24SP	24"	Primo	1115662-NLA See Note(s) C	1026138	1032240	40117X021	1015756
24SP	24"	KIK®	1114112 See Note(s) C	1062852	N/A	40117X021	1015756
24HPS	24"	Continental	1116713-NLA See Note(s) C	1026139-NLA	N/A	40117X021	1015756
24HPS	24"	Pneumatic	1116709-NLA See Note(s) C, E	40176X060	40176X061	40117X021	1015756
24HPS	24"	Pneumatic (Pair)	1116710	40176X060	40176X061	40117X021	1015756
24HPS	24"	Pneumatic w/ Flat Free	1116715 See Note(s) A, C	40176X060	1010906	40117X021	1015756
24HPS	24"	Primo	1116711 See Note(s) C	1026138	1032240	40117X021	1015756
24HPS	24"	KIK®	1114138 See Note(s) C	1062852	N/A	40117X021	1015756
SPIN24	24"	Primo (Single #)	1112338-NLA See Note(s) B, D	1026138	1105941	40117X021	N/A
SPIN24	24"	Primo (1 Pair)	1112353-NLA See Note(s) B, D	1026138	1105941	1104350	N/A
559TS	25"	Primo	1116717-NLA See Note(s) C	1044947	1029471	40117X021	1015756
SPIN25	25"	Primo (Single #)	1112348-NLA See Note(s) B, D	1044947	1104326	1104350	N/A
SPIN25	25"	Primo (1 Pair)	1112354-NLA See Note(s) B, D	1044947	1104326	1104350	N/A
26HPS	26"	Continental	1115653-NLA See Note(s) C	1026143-NLA	N/A	40117X021	1015756
26HPS	26"	Pneumatic	1115652-NLA See Note(s) F, C	1026142	1026140	40117X021	1015756
26HPS	26"	Pneumatic	1116725-NLA	N/A	N/A	N/A	N/A
26HPS	26"	Primo	1115654 See Note(s) C	1029470	1029471	40117X021	1015756
SPIN26	26"	Primo (Single #)	1112343-NLA See Note(s) D, B	1029470	1104326	1104350	N/A

Spoke Rear Wheels (without Handrims)

Wheel Style	Wheel Size	Tire Type	Wheel Assembly	Tire	Tube Insert	Bearing	Hub Spacer
SPIN26	26"	Primo (1 Pair)	1112355-NLA See Note(s) D, B	1029470	1104326	1104350	N/A
26SP	26"	Continental	1116721-NLA See Note(s) C	1026143-NLA	N/A	40117X021	1015756
26SP	26"	Pneumatic	1116719-NLA See Note(s) C	1026142	1026140	40117X021	1015756
26SP	26"	Primo	1116723-NLA See Note(s) C	1029470	1029471	40117X021	1015756
700SP	27"	Umma Gumma	1116727-NLA See Note(s) C	1045836-NLA	1045837-NLA	40117X021	1015756
NOTE: A - Stem Hole Plugs (Part Number 1030936) are used with Flat Free and Urethane Tires. B - As of 7/15/02 Spinergy Wheels now have 18 spokes instead of 30. For chairs built Before 7/15/02, it is recommended when one wheel is requested that the wheels be replaced in pairs. C - As of 11/08/02 these wheels now have a Black finish. For chairs built Before 11/08/02, it is recommended when one wheel is requested that the wheels be replaced in pairs. D - As of 11/17/08 Spinergy Wheels changed from black spoke, black hub to a black spoke, silver hub appearance. For chairs built Before 11/17/08, it is recommended when one wheel is requested that the wheels be replaced in pairs. E - Wheels must be replaced in pairs. See Part number 1116710 for replacement. F - No longer available as an individual service part. Wheel must be replaced in pairs, use 1116725 as its replacement.							

Spoke Rear Wheels (without Handrims)

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A	1030936	Plug, Valve Stem	1	1
	A	See Chart	Wheels without Handrims - Spoke	1	1
NOTE: A - Not Shown					

Spoke Rear Wheels (with Handrims)

Wheel Style	Tire Type	Anodized Aluminum	Grey Plastic Coated	Black Plastic Coated	8 Vertical Projection	8 Oblique Projection
22SP	Pneumatic	1115588-NLA See Note(s) B	1049452-NLA	1115591-NLA See Note(s) B	1115425-NLA See Note(s) B	N/A
22SP	Pneumatic with Flat Free	1115590-NLA See Note(s) B	1049456-NLA	1115593 See Note(s) B	1115426-NLA See Note(s) B	N/A
24SP	Pneumatic	1115594-NLA See Note(s) B	1049460-NLA	1115599 See Note(s) B	1115427-NLA See Note(s) B	1115428-NLA See Note(s) B
24SP	Pneumatic with Flat Free	1115597 See Note(s) B	1049465-NLA	1115601 See Note(s) B	1115429-NLA See Note(s) B	1115430-NLA See Note(s) B
24SP	Primo	1115602-NLA See Note(s) B	1049470-NLA	1115603-NLA See Note(s) B	1115431-NLA See Note(s) B	1115432-NLA See Note(s) B
24SP	Continental	1115606-NLA See Note(s) B	1049475-NLA	1115607-NLA See Note(s) B	1115433-NLA See Note(s) B	1115434-NLA See Note(s) B
24SP	KIK®	1115645 See Note(s) B	1091332-NLA	1115646 See Note(s) B	1115447-NLA See Note(s) B	1115448-NLA See Note(s) B
24HPS	Pneumatic	1115622 See Note(s) B	1049496-NLA	1115623-NLA See Note(s) B	1115439-NLA See Note(s) B	N/A
24HPS	Pneumatic with Flat Free	1115625-NLA See Note(s) B	1049500-NLA	1115626-NLA See Note(s) B	1115440-NLA See Note(s) B	N/A
24HPS	Primo	1115628 See Note(s) B	1049504-NLA	1115629 See Note(s) B	1115441-NLA See Note(s) B	N/A
24HPS	Continental	1115631-NLA See Note(s) B	1049508-NLA	1115632-NLA See Note(s) B	1115442-NLA See Note(s) B	N/A
24HPS	KIK®	1115649 See Note(s) B	1091337-NLA	1115650 See Note(s) B	1115449-NLA See Note(s) B	N/A
SPIN24	Primo	1112339 See Note(s) A, C	N/A	N/A	N/A	N/A
559TS	Primo	1115610 See Note(s) B	1049480-NLA	1115611-NLA See Note(s) B	1115435-NLA See Note(s) B	N/A
SPIN25	Primo	1112349-NLA See Note(s) A, C	N/A	N/A	N/A	N/A
26SP	Pneumatic	1115613-NLA See Note(s) B	1049484-NLA	1115614-NLA See Note(s) B	1115436-NLA See Note(s) B	N/A
26SP	Primo	1115616-NLA See Note(s) B	1049488-NLA	1115617-NLA See Note(s) B	1115437-NLA See Note(s) B	N/A
26SP	Continental	1115619-NLA See Note(s) B	1049492-NLA	1115620-NLA See Note(s) B	1115438-NLA See Note(s) B	N/A
26HPS	Pneumatic	1115636-NLA See Note(s) B	1049555-NLA	1115637 See Note(s) B	1115444-NLA See Note(s) B	N/A
26HPS	Primo	1115639-NLA See Note(s) B	1049559-NLA	1115640 See Note(s) B	1115445-NLA See Note(s) B	N/A
SPIN26	Primo See Note(s) A	1112344-NLA See Note(s) C, A	N/A	N/A	N/A	N/A
700SP	Umma Gumma	1115634 See Note(s) B	1049553-NLA	N/A	N/A	N/A

Spoke Rear Wheels (with Handrims)

NOTE: A - As of 7/15/02 Spinergy Wheels now have 18 spokes instead of 30. For chairs built Before 7/15/02, it is recommended when one wheel is requested that the wheels be replaced in pairs.
B - As of 11/8/02 these wheels now have a Black finish. For chairs built Before 11/8/02, it is recommended when one wheel is requested that the wheels be replaced in pairs.
C - As of 11/17/08 Spinergy Wheels changed from black spoke, black hub to a black spoke, silver hub appearance. For chairs built Before 11/17/08, it is recommended when one wheel is requested that the wheels be replaced in pairs.

When ordering a Spoke Wheel with Handrim attached, the standard handrim mounting process uses the outside holes of the handrim mounting tabs. To use the inside mounting holes to reduce the overall width of the chair, you must order the wheel and handrim assembled as a special order.

Wheels - Handrims

Wheel Style	Anodized Aluminum	Grey Plastic Coated	Black Plastic Coated	8 Vertical Projections (Grey)	8 Oblique Projections (Grey)	12 Vertical Projections (Grey)	Attachment Points
22COM	1033373	1026419-NLA See Note(s) A	1115401	1046922-NLA	N/A	N/A	7
22SP	1043443	1043499-NLA See Note(s) A	1043689-NLA	1045506-NLA	N/A	N/A	4
24COM7	1033372	1015784-NLA See Note(s) A	1115397	1015432-NLA	1015434-NLA	1015433-NLA	7
24SP	1026136	1029483-NLA See Note(s) A	1035612	1115408-NLA	1045510-NLA	N/A	6
SPIN24 (30 Spoke)	1026136	1029483-NLA See Note(s) A	1035612	1115408-NLA	N/A	N/A	6
SPIN24 (18 Spoke)	1114300	N/A	N/A	N/A	N/A	N/A	6
24HPS	1026136	1029483-NLA See Note(s) A	1035612	1115408-NLA	1045510-NLA	N/A	6
559TS	1029493	1043251-NLA See Note(s) A	1043249	1045508-NLA	N/A	N/A	6
SPIN25 (30 Spoke)	1029493	1043251-NLA See Note(s) A	1043249	1045508-NLA	N/A	N/A	6
SPIN25 (18 Spoke)	1114301-NLA	N/A	N/A	N/A	N/A	N/A	6
26SP	1026137	1029484-NLA See Note(s) A	1035613	1045509-NLA	N/A	N/A	6
26HPS	1026137	1029484-NLA See Note(s) A	1035613	1045509-NLA	N/A	N/A	6
SPIN26 (30 Spoke)	1026137	1029484-NLA See Note(s) A	1035613	1045509-NLA	N/A	N/A	6
SPIN26 (18 Spoke)	1114302-NLA See Note(s) B	N/A	N/A	N/A	N/A	N/A	6
700SP	1047086-NLA	N/A	1115413-NLA	N/A	N/A	N/A	6
NOTE: A - Grey plastic coated Handrim is no longer available; if a replacement is needed, order the black plastic coated as replacement in pairs so the Handrims on both sides of the chair match. B - Part No Longer Available individually. See Part # 1112344 for replacement.							


Wheels - Hardware - Handrims

(Hardware Kits To Attach A Single Handrim Only)

Wheel Style	Anodized Aluminum	Grey/Black Plastic Coated	8 Vertical Projections	8 Oblique Projections	12 Vertical Projections
22COM	1026384	1026383	1026380	N/A	N/A
22SP	1043708	1043708	1026910	N/A	N/A
24COM7	1026384	1026383	1026380	1026381	1026382
24SP	1026910	1026910	1026910	1026910	N/A
SPIN24	1026910	N/A	N/A	N/A	N/A
24HPS	1026910	1026910	1026910	1026910	N/A
559TS	1026910	1026910	1026910	N/A	N/A
3SCF	N/A	N/A	N/A	N/A	N/A
SPIN25	1026910	N/A	N/A	N/A	N/A
26SP	1026910	1026910	1026910	N/A	N/A
26HPS	1026910	1026910	1026910	N/A	N/A
SPIN26	1026910	N/A	N/A	N/A	N/A
700SP	1026910	N/A	N/A	N/A	N/A

High Mount Push to Lock Wheel Locks

High Mount Push to Locks

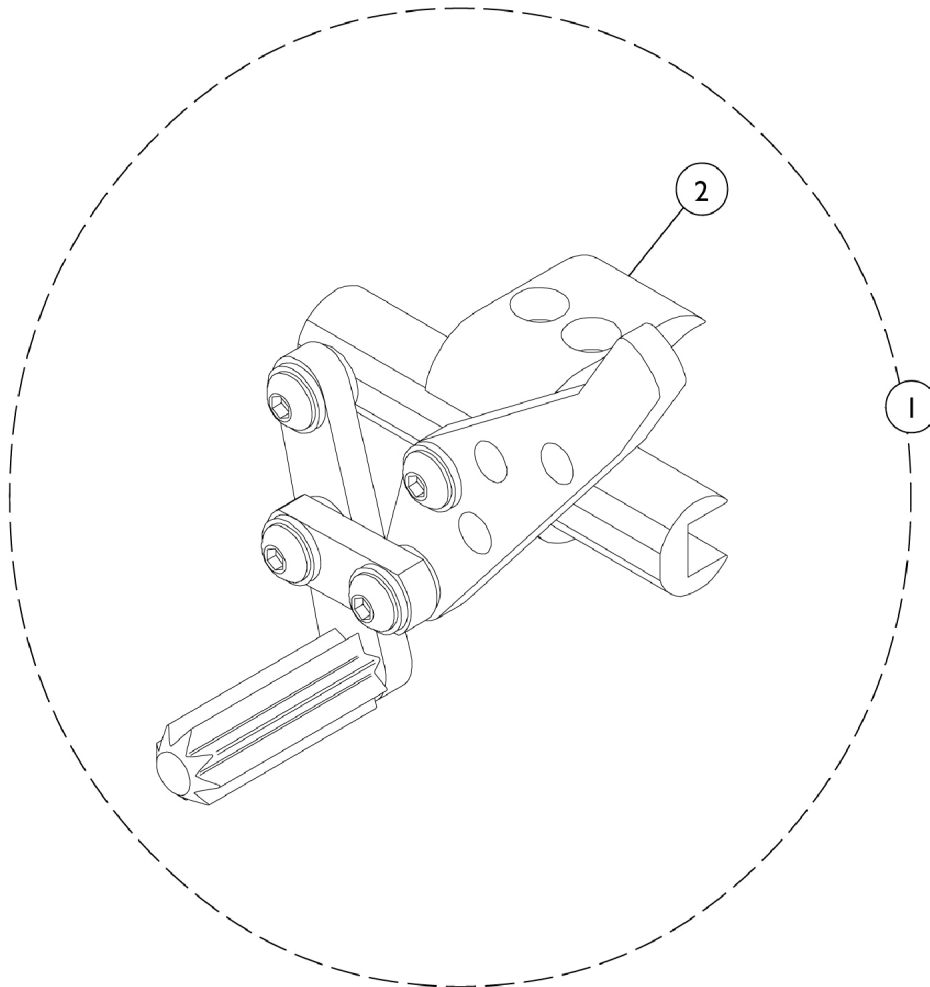

High Mount Push to Lock Wheel Locks


High Mount Push to Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1046406-NLA	NLA - Wheel Lock, High Mount, Push to Lock - Right	1	1
1	A	1046407-NLA	NLA - Wheel Lock, High Mount, Push to Lock - Left	1	1
1		1125459	Wheel Lock Assembly, Push/Pull - Right	1	1
1		1125460	Wheel Lock Assembly, Push/Pull - Left	1	1
2		1068210-NLA	Kit, Wheel Lock Attaching Hardware (1" Tubing)	1	1
			Screw, Allen Head (1/4-20 x 1")	1	2
			Clamp, Half, Non-Threaded	1	1
			Clamp, Half, Threaded	1	1
NOTE: A - No Longer Available. Order Push/Pull Wheel Locks 1125459 RT and 1125460 LT as a pair to match.					

Wheel Locks

Push/Pull to Lock



Wheel Locks can be converted from Push to Pull or Pull to Push. See Prospin X4 Owner's Manual (1125104) for instructions.

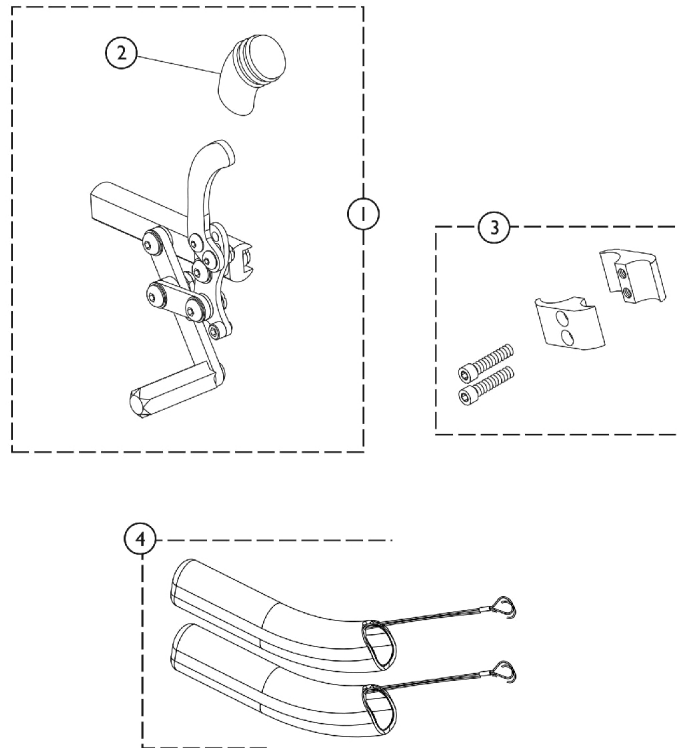

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1125459	Wheel Lock Assembly, Push/Pull - Right	1	1
1	A,C	1125460	Wheel Lock Assembly, Push/Pull - Left	1	1
1	B,C	1134129	Wheel Lock Assembly, Push/Pull for Amputee - Right	1	1
1	B,C	1134130	Wheel Lock Assembly, Push/Pull for Amputee - Left	1	1
2		1130584	Package, Cap, Brake Lever End, Black	10	1
3	D	1068210-NLA	Kit, Wheel Lock Attaching Hardware (1" Tubing)	1	1
			Screw, Allen Head (1/4-20 x 1")	1	2
			Clamp, Half, Non-Threaded	1	1
			Clamp, Half, Threaded	1	1
4		1146498	Handle, Wheel Lock Extension, Black (6") (Pair)	1	2
NOTE: A - Chairs without Amputee Attachment. Push/Pull Wheel Locks serve as replacements for High Mount Push to Lock that is No Longer Available. Order in pairs to match. B - Chairs with Amputee Attachment C - Wheel Locks can be converted from Push to Pull or Pull to Push. See Owner's Manual for instructions. D - Includes 2 ea. half clamps and 2 ea. screws					

High Mount Wheel Locks

Pull to Lock

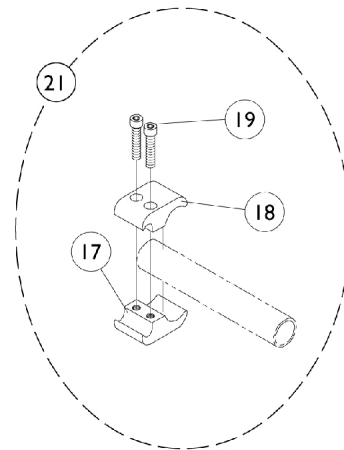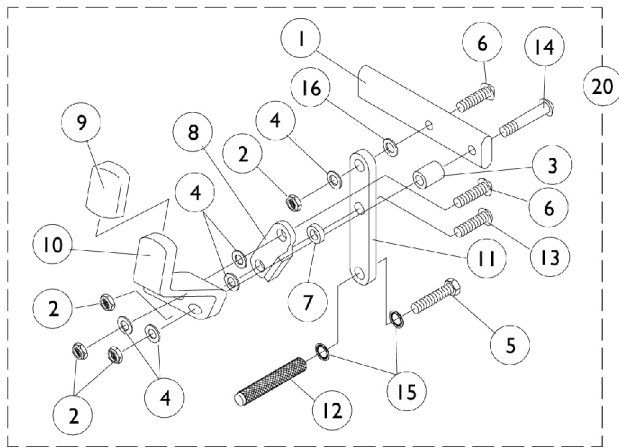


High Mount Wheel Locks

Pull to Lock

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1028399-NLA	NLA - Strap, Wheel Lock	1	1
2		1025765	Locknut (1/4-28)	1	4
3		1020363-NLA	NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
4		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	5
5	A	1055346-NLA	NLA - Screw, Hex Head (1/4-28 x 3/4")	1	1
6		1055305-NLA	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	2
7		1028411-NLA	NLA - Spacer (1/4 x 1/2 x 5/32")	1	1
8		1018733-NLA	NLA - Link, Stop, Black	1	1
9		1006490	Tip, Plastic, Black	1	2
10		1031780-NLA	NLA - Handle, Pull/ Lock, Black - Right	1	1
10		1031781-NLA	NLA - Handle, Pull/ Lock, Black - Left	1	1
11		1028416-NLA	NLA - Link, Wheel Lock, Black	1	1
12	A	1029420-NLA	NLA - Shoe, Wheel Lock	1	1
13		1055304	Screw, Socket Button Head (1/4-28 x 5/8")	1	1
14		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
15		1026108	Lockwasher (1/4 x 15/32 x 1/64")	1	2
16		1026907	Washer (1/4 x 1/2 x 1/16")	1	1
17		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
18		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
19		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
20	B	1058941-NLA	NLA - Wheel Lock Assembly, Pull to Lock - Right	1	1
20	B	1058942-NLA	NLA - Wheel Lock Assembly, Pull to Lock - Left	1	1
21	C	1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
			NLA - Clamp, Wheel Lock, Threaded (1")	1	2
			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4
NOTE: A - Order part number 1063507 (Wheel Lock Shoe Kit) for all Chairs Made Before 10/25/96. B - Includes items 1 - 16. C - Includes items 17 - 19.					


Hill Holder Wheel Locks & Attaching Hardware

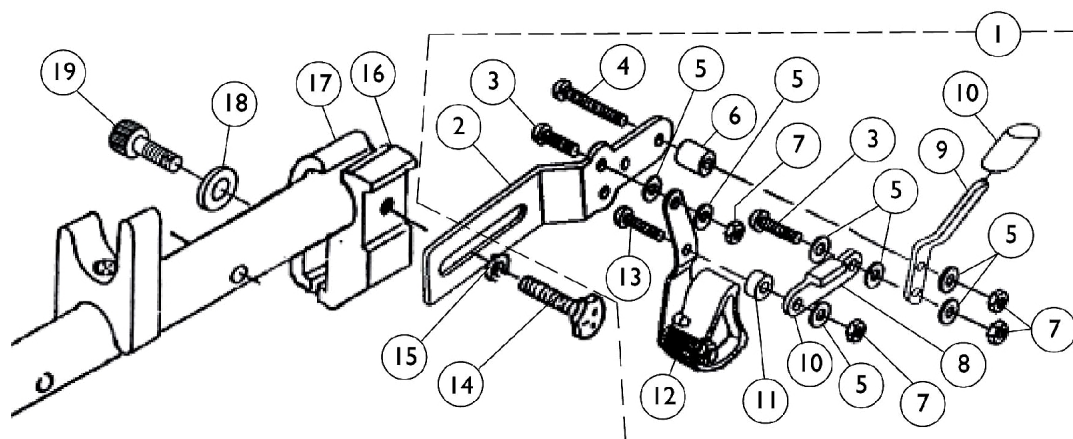

Hill Holder Wheel Locks & Attaching Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1026712	Hill Holder Wheel Lock Assembly - Right	1	1
2			Bracket, Black	1	1
3			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
4			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
5			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
6			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
7			Locknut (1/4-28)	1	4
8			NLA - Link, Stop, Black	1	1
9			NLA - Handle, Wheel Lock, Black - Right	1	1
10			Tip, Plastic, Black	1	1
11			NLA - Spacer	1	1
12			Hill Holder Assembly - Right	1	1
13			NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
1	A	1026713	Hill Holder Wheel Lock Assembly - Left	1	1
2			Bracket, Black	1	1
3			Screw, Socket Button Head (1/4-28 x 5/8")	1	2
4			NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	1
5			Washer, Nylon (1/4 x 1/2 x 1/32")	1	7
6			NLA - Spacer (1/4 x 1/2 x 1/2")	1	1
7			Locknut (1/4-28)	1	4
8			Tip, Plastic, Black	1	1
9			NLA - Handle, Wheel Lock, Black - Left	1	1
10			NLA - Link, Stop, Black	1	1
11			NLA - Spacer	1	1
12			NLA - Hill Holder Assembly - Left	1	1
13	B,C	1026352	NLA - Screw, Socket Button Head (1/4-28 x 3/4")	1	1
			Kit, Wheel Lock Hardware	2	1
14			NLA - Bolt, Hex Head Cap (1/4-20 x 3/4")	1	2
15			Washer (1/4 x 5/8 x 3/32")	1	2
16			NLA - Clamp, Half, Threaded	1	2
17			NLA - Clamp, Half, Non-Threaded	1	2
18			Washer, Nylon (1/4 x 7/16 x 5/64")	1	2
19			Screw, Socket Head (1/4-20 x 1-1/4")	1	2
NOTE: A - Includes items 2-13 B - Not Shown C - Includes items 14-19					


Undermount Wheel Locks

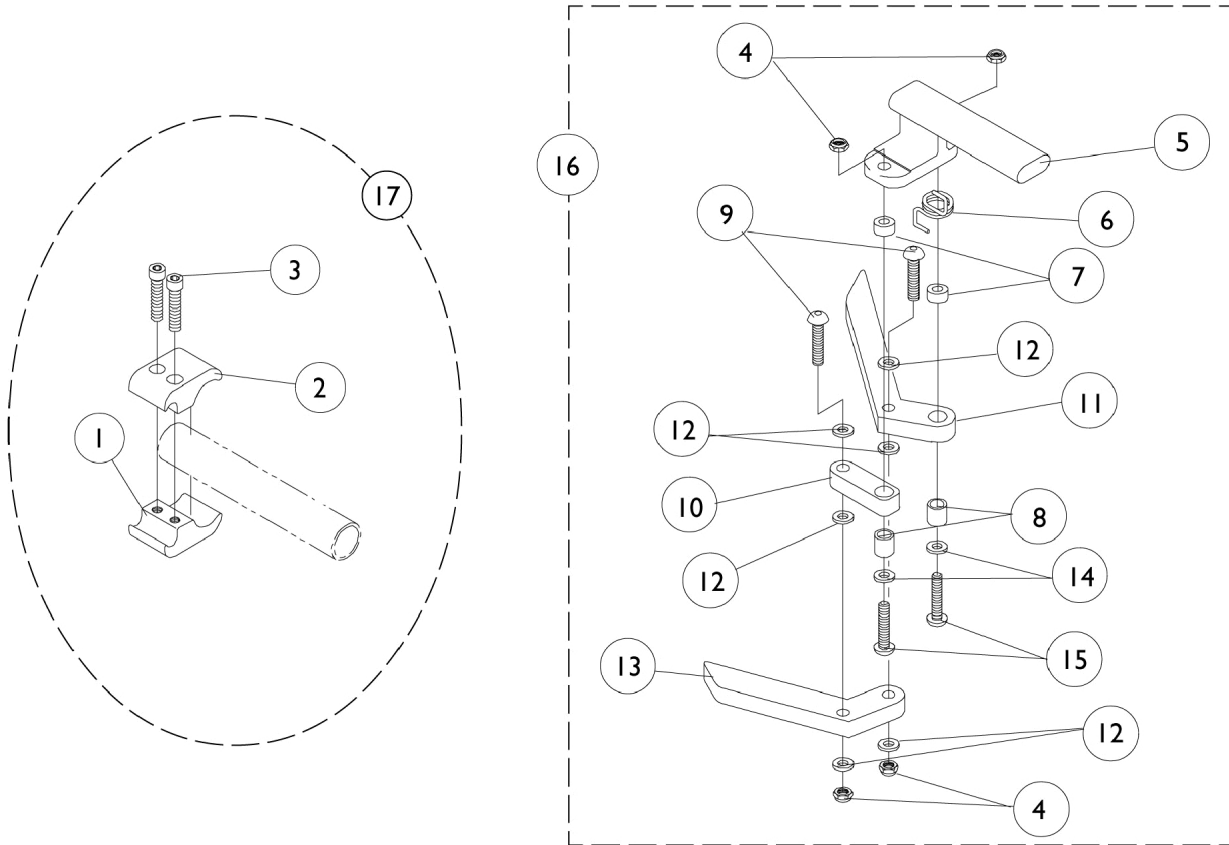

Undermount Wheel Locks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1026096-NLA	NLA - Clamp, Wheel Lock, Threaded (1")	1	1
2		1026097-NLA	NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	1
3		1027268	Screw, Allen Head (1/4-20 x 1")	1	2
4		1025765	Locknut (1/4-28)	1	4
5		1037120-NLA	NLA - Bracket, Mounting, Black - Right	1	1
5		1037121-NLA	NLA - Bracket, Mounting, Black - Left	1	1
6		1036519-NLA	NLA - Spring, Torsion - Right	1	1
6		1036520-NLA	NLA - Spring, Torsion - Left	1	1
7		1026104-NLA	NLA - Spacer	1	2
8		1005844-NLA	NLA - Bushing	1	2
9		1055307-NLA	NLA - Screw, Allen Button Head (1/4-28 x 1")	1	2
10		1025188-NLA	NLA - Link, Wheel Lock, Black	1	1
11		1024962-NLA	NLA - Handle, Wheel Lock, Black	1	1
12		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	6
13		1024961-NLA	NLA - Shoe, Wheel Lock	1	1
14		1025728	Washer (1/4 x 1/2 x 1/32")	1	2
15		1059284-NLA	NLA - Screw, Socket Button Head (1/4-28 x 1-1/4")	1	2
16	A	1036647-NLA	NLA - Wheel Lock, Undermount - Right	1	1
16	A	1036648-NLA	NLA - Wheel Lock, Undermount - Left	1	1
17	B	1039344	Kit, Wheel Lock Attaching Hardware (1" tubing)	1	1
			NLA - Clamp, Wheel Lock, Threaded (1")	1	2
			NLA - Clamp, Wheel Lock, Non-Threaded (1")	1	2
			Screw, Allen Head (1/4-20 x 1")	1	4
NOTE: A - Includes items 4 - 15. B - Includes items 1 - 3.					

Adjustable Angle Footplate Assemblies - AT5543

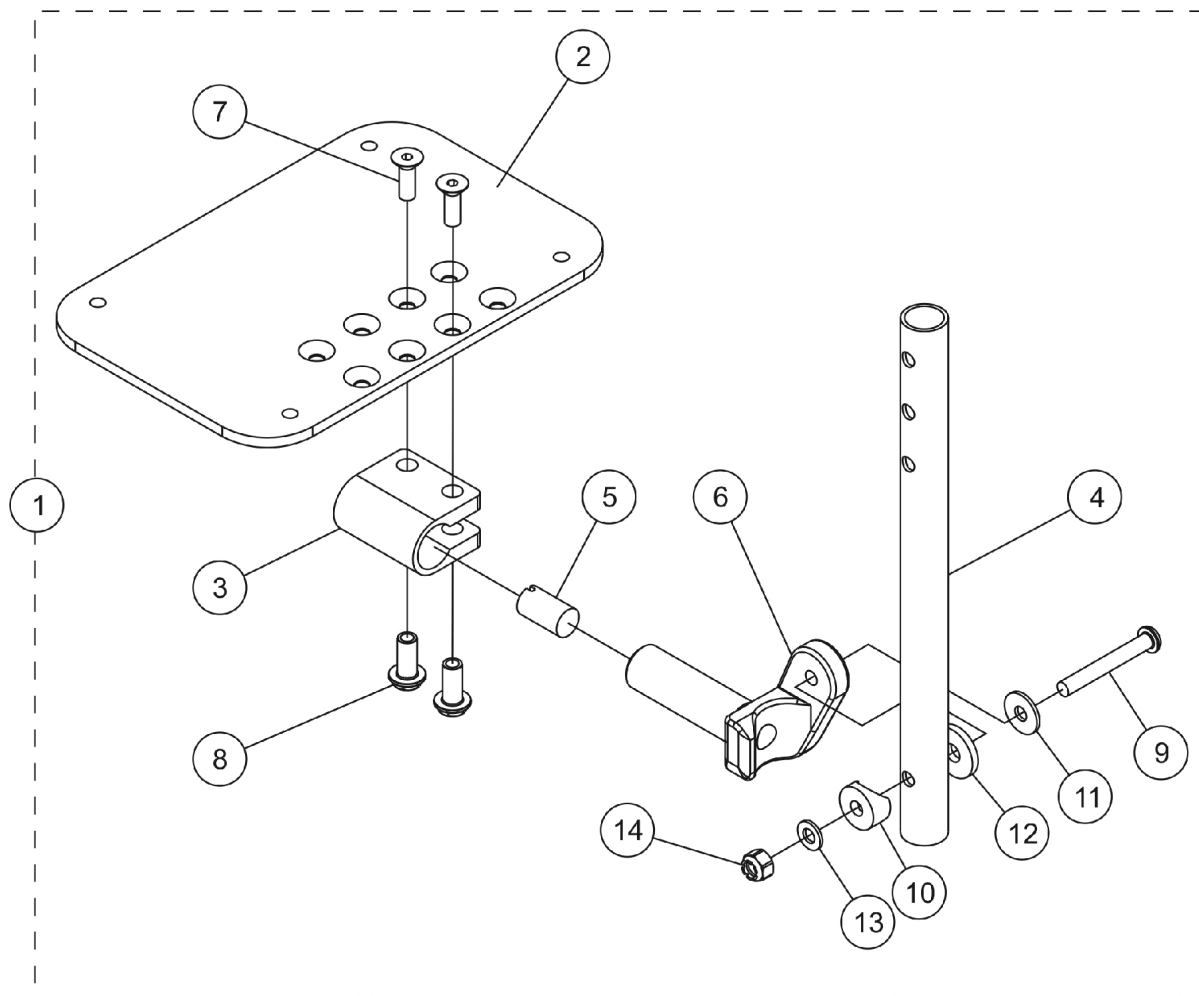

Adjustable Angle Footplate Assemblies - AT5543

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	D	See Chart	Adjustable Angle Footplate Assembly	1	1
2	E	1113571	Footplate, Adjustable Angle, Small, Black (3-1/4" W x 8" L)	1	1
2	E	1113572	Footplate, Adjustable Angle, Medium, Black (4-1/4" W x 8" L)	1	1
2	E	1113573	Footplate, Adjustable Angle, Large, Black (5-1/4" W x 8" L)	1	1
2	E	1113574	Footplate, Adjustable Angle, Extra Large, Black (6-1/4" W x 8" L)	1	1
3	A	1058517	Clamp, Footplate	1	1
4	B	1061967	Tube, Pivot/Slide, Straight, Black (5-1/2")	1	1
4	C	1069350-NLA	NLA - Tube, Pivot/Slide, Curved V Front, Black (5-1/2")	1	1
5		1048816	Screw, Nylon Adjustment (1/2-13 x 7/8")	1	1
6		1052102	Hinge, Footplate - Right	1	1
6		1052103	Hinge, Footplate - Left	1	1
7	A	1029545	Screw, Socket Flat Head (1/4-20 x 3/4")	1	2
8	A	1082686	Nut, Barrel, Hex Head (1/4-20 x 11/16")	1	2
9		1022831	Screw, Socket Button Head (1/4-20 x 1-7/8")	1	1
10		1025457	Spacer, Coved (1/4 x 3/4 x 11/32")	1	1
11		1026900	Washer (1/4 x 23/32 x 1/16")	1	1
12		1016301	Washer, Nylon (5/16 x 7/8 x 1/8")	1	1
14		1025736	Locknut (1/4-20)	1	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	1


NOTE: A - Items 3, 7 and 8 must be ordered together for chairs manufactured before 8/10/99 due to a hardware change.
 B - Tapered Front Frame
 C - "V" Front Frame
 D - Includes Items 2 thru 14.
 E - This footplate was revised 9/2/2008 with rounder corners and accessory holes changed from 8 to 4. Recommended to replace footplates in pairs to match sides.

#1 Adjustable Angle Footplate Assembly

Chair Width	Tapered Front Frame -Right	Tapered Front Frame -Left	"V" Front Frame -Right	"V" Front Frame -Left
14-15" Wide (3-1/4" Footplate)	1061974-NLA	1061975-NLA	1069351-NLA	1069352-NLA
16-17" Wide (4-1/4" Footplate)	1061976-NLA See Note(s) A	1061977-NLA See Note(s) A	1069353-NLA	1069354-NLA
18-19" Wide (5-1/4" Footplate)	1061978-NLA	1061979-NLA	1069355-NLA	1069356-NLA
20" Wide (6-1/4" Footplate)	1062586	1062587	1069355-NLA	1069356-NLA

NOTE: A - Adjustable Angle Footplates 1061976 Rt./ 1061977 Lt. are NLA. Order 1060436 Rt./1060437 Lt. with 1061967 and have assembled.

Rigid Tubular Footplate Tapered Front Only

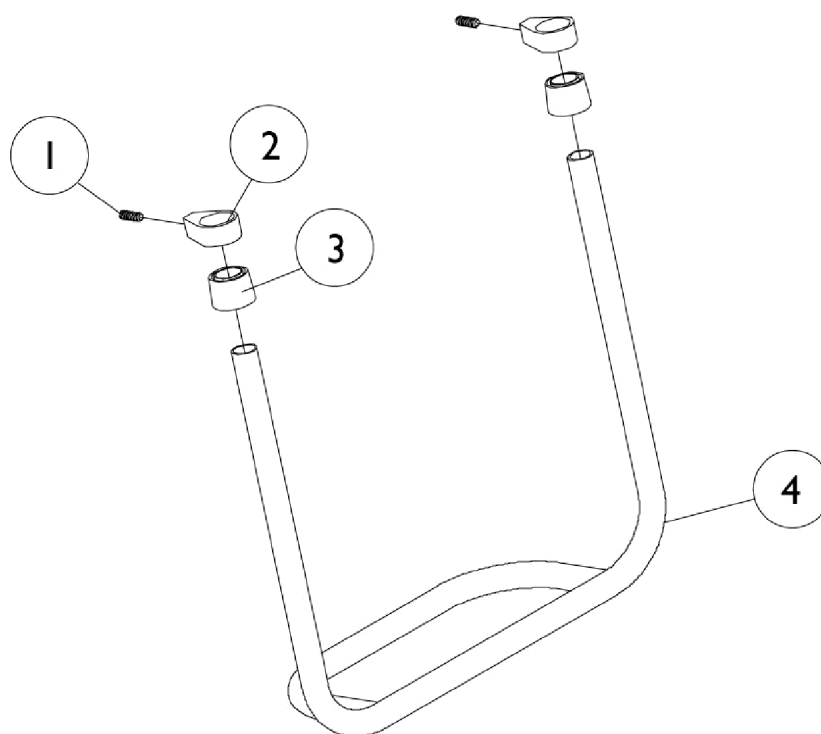

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1026911-NLA	NLA - Screw, Socket Set Cone Point (1/4-20 x 1/2")	1	2
2		1024749	Clamp	1	2
3		1026124	Collar, Anti-Rattle	1	2
4		See Chart	#4 Rigid Tubular Footplate	1	1


Rigid Tubular Footplate

Tapered Front Only

#4 Rigid Tubular Footplates

Chair Width	Part Number
14"	1043272-NLA
15"	1043273-NLA
16"	1043274-NLA
17"	1043275-NLA
18"	1043276-NLA
19"	1043277-NLA
20"	1043278-NLA
21"	1082974
22"	1082980

One Piece Adjustable Angle Footplate

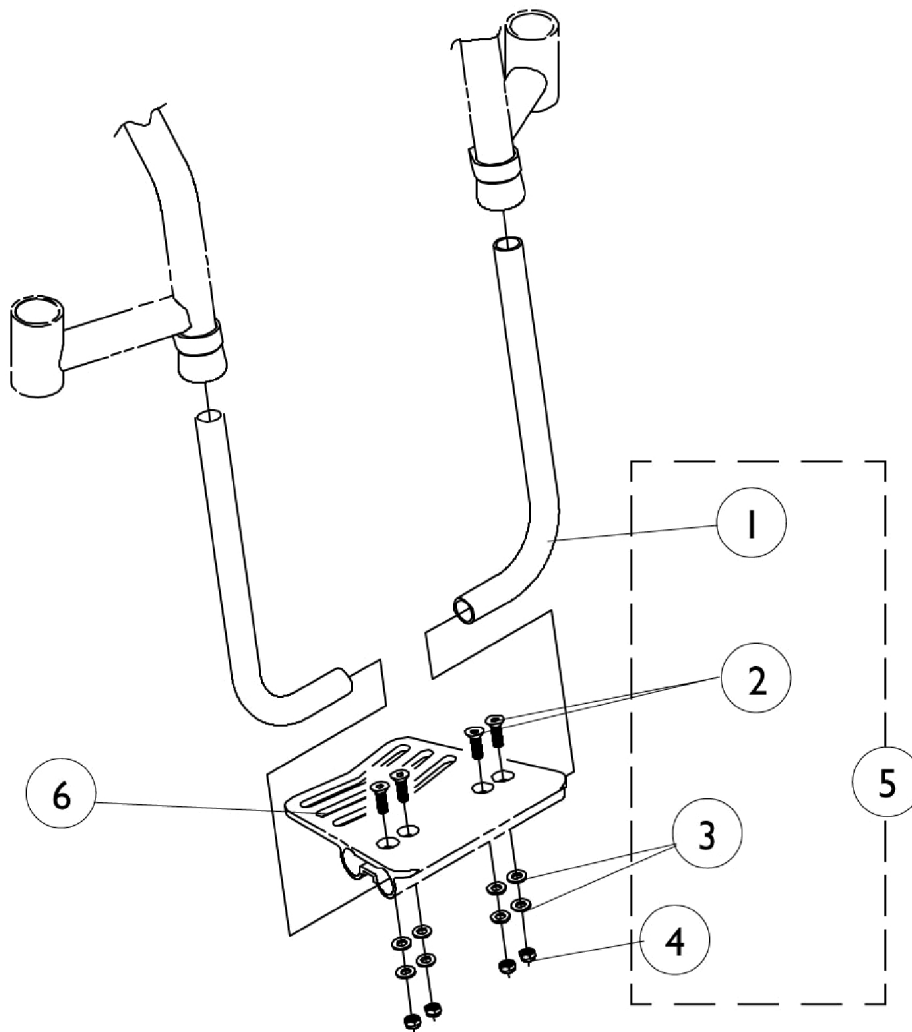



One Piece Adjustable Angle Footplate

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1046409-NLA	NLA - Tube, Support, Footplate, Black	1	2
1	B	1056296-NLA	NLA - Support Tube, Footplate	1	2
2		1046421-NLA	NLA - Screw, Socket Flat Head (1/4-28 x 1")	1	4
3		1013971	Washer, Nylon (1/4 x 7/16 x 5/64")	1	8
4		1025765	Locknut (1/4-28)	1	4
5	A,C	1046420-NLA	NLA - Hardware, Footplate Mounting	1	1
5	B,C	1058667-NLA	NLA - Hardware, Footplate Mounting	1	1
6		See Chart	#6 Footplate - One Piece Adjustable Angle	1	1
NOTE: A - Tapered Front Frame B - "V" Front Frame C - Includes items 1-4					

#6 One Piece Adjustable Angle Footplates

Chair Width	Tapered Front Frame	"V" Front Frame
12-13"	1046460-NLA	1046461-NLA
14"	1046461-NLA	1046462
15"	1046462	1046463-NLA
16"	1046463-NLA	1046464-NLA
17"	1046464-NLA	1046465-NLA
18"	1046465-NLA	1046466
19-22"	1046466	1046466

Rigid Side Guards

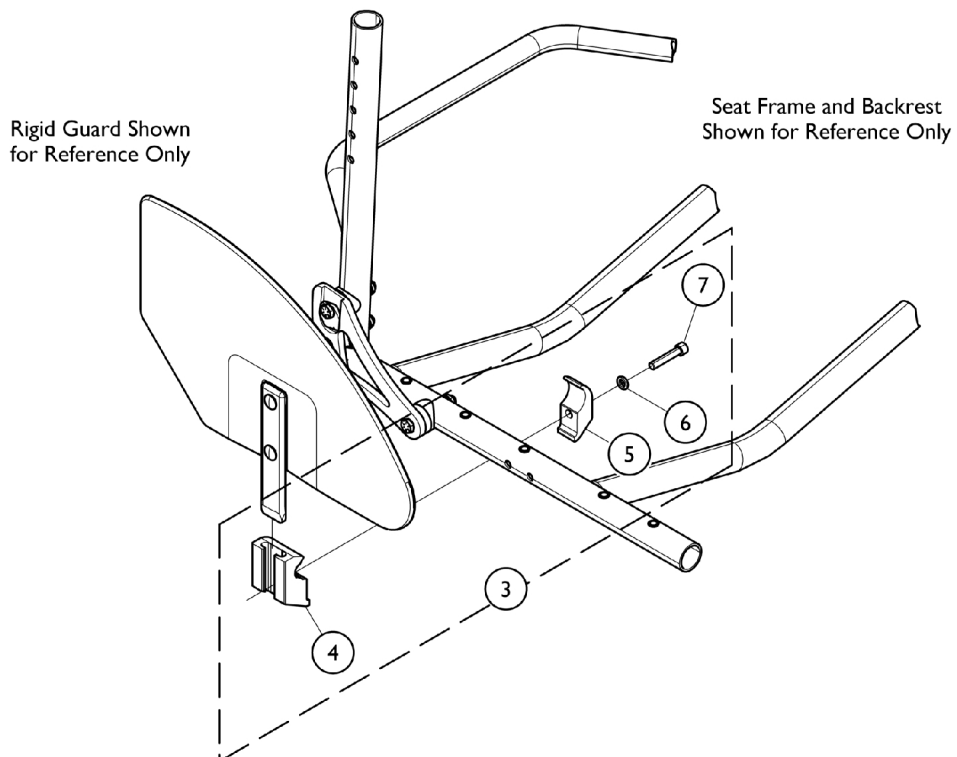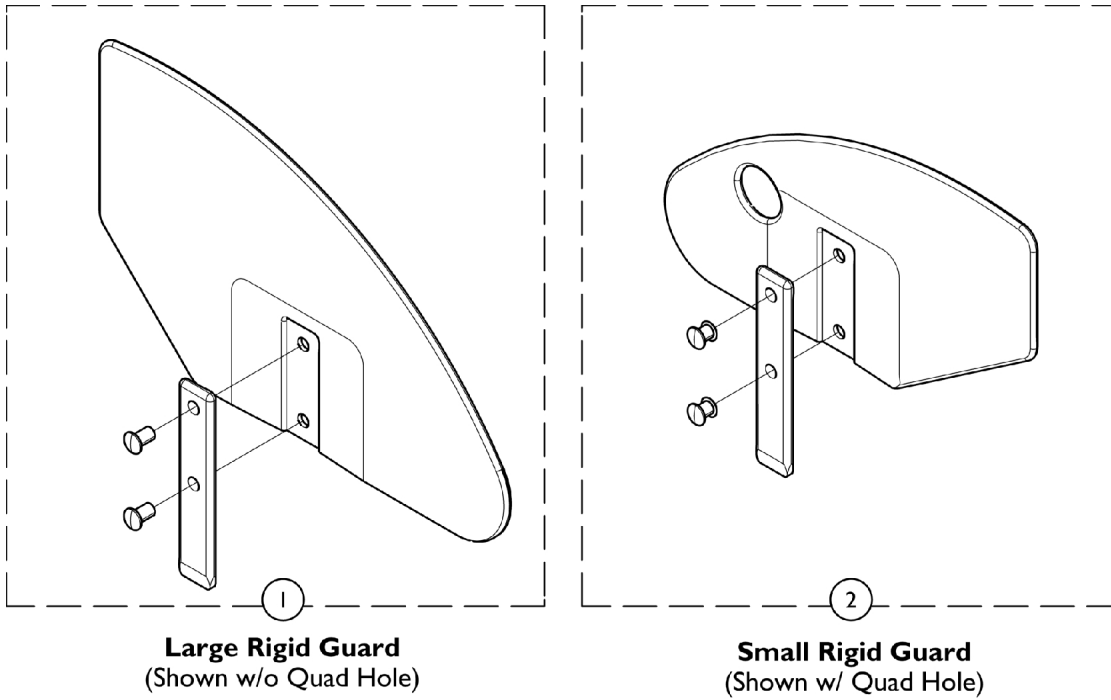


Rigid Side Guards


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1036344	Guard, Side, Large w/o Quad Hole - Right (7-3/4" H x 12-3/8" D)	1	1
1		1036345	Guard, Side, Large w/o Quad Hole - Left (7-3/4" H x 12-3/8" D)	1	1
1		1036405	Guard, Side, Large w/ Quad Hole - Right (7-3/4" H x 12-3/8" D)	1	1
1		1036406	Guard, Side, Large w/ Quad Hole - Left (7-3/4" H x 12-3/8" D)	1	1
2		1036407	Guard, Side, Small w/ Quad Hole - Right (6-1/4" H x 11-1/4" D)	1	1
2		1036408	Guard, Side, Small w/ Quad Hole - Left (6-1/4" H x 11-1/4" D)	1	1
2		1036346	Guard, Side, Small w/o Quad Hole - Right (6-1/4" H x 11-1/4" D)	1	1
2		1036347	Guard, Side, Small w/o Quad Hole - Left (6-1/4" H x 11-1/4" D)	1	1
3		1036342	Kit, Side Guard Hardware (Pair)	2	1
4		1032249	Clamp, Half, Threaded	1	1
5		1018692-NLA	NLA - Clamp, Half, Non-Threaded	1	1
6		1013971	Washer, Nylon (1/4 x 7/16 x 5/64")	1	1
7		1003545	Screw, Socket Head (1/4-20 x 1-1/4")	1	1
NOTE: A - Includes items 4-7					

Push Handles

Bolt On and Quick Release

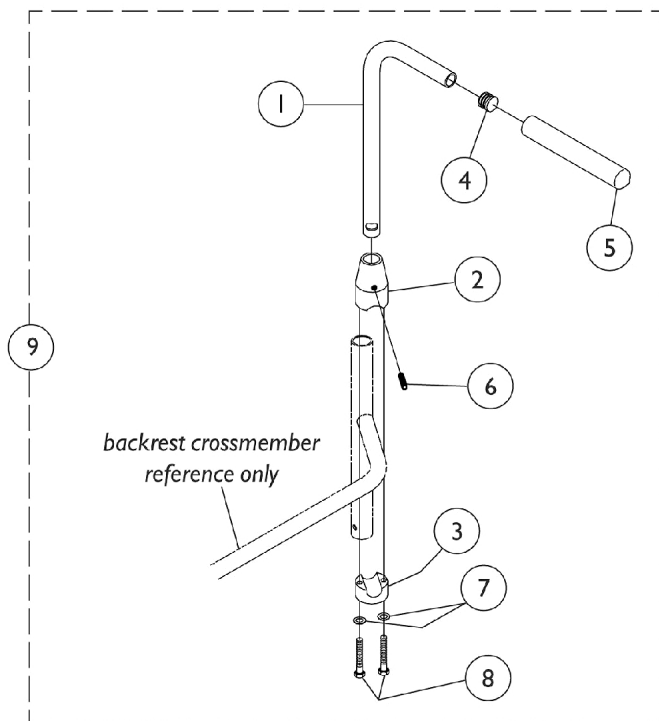

Bolt On Push Handle

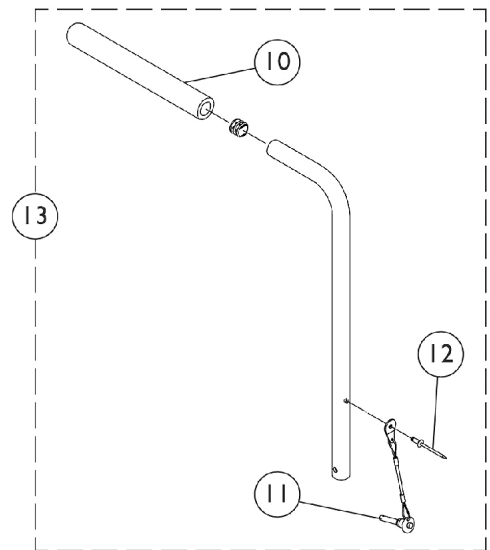

Quick Release Push Handle

(Not available on chairs with AT99 arms)

Push Handles

Bolt On and Quick Release

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1032209-NLA	NLA - Handle, Push, Black - Right	1	1
1		1032210-NLA	NLA - Handle, Push, Black - Left	1	1
2		1032211-NLA	NLA - Clamp, Threaded, Top	1	1
3		1032212-NLA	NLA - Clamp, Non-Threaded, Bottom	1	1
4		1027097	Plug Button, Black (3/4")	1	1
5		1028431-NLA	NLA - Grip, Foam, Black (5")	1	1
6		1032217-NLA	NLA - Screw, Set Flat Point (1/4-20 x 1")	1	1
7		1026907	Washer (1/4 x 1/2 x 1/16")	1	2
8		1025713-NLA	NLA - Screw, Hex Head (1/4-20 x 1")	1	2
9	A	1032213-NLA	NLA - Push Handle Assembly w/ Hardware, Bolt On (Pair)	2	1
10		1006229	Grip, Foam (8")	1	1
11		1045470	Pin, Quick Release with Lanyard	1	1
12		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	1
13	B	1046290-NLA	NLA - Handle, Push with Quick Release Pin, Black	1	1
NOTE: A - Includes items 1-8 B - Includes Push Handle and items 10-12					

Miscellaneous Accessories

Part Number	Description	Package Quantity
1046487-NLA	Amputee Kit	1
1046425 See Note(s) A	Anti-Tipper	1
1109322-NLA	Back Pack (12-15" Wide) 11" W x 13" H x 4" D	1
1109323	Back Pack (16-22" Wide) 14" W x 12" H x 4" D	1
1046282-NLA	Basketball Roller - Adj. Angle Footboard	1
1029419-NLA	Basketball Roller - One Piece Tubular Footplate	1
1027644-NLA	Calf Strap - 12-15" Wide (26" Long)	1
1032243	Calf Strap - 16-22" Wide (36" Long)	1
1068210	Clamp with Hardware, 3/4" to 1"	1
1028452-NLA	Fabric Clothing Protector - 14-15" D, Right	1
1028453-NLA	Fabric Clothing Protector - 14-15" D, Left	1
1028454-NLA	Fabric Clothing Protector - 16-22" D, Right	1
1028455-NLA	Fabric Clothing Protector - 16-22" D, Left	1
1026846-NLA See Note(s) B	Impact Guard	1
1004310	Positioning Strap, Auto Style 12-19" W	1
1034625	Positioning Strap, Auto Style 20-22" W	1
1034744	Positioning Strap, Hook & Loop 12-19" Wide	1
1034745	Positioning Strap, Hook & Loop 20-22" Wide	1
1109324	Seat Pouch 8-1/2" W x 7" H x 4" D	1
1032147-NLA	Spoke Guard - 22" Spoke/Comp. Wheels	1
1028442-NLA	Spoke Guard -24" Composite Wheels	1
1028441-NLA	Spoke Guard -24" Spoke Wheel	1
1028443-NLA	Spoke Guard - 25"/26" Spoke/Comp. Wheels	1
40176X010-NLA	Tire Pump	1
1163258-NLA	Kit, Wheel Lock Extension Handle w/ Instruction Sheet (6") (Pair)	2

Miscellaneous Accessories

NOTE: A - Anti-tipper includes snap button fastener
B - For Rigid Tubular Style Footrest



Invacare Corporation

www.invacare.com

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Customer Service
800-333-6900

© 2022 Invacare Corporation. Elyria, Ohio.
Invacare, Yes, you can., A-4, and the
Medallion Design are registered trademarks
of Invacare Corporation. KIK is a registered
trademark of KIK Technology, Inc. (Revised
06/17/22)

Form No. 94-162