


Lateral Trunk Support

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

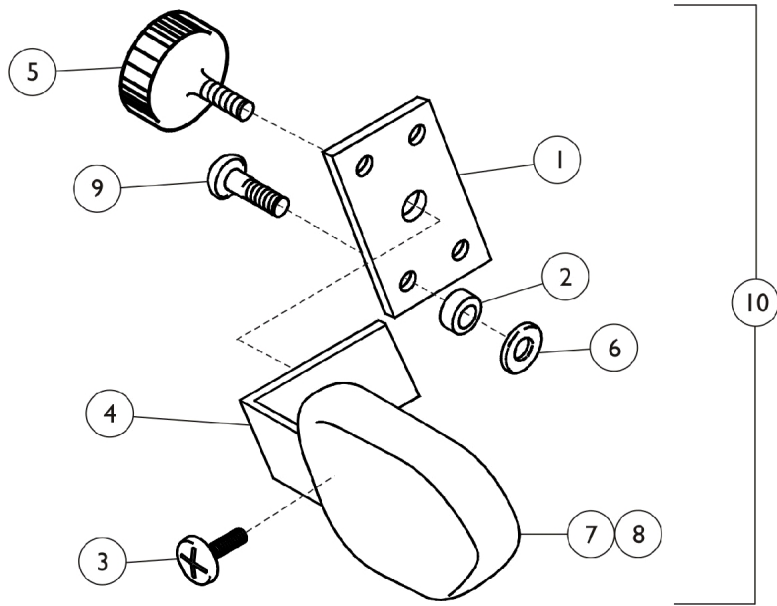

Lateral Trunk Support

THE PRODUCTION OF THE HTR IS CURRENTLY SUSPENDED AND SOME SERVICE PARTS MAY NOT BE AVAILABLE TO ORDER.

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	A,C,E	1129626	Kit, Lateral Hardware - Grey	1	1
	A,B,D	HRA20K	Kit, Lateral Hardware - Chrome	1	1
1			Plate, Headwing - Chrome	1	1
2			Spacer, Nylon (3/16 x 5/16 x 5/16")	1	4
3			NLA - Screw (#10-24 x 7/8")	1	6
4			Bracket, "L", Chrome	1	1
5			Knob, Adjustment	1	1
6		1013218	Washer (#10 Flat)	1	4
7		450110-25-NLA	Cushion and Cover, Mauve - Right	1	1
7		450111-25-NLA	Cushion and Cover, Mauve - Left	1	1
7		450110-26-NLA	NLA - Cushion and Cover, Grotto - Right	1	1
7		450111-26-NLA	Cushion and Cover, Grotto - Left	1	1
7		450110-27-NLA	Cushion and Cover, Blueberry - Right	1	1
7		450111-27-NLA	Cushion and Cover, Blueberry - Left	1	1
8		450117-25	Cover, Lateral Support, Mauve	1	1
8		450117-26	Cover, Lateral Support, Grotto	1	1
8		450117-27-NLA	NLA - Cover, Lateral Support, Blueberry	1	1
9		1108131-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 1")	1	4
9		1108193-NLA	NLA - Screw, Phillips Pan Head (#10-24 x 3/4")	1	4
10	D	1072292	Lateral Trunk Support Assembly, Mauve - Chrome	2	1
10	E	1129604	Lateral Trunk Support Assembly, Mauve - Grey	2	1
10	D	1072299	Lateral Trunk Support Assembly, Grotto - Chrome	2	1
10	E	1129609	Lateral Trunk Support Assembly, Grotto - Grey	2	1
10	D	1072300	Lateral Trunk Support Assembly, Blueberry - Chrome	2	1
10	E	1129610	Lateral Trunk Support Assembly, Blueberry - Grey	2	1

NOTE: A - Not Shown.
 B - Kit Includes Items 1-5. To be used on Chairs with Chrome Finish.
 C - Kit Includes Grey Finish Plate, Spacers, Screw, Knob and Grey Finish Bracket.
 D - Before 1/24/05. For use on chairs with "Chrome Frame Finish."
 E - After 1/23/05. For use on chairs with "Grey Frame Finish."