



Captain Seat Assembly (Before 9/1/02)

NOTE: Low Back (LB.) Captain Seat description changed to Extended Back (E.B.) Captain Seat as of 10/02/00.


NOTE: Captain Seats incorporate an assisted recline of approximately 90 deg.- 114 deg. for extended backs and 90 deg. - 170 deg. for high back 18 " wide models.

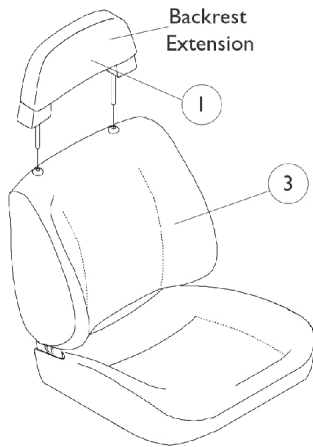

Extended Back 18" Wide

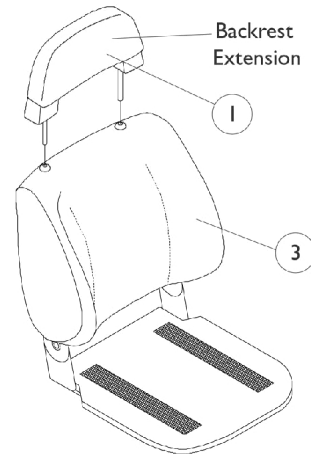


Extended Back, Forward Folding 18" Wide w/ Solid Seat Base

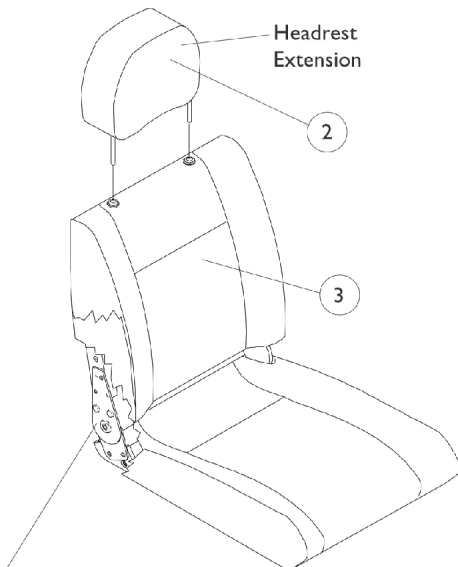

NOTE: High back 18" wide captain seat includes the actuator hardware. See seat frame and actuator hardware page.

High Back 18" Wide w/ Full Recline

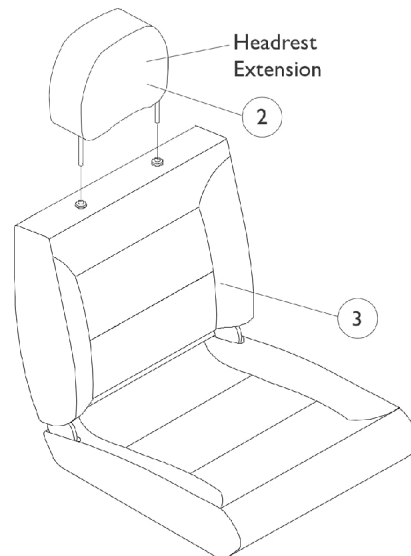

High Back 22" Wide w/ Semi Recline

Captain Seat Assembly

(Before 9/1/02)

NOTE: Low Back (LB.) Captain Seat description changed to Extended Back (E.B.) Captain Seat as of 10/02/00.

NOTE: Captain Seats incorporate an assisted recline of approximately 90 deg.- 114 deg. for extended backs and 90 deg. - 170 deg. for high back 18 " wide models.

Item Number	Description	Grey Fabric Vinyl	All Grey Vinyl	Brunswick Buff All Tan Vinyl
1.	Backrest, Extended Back (E.B.) (Before 3/25/98)	1071578-NLA See Note(s) A	N/A	N/A
1.	Backrest, Extended Back (3/25/98-8/3/99) and (E.B. Forward Folding Before 5/12/00)	1078997-NLA See Note(s) B	N/A	N/A
1.	Backrest, Extended Back (After 8/3/99) and (E.B. Forward Folding after 5/11/00)	1090374-NLA See Note(s) C	N/A	N/A
1.	Backrest, Extended Back (Before 7/22/99)	N/A	1081285-NLA See Note(s) B	1086109-NLA See Note(s) B
1.	Backrest, Extended Back (After 7/21/99)	N/A	1090375-NLA See Note(s) C	1090376-NLA See Note(s) C
2.	Headrest, High Back (Before 8/16/99)	1071576-NLA See Note(s) D	N/A	N/A
2.	Headrest, High Back (After 8/15/99) (18" and 22" Wide)	1090379-NLA See Note(s) C	N/A	N/A
2.	Headrest, High Back	N/A	1090380-NLA See Note(s) C	N/A
3.	18" Wide Extended Back (Before 8/4/99) w/Mtg. Screws	1091124-NLA	N/A	N/A
3.	18" Wide Extended Back (After 8/3/99)	1090371-NLA	N/A	N/A
3.	18" Wide Extended Back (Before 7/22/99)	N/A	1081284-NLA	1086108-NLA
3.	18" Wide Extended Back (After 7/21/99)	N/A	1090372-NLA	1090373-NLA
3.	18" Wide Extended Back Forward Folding (Before 5/12/00)	1085104-NLA	N/A	N/A

Captain Seat Assembly

(Before 9/1/02)

NOTE: Low Back (LB.) Captain Seat description changed to Extended Back (E.B.) Captain Seat as of 10/02/00.

NOTE: Captain Seats incorporate an assisted recline of approximately 90 deg.- 114 deg. for extended backs and 90 deg. - 170 deg. for high back 18 " wide models.

Item Number	Description	Grey Fabric Vinyl	All Grey Vinyl	Brunswick Buff All Tan Vinyl
3.	18" Wide Extended Back Forward Folding (After 5/11/00)	1090370-NLA	N/A	N/A
3.	18" Wide High Back (Before 8/16/99)	1070456-NLA See Note(s) E	N/A	N/A
3.	18" Wide High Back (After 8/15/99)	1091068-NLA See Note(s) E	N/A	N/A
3.	18" Wide High Back with Full Recline	N/A	1091069-NLA	N/A
3.	22" Wide High Back with Semi Recline	1098287-NLA	N/A	N/A

NOTE: A - 9-29/64" Backrest Post Spacing
 B - 9" Backrest Post Spacing
 C - 4-3/4" Backrest/Headrest Post Spacing
 D - 4-23/32" Headrest Post Spacing
 E - High back 18" wide captain seats includes all actuator attaching hardware.

NOTE: When converting between high back and extended back captain seats, the recliner wiring harness with sensor cable must be ordered as well.

NOTE: For captain seats marked NLA, use captain seats dated after 8/31/02 as replacements.