

P7E™ **Power Wheelchair**



Yes, you can.

Parts Catalog Usage Guide

The information contained in this document is subject to change without notice. This catalog contains service parts for Form No. 95-167. If you have any questions regarding this catalog, please contact one of our Customer Service Representatives.

For Customer Service Support Call:

1-800-333-6900

For Faxed In Orders Call:

1-800-678-4682 or

1-440-366-0709

For Technical Service Call:

1-800-832-4707

Literature Can Be Found at:

www.invacare.com or

1-800-828-6272

SYMBOL DEFINITION

Throughout this catalog you will notice letters to draw your attention to specific footnotes which are located at the bottom of the page or below specified charts.

Examples:

A,B,C etc. = Special Comments

NOTE: = General information pertaining to that specific page

NLA = No Longer Available

HARDWARE MEASUREMENTS

The measurements for the hardware will be listed as follows:

Screws, Bolts Diameter X Thread Count/Pitch X Length

Washers Inside Diameter X Outside Diameter X Thickness

Spacers Inside Diameter X Outside Diameter X Length

For your convenience, we have supplied an area for you to list your Invacare Representatives.

Name	Phone Numbers

© 2022 Invacare Corporation. Elyria, Ohio. Invacare, Yes, you can., and the Medallion Design are registered trademarks and P7E and MKIV are trademarks of Invacare Corporation. Bussman is a registered trademark of Cooper Industries. MIDI is a registered trademark of Littelfuse Inc., Dynamic is a registered trademark of Dynamic Controls Limited.


TABLE of CONTENTS

Description	Page Number
Accessories - Headrest Extension - Upholstery & Hardware	13
Accessories - Miscellaneous	43
Accessories - Power Wheelchair Tool Kit	44
Arms - Adjustable Height	17
Arms - Fixed Height	15
Battery Boxes - Chargers - Harnesses - Trays	27
Casters & Forks	19
Crossbraces - Hardware - Seat Extensions	9
Electronics - INT. Controller	41
Electronics - MKIV RII Joystick, Controller	38
Frame Finishes - Touch-Up Paints - Upholstery Color	6
Frames - Back Canes & Hardware	7
Front Riggings - Footrest - P93	30
Front Riggings - Footrest - PW93	32
Front Riggings - Legrest - P904A/T934A	34
Front Riggings - Pivot/Slide Tube & Footplate - P904 and T934A	37
Motor Gearbox Assembly	25
Packaging - Literature - Decals	4
Upholstery - Back & Seat	11
Warranty Replacement Parts	46
Wheel Locks	23

TABLE of CONTENTS

Description	Page Number
Wheels - Rear	21

Packaging - Literature - Decals

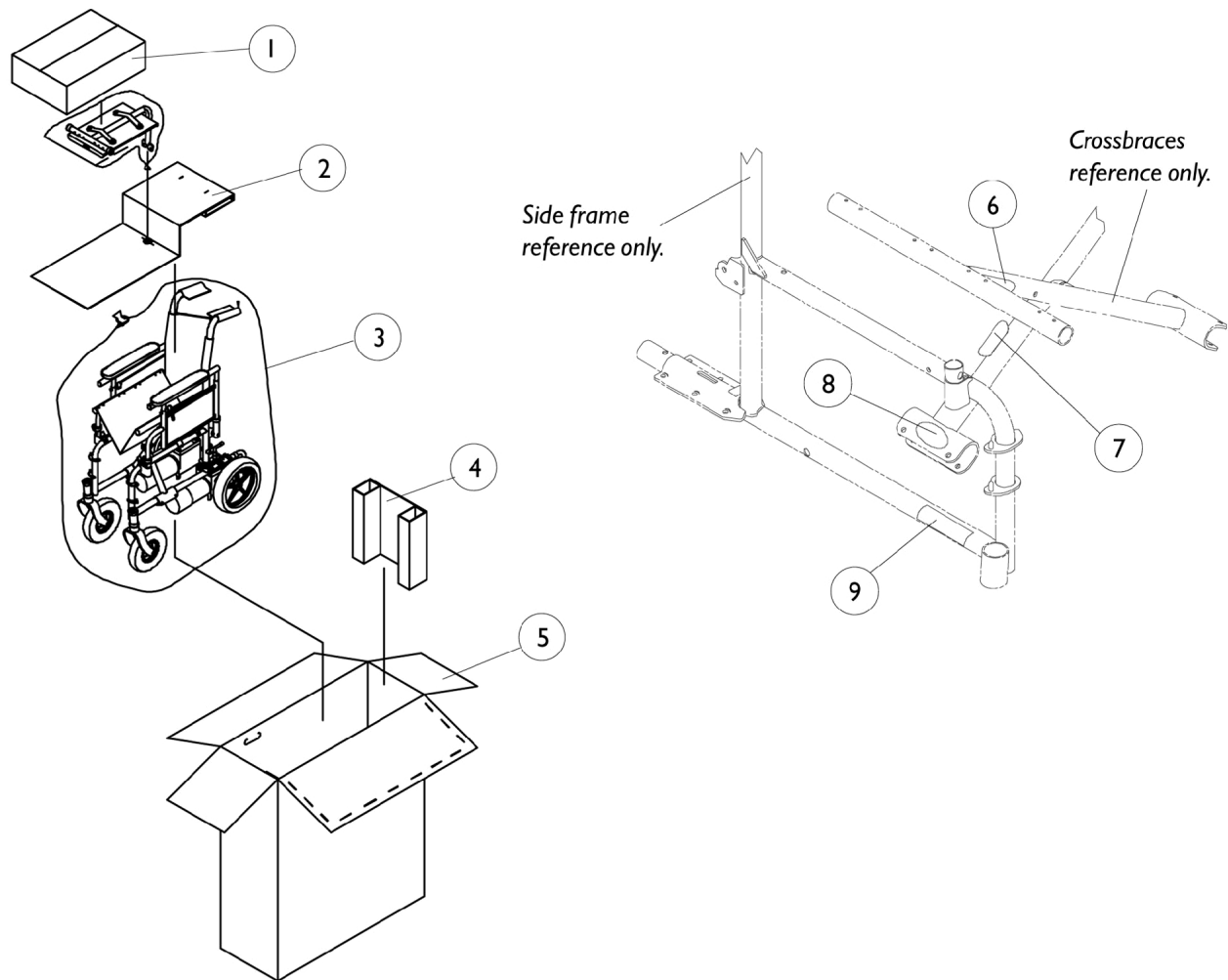

Packaging - Literature - Decals

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		40131X011-NLA	NLA - Carton, Front Riggings	1	1
2		1023166-NLA	NLA - Insert	1	1
3		1015592	Bag, Poly (72" L x 35" W x 19" D)	1	1
4		1023165-NLA	NLA - Insert, Carton	1	1
5		1074355-NLA	NLA - Carton	1	1
6		1035900	Decal, Transportation	1	1
7		60103X013-NLA	NLA - Plate, Name, Patent Numbers	1	1
8	B	1090146-NLA	NLA - Decal, Invacare 3-D Logo (1-19/32" L)	1	2
9	B	1095219-NLA	NLA - Decal, P7E	1	2
	A	1065145-NLA	NLA - Decal, Virtual Service	1	2
	A	1095273-NLA	NLA - Instruction Manual, MKIV RII LP	1	1
	A	1061807-NLA	NLA - Owner's Manual	1	1
NOTE: A - Not Shown B - Invacare logo added and model decal changed 1/11/2000. When replacing frames or crossbraces before 1/11/2000 order decals in pairs to match.					

Frame Finishes -Touch-Up Paints - Upholstery Color


Frame Finishes and Touch-Up Paints

Part Number	Description	Bottle Size
1031450-NLA	NLA - Paint, Touch-Up, 24P Wet Black	.6 oz
1046327-NLA	NLA - Paint, Touch-Up, 34P Flat Black	.6 oz
1046321-NLA	NLA - Paint, Touch-Up, 61P Electric Red	.6 oz
1046320-NLA	NLA - Paint, Touch-Up, 62P Electric Blue	.6 oz
1101320-NLA	NLA - Paint, Touch-Up, 69P Teflon Black	.6 oz
1073892-NLA	NLA - Paint, Touch-Up, 80P Evergreen w/ Navy	.6 oz
1101325-NLA	NLA - Paint, Touch-Up, 101P Jeweled Green	.6 oz

Upholstery Color

U05 Black Lightweight Vinyl

Frames - Back Canes & Hardware

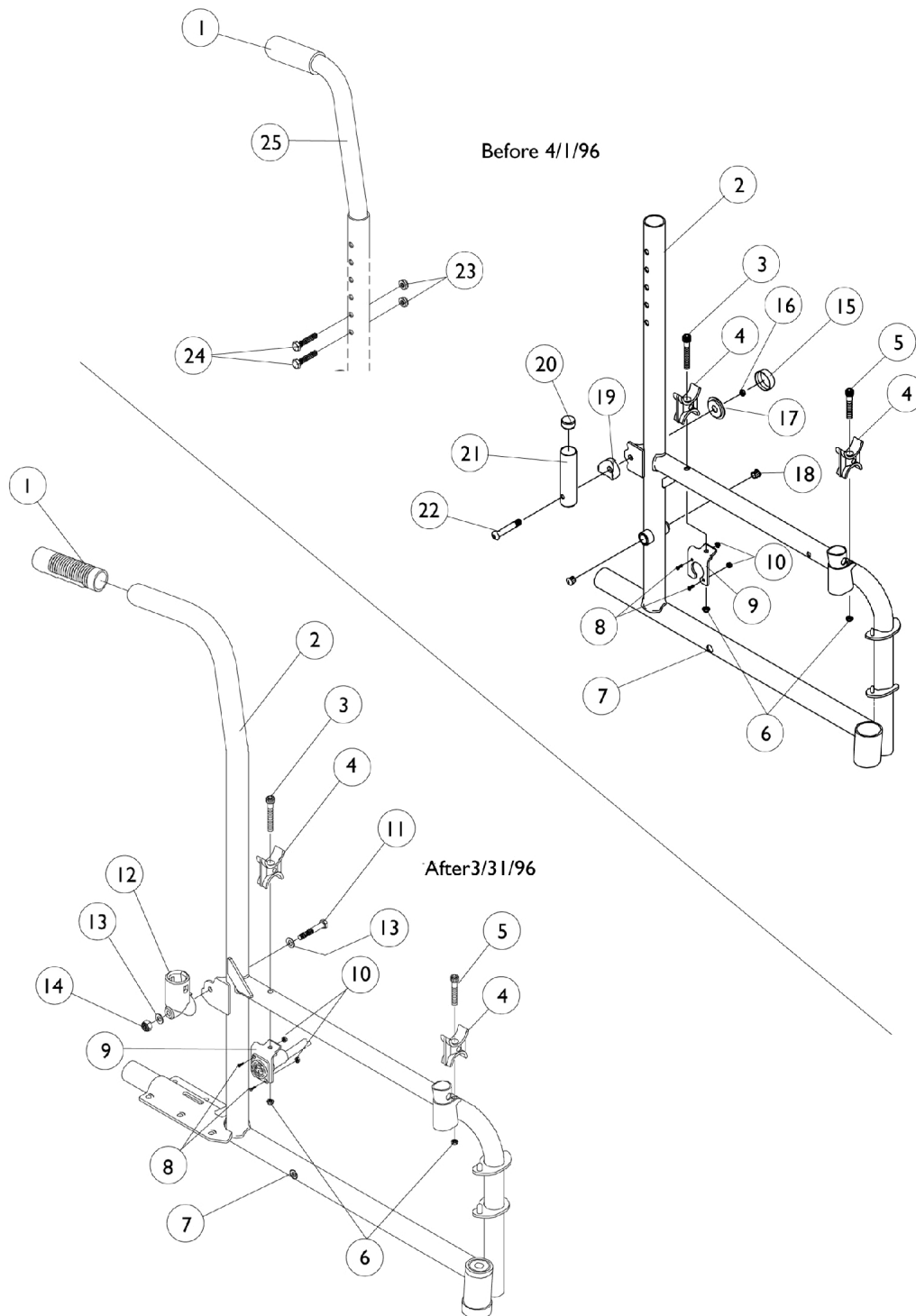

Frames - Back Canes & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C	1006229	Grip, Foam (8")	1	1
1	D	1021261	Grip, Hand (6")	1	1
2	A,B,C	8881039395-NL A	NLA - Frame, Color - Right	1	1
2	A,B,C	8881039396-NL A	NLA - Frame, Color - Left	1	1
2	A,B,D	8881061512-NL A	NLA - Frame, Side, Color - Right	1	1
2	A,B,D	8881061513-NL A	NLA - Frame, Side, Color - Left	1	1
3		1003424	Screw, Socket Head (1/4-20 x 1-3/4")	1	1
4		1015390	Guide, Seat	1	1
5		1004170	Screw, Socket Head (1/4-20 x 1-1/2")	1	1
6		7930333	Locknut (1/4-20)	25	1
7		1058520-NLA	NLA - Fastener, Panel Snap-In	1	1
8		7930318-NLA	NLA - Package, Screw, Phillips Pan Head (#4-40 x 1/2")	25	1
9		1053837-NLA	NLA - Bracket, Charger Connector, Black	1	1
10		P0870X000	Nut, Hex Keps (#4-40)	1	2
11		1055298	Screw, Hex Head (1/4-28 x 1-5/8")	1	1
12		1056406	Cup, Socket Pivot	1	1
13		1026907	Washer (1/4 x 1/2 x 1/16")	1	4
14		7930334-NLA	NLA - Locknut (1/4-28)	25	1
15		1016291	Cap, Hub, Black	1	1
16		7930364-NLA	NLA - Package, Locknut (1/4-20)	25	1
17		1016290-NLA	NLA - Washer, Caplugs (9/32 x 1 x 3/32")	1	1
18		1027097	Plug Button, Black (3/4")	1	2
19		1015426	Spacer, Coved Cam - Right	1	1
19		1015427	Spacer, Coved Cam - Left	1	1
20		1023975-NLA	NLA - Bushing (3/8 x 3/4 x 3/8")	1	1
21		1015856	Socket, Armrest	1	1
22		1023968-NLA	NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
23		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
24		1057955	Screw, Hex Head (1/4-20 x 1-1/4")	1	4
25		1028372	Back Cane Assembly, 10 Degree Bend	1	2
NOTE: A - Specify frame color. B - Decals not included with frame. C - Before 4/1/96 D - After 3/31/96					


Crossbraces - Hardware - Seat Extensions

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1029501	Plug Button, Black (1")	1	4
2		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	4
3		1012180	Bracket, Pivot, Black	1	2
4		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	1
5		1012172-NLA	NLA - Clamp, Saddle	1	2
6	A,B	1016252-NLA	NLA - Crossbrace, Black (16" W)	1	2
6	A,B	1016253	Crossbrace, Black (18" W)	1	2
6	A,C	1061299-NLA	NLA - Crossbrace, Black (16" W)	1	2
6	A,C	1061300-NLA	NLA - Crossbrace, Black (18" W)	1	2
7	D,F	1014666-NLA	NLA - Link, Pivot, Black (14 - 16" W)	1	2
		7200188-NLA	NLA - Pivot Link w/ Hardware (14 - 16" W)	1	1
7	E	1014668	Link, Pivot, Black (17 - 18" W)	1	2
8		1024793-NLA	NLA - Screw, Hex Head (1/4-20 x 1-3/4")	1	2
9		1098625	Screw, Hex Head (5/16-24 x 3-1/2")	1	1
10		16308X000	Washer, Flat (5/16 x 11/16 x 1/16")	1	2
11		1016263	Spacer, Coved (3/4 x 1 x 1/2")	1	4
12		7930313	Package, Washer, Nylon (5/16 x 7/8 x 1/8")	25	1
13		7930330-NLA	NLA - Locknut (5/16-24)	25	1
14		1037913-NLA	NLA - Bushing (5/16 x 7/16 x 9/32")	1	1
15		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	1
16		7930333	Locknut (1/4-20)	25	1
17		1058616-NLA	NLA - Kit, Seat Extension - 1" (Pair)	2	1
17		1058617	Kit, Seat Extension - 2" (Pair)	2	1
18		1026298	Package, Screw with Washer (#10 x 7/8") (Pair)	2	1
18		1076284	Screw w/ Washer (#10 x 7/8")	1	2
NOTE: A - Decals not included with crossbraces. B - Before 4/1/96 C - After 3/31/96 D - 14-16" wide E - 17-19" wide F - Pivot Link is no longer available as a service part individually. Order part #7200188 Pivot Link w/ Hardware (not shown) as a replacement. Includes items 2-4, 7 and 8.					

Upholstery

Back & Seat

NOTE: For headrest extension upholstery, see titled page under accessories "Headrest Extension, Upholstery and Hardware"

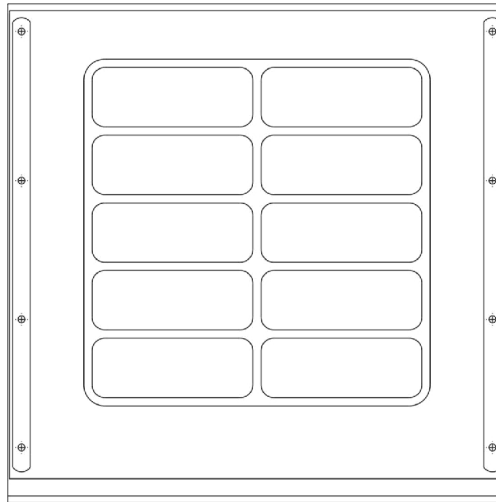

Seat and Back Upholstery
- Vinyl

Seat Upholstery

Seat Width	Depth 16"	Depth 17"	Depth 18"
16" W	8881016920-NLA	8881020690-NLA	8881023360-NLA
18" W	8881016918-NLA	8881020691-NLA	8881023359-NLA
Specify upholstery color when ordering. See Frame Finishes, Touch-Up Paints and Upholstery Color page.			

Back Upholstery

Back Width	Back Height 16"
16" W	8881017856-NLA
18" W	8881016582
Specify upholstery color when ordering. See Frame Finishes, Touch-Up Paints and Upholstery Color page.	

Upholstery


Back & Seat

NOTE: For headrest extension upholstery, see titled page under accessories "Headrest Extension, Upholstery and Hardware"

Hardware

Back and Seat Upholstery Hardware	Part Number
Consists of: 8 ea. Screw with Washer (10 x 7/8")	1027057-NLA


Headrest Extension - Upholstery & Hardware

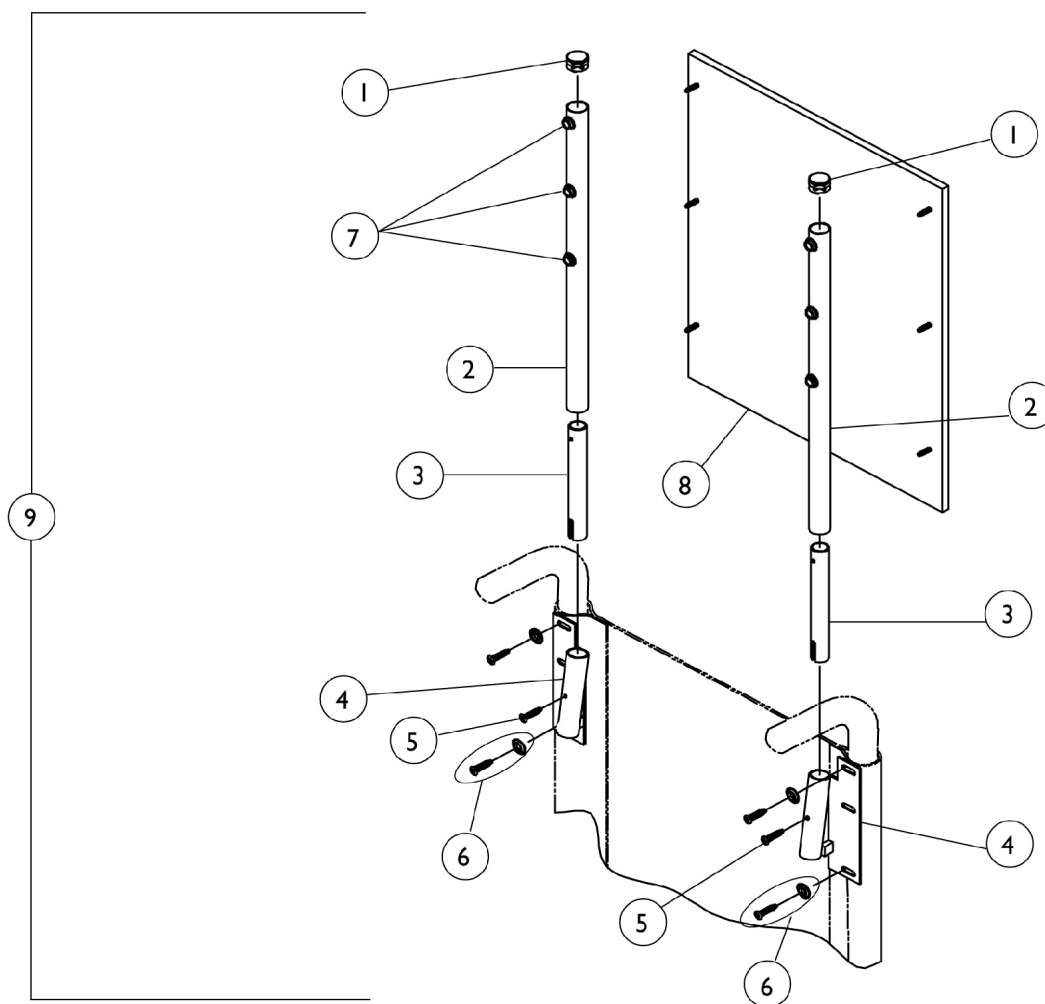

Headrest Extension - Upholstery & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	2
2		1054605-NLA	NLA - Tube, Headrest Extension	1	2
3		1028756-NLA	NLA - Tube, Insert	1	2
4		1028752-NLA	NLA - Bracket, Headrest, Black - Right	1	2
4		1028753	Bracket, Headrest, Black - Left	1	2
5		1028484	Screw, Phillips Pan Head Tap (#10-32 x 1/2")	1	2
6		1031095	Screws with Washers (#10-16 x 7/8")	4	1
6		1076284	Screw w/ Washer (#10 x 7/8")	1	4
7		47447X009-NLA	NLA - Screw, Phillips (#8 x 3/8")	1	6
8	A	8881052588-NL A	NLA - Upholstery, Headrest, Nylon (16" W)	1	1
8	A	8881052590-NL A	NLA - Upholstery, Headrest, Nylon (18" W)	1	1
9	A,B	8881052778-NL A	NLA - Headrest Extension Assembly, Nylon, Chrome (16" W)	1	1
9	A,B	8881052780-NL A	NLA - Headrest Extension Assembly, Nylon, Chrome (18" W)	1	1
NOTE: A - Specify upholstery color U240 when ordering. B - Includes items 1-8					




Fixed Height Arms & Hardware

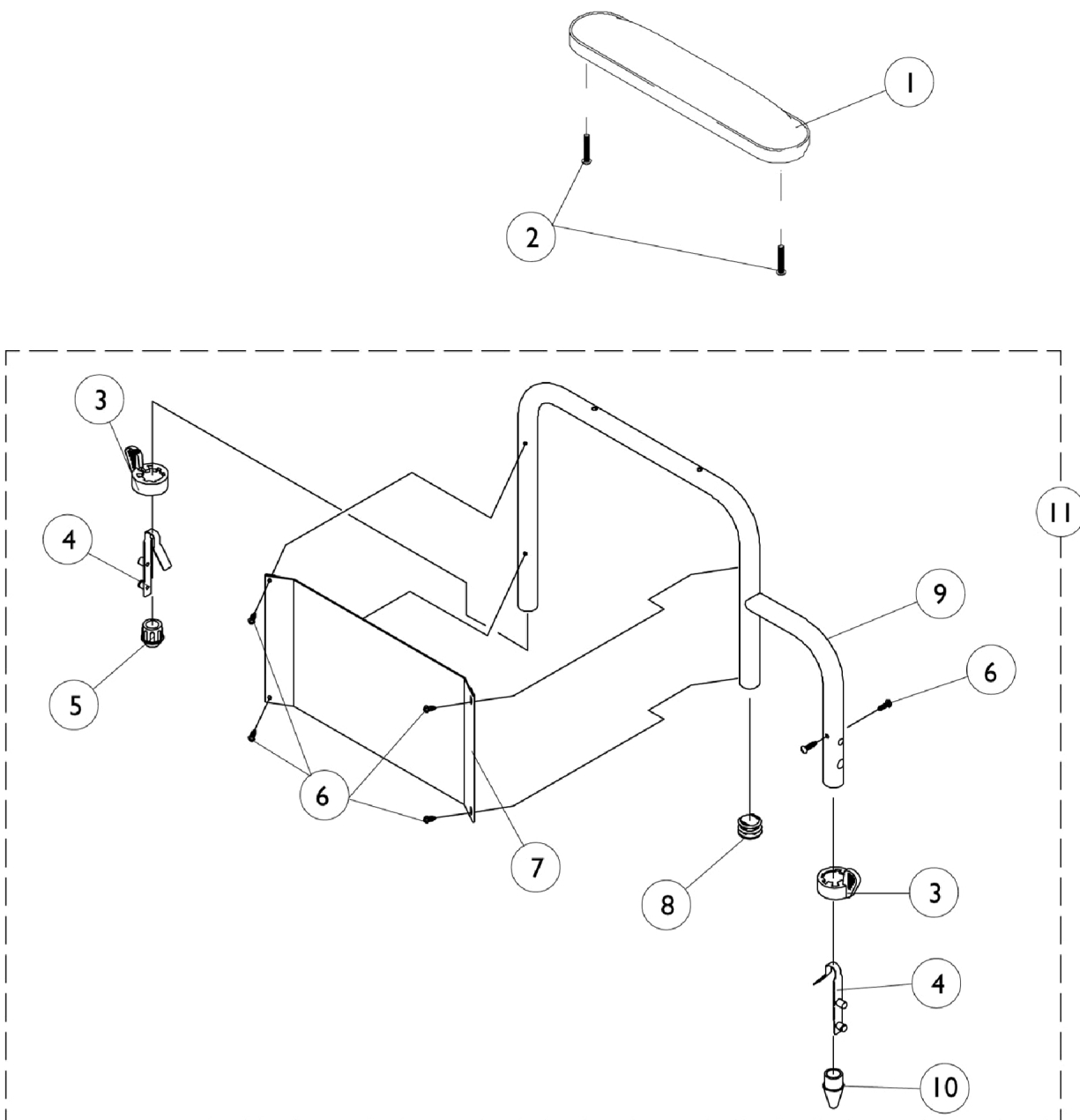

Fixed Height Arms & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881011813-NL A	NLA - Pad, Arm, Full Length w/ Law Label	1	1
1	A	8881053569	Kit, Arm Pad w/ Screws, Full Length	2	1
1	A	8881019747-NL A	NLA - Pad, Arm - Desk Length	1	1
1	A	8881053568	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
2		7930361-NLA	NLA - Package, Screw, Phillips Round Head (#10-32 x 1-1/8")	25	1
3		1015450	Handle, Quad Release	1	2
4		1018225	Spring, Torsion with Button Lock	1	2
5		1036484	Plug, Armrest, Black (3/4") (18GA)	1	1
6		1025729	Screw, Phillips Pan Head Tap (#8-32 x 3/8")	1	6
7	C	1064132	Guard, Clothing, Desk Length - Right/ Left	1	1
7	C	1064131	Guard, Clothing, Full Length - Right/ Left	1	1
8		1027097	Plug Button, Black (3/4")	1	1
9	B	1061516-NLA	NLA - Tube, Arm, Desk, Color - Right	1	1
9	B	1061517-NLA	NLA - Tube, Arm, Desk, Color - Left	1	1
9	B	1061521-NLA	NLA - Tube, Arm, Full, Color - Right	1	1
9	B	1061586-NLA	NLA - Tube, Arm, Full, Color - Left	1	1
10		1020060	Plug, Arm	1	1
11	B,C	8881061519-NL A	NLA - Arm Base, Desk, Color - Right	1	1
11	B,C	8881061520-NL A	NLA - Arm Base, Desk, Color - Left	1	1
11	B,C	8881061523-NL A	NLA - Arm Base, Full, Color - Right	1	1
11	B,C	8881061524-NL A	NLA - Arm Base, Full, Color - Left	1	1
NOTE: A - Specify upholstery color when ordering. B - Specify frame color from Frame Finishes, Touch-Up Paints and Upholstery Color page. C - Stainless steel clothing guards were replaced w/ABS plastic. Chairs manufactured before 7/30/97 should be ordered in pairs to match.					

Adjustable Height Arms & Hardware

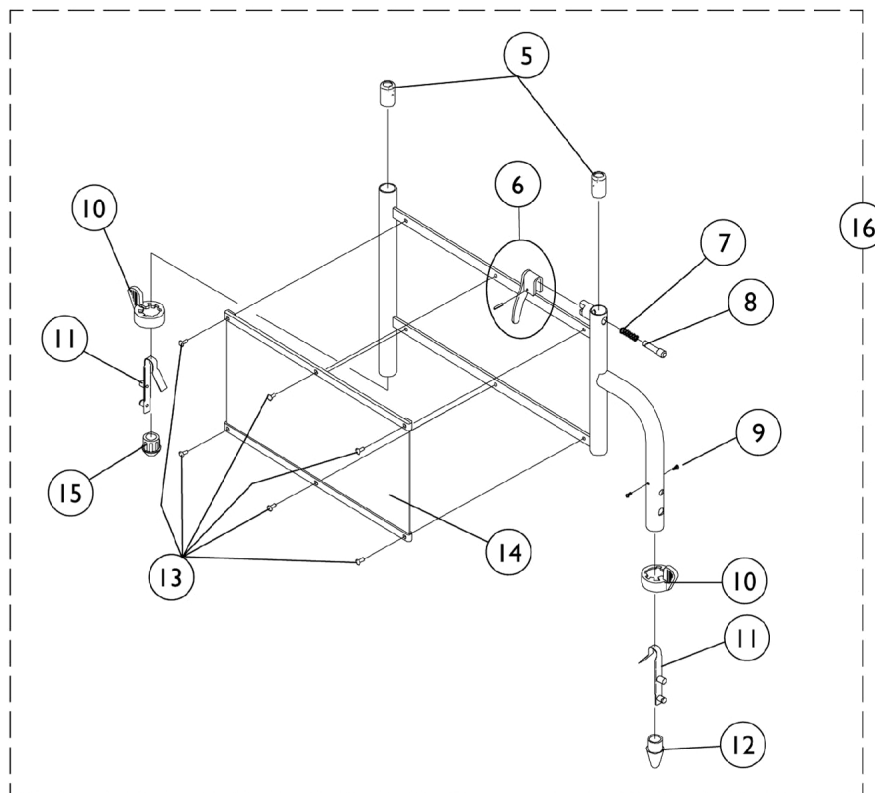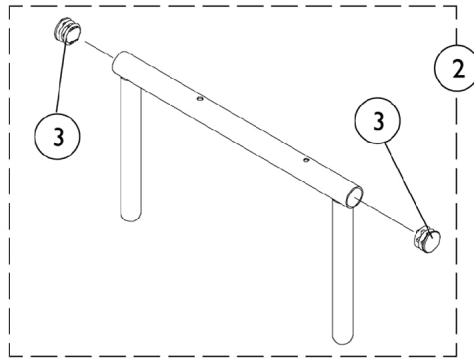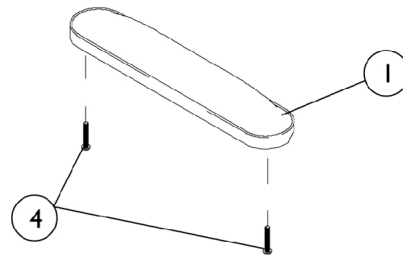


Adjustable Height Arms & Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881011813-NL A	NLA - Pad, Arm, Full Length w/ Law Label	1	1
1	A	8881053569	Kit, Arm Pad w/ Screws, Full Length	2	1
1	A	8881019747-NL A	NLA - Pad, Arm - Desk Length	1	1
1	A	8881053568	Kit, Arm Pad w/ Screws, Desk Length (Pair)	2	1
2		1038470	Arm Tube Assembly, Conventional w/ Plug Buttons, Desk Length, Black - Right/Left	1	1
2		1038472	Arm Tube Assembly w/ Plug Buttons, Full Length, Black - Right/Left	1	1
3		1027095	Plug Button, Black (7/8")	1	2
4		7930361-NLA	NLA - Package, Screw, Phillips Round Head (#10-32 x 1-1/8")	25	1
5		1036150	Anti-Rattle	1	2
6		1090768	Kit, Lock Handle w/Rivets (Pair)	2	1
7		47471X074	Spring, Arm Lock	1	1
8		47471X068	Pin, Locking	1	1
9		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
10		1015450	Handle, Quad Release	1	2
11		1018225	Spring, Torsion with Button Lock	1	2
12		1020060	Plug, Arm	1	1
13		1028645	Rivet	1	6
14	C	1064129	Guard, Clothing	1	1
15		1036484	Plug, Armrest, Black (3/4") (18GA)	1	1
16	B,C	8881061526-NL A	NLA - Arm Base, Color - Right	1	1
16	B,C	8881061527	Arm Base, Color - Left	1	1

NOTE: A - Specify upholstery color when ordering from Frame Finishes, Touch-Up Paints and Upholstery Color page.
 B - Specify frame color when ordering from Frame Finishes, Touch-Up Paints and Upholstery Color page.
 C - Stainless steel clothing guards were replaced w/ABS plastic. Chairs manufactured before 7/30/97 should be ordered in pairs to match.


Casters & Forks

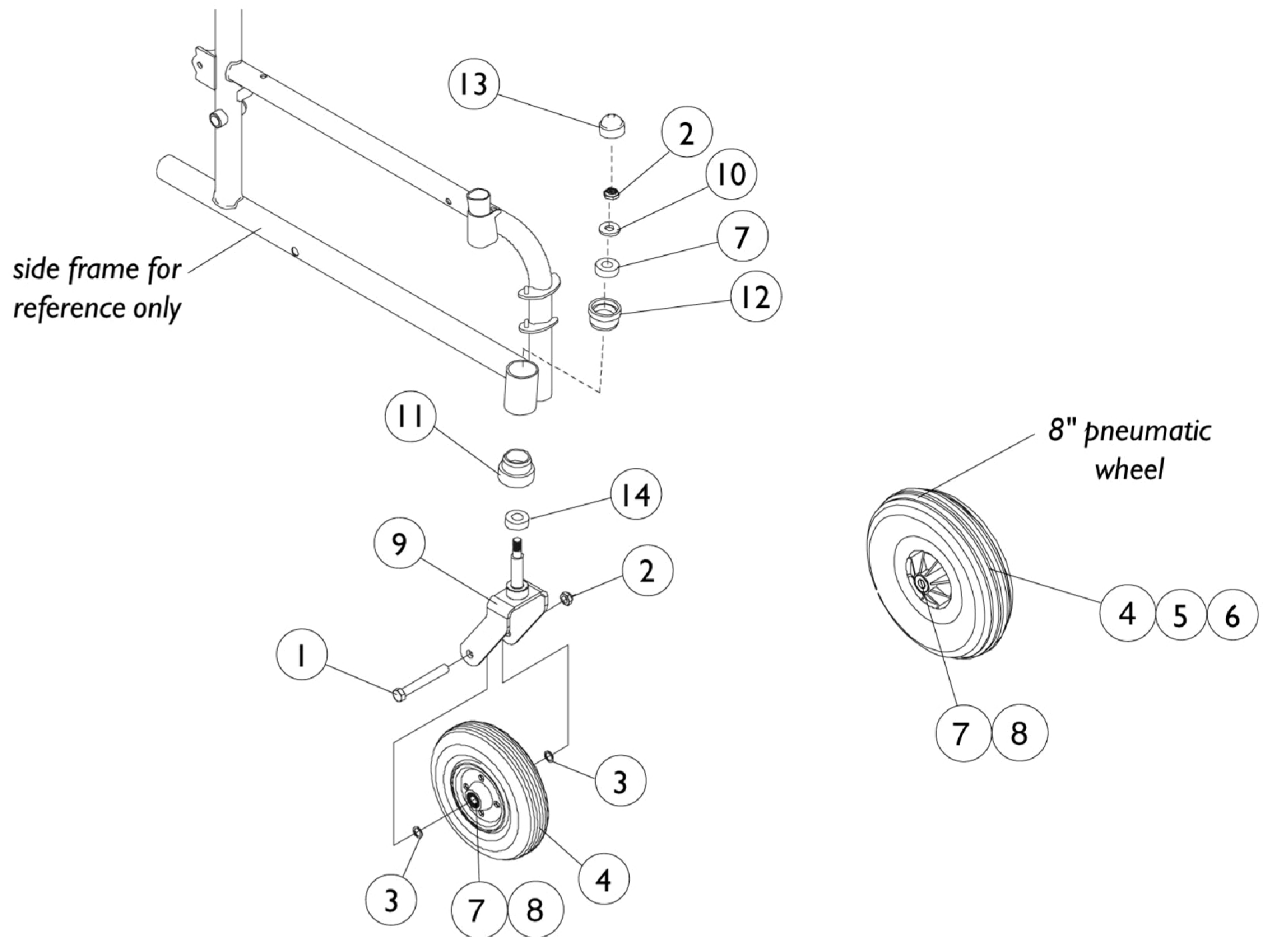

Casters & Forks

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		57667X007-NLA	NLA - Screw, Hex Head (7/16-20 x 3-1/4")	1	1
2		7930352-NLA	NLA Locknut (7/16-20)	25	1
3		1016032	Washer, Steel (15/32 x 5/8 x 3/64")	1	2
4	B,D	1070811	Wheel Assembly, Semi-Pneumatic, Urethane, Light Grey (8" x 1-3/4")	1	1
4		1071666	Wheel Assembly, Semi-Pneumatic, Urethane, Light Grey (8" x 1-3/4") (Pair)	2	1
4	D	1002002	Wheel Assembly w/ Bearings, Pneumatic (8" x 2")	1	1
5		40176X034	Tire, Pneumatic (8" x 2")	1	1
6		1034942	Tube, Pneumatic Inner (8 x 2")	1	1
7		1006049	Bearing (7/16")	1	2
8		1020592	Spacer, Bearing (7/16 x 5/8 x 1-3/4")	1	1
9	A,C	8881079862-NLA	NLA - Fork and Stem, Color (6")	1	1
10		40014X044-NLA	NLA - Washer, Nylon (7/16 x 1 x 1/8")	1	1
11		1031829-NLA	NLA - Sleeve, Lower Bearing	1	1
12		1028257-NLA	NLA - Sleeve, Bearing, Upper	1	1
13		1001843	Cap, Head Tube, Black	1	1
13		1131189	Package, Head Tube Cap	10	1
14		40117X006	Bearing (5/8")	1	1
NOTE: A - Specify frame color 24P when ordering from Frame Finishes, Touch-Up Paints and Upholstery Color page.. B - Chairs manufactured before 6/9/97 order in pairs (#1071666). C - Forks must be ordered in pairs for chairs manufactured before 10/5/98. D - Includes items 7 and 8					

Rear Wheels & Motor Hardware

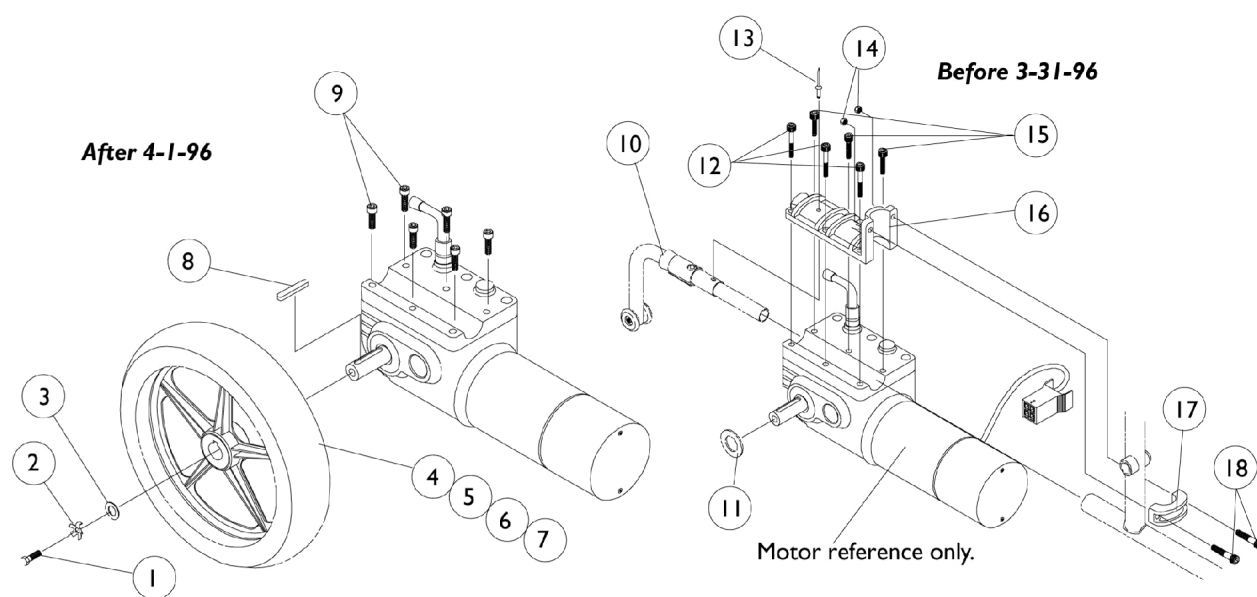


NOTE: Items 11 thru 18 only used on chairs before 3-31-96

Rear Wheels & Motor Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		7930322-NLA	NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm)	25	1
2		1064179	Kit, Locking Tab Washer w/ Instruction Sheet (Pair)	2	1
3		1079247	Washer, Flat (21/64 x 53/64 x 7/64")	1	1
4		1064178-NLA	NLA - Drive Wheel Assembly, 5 Spoke, Pneumatic (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
4		1064177	Drive Wheel Assembly, 5 Spoke w/ Foam Fill Insert (12-1/2 x 2-1/4") Includes: Locking Tab Washer with Instruction Sheet	1	1
5		1011524	Tire, Pneumatic, Grey (12-1/2" x 2-1/4")	1	1
6		1011525	Tube, Inner, Black (12-1/2 x 2-1/4")	1	1
6	A	1073936-NLA	NLA - Insert, Flat Free	1	1
7		1064176-NLA	NLA - Rim with Locking Tab	1	1
8		1023264	Key (6mm x 6mm x 40mm)	1	1
9		1057824-NLA	NLA - Screw, Socket Head (M6 x 16)	1	6
10	B	1034131-NLA	NLA - Anti-Tipper	1	1
10	C	1061684-NLA	NLA - Anti-Tipper	1	1
11		1023261-NLA	NLA - Washer, Steel {11/16 x 1-3/16 x 1/8")	1	1
12	D	1023257-NLA	NLA - Screw, Socket Head (M6 x 25MM)	1	3
13		1007094-NLA	NLA - Rivet, Pop (3/16") (UI Battery Boxes Only)	1	1
14		7930307-NLA	NLA - Locknut (1/4-20")	25	1
15		1023258-NLA	NLA - Screw, Socket Head Cap (M6 x 1 x 40mm)	1	3
16		1035012-NLA	NLA - Half Clamp, Motor	1	1
17		1035013-NLA	NLA - Half Clamp, Tube	1	1
18		7930304-NLA	NLA - Screw, Socket Head (1/4-20 x 1-1/4")	25	1
NOTE: A - As of 10/20/99 the "solid" flat free insert was replaced by a "foam fill" process. When a flat free insert is requested, a complete wheel must be ordered. B - Before 3/31/96 C - After 4/1/96 D - No Longer Available.					


Wheel Locks and Hardware

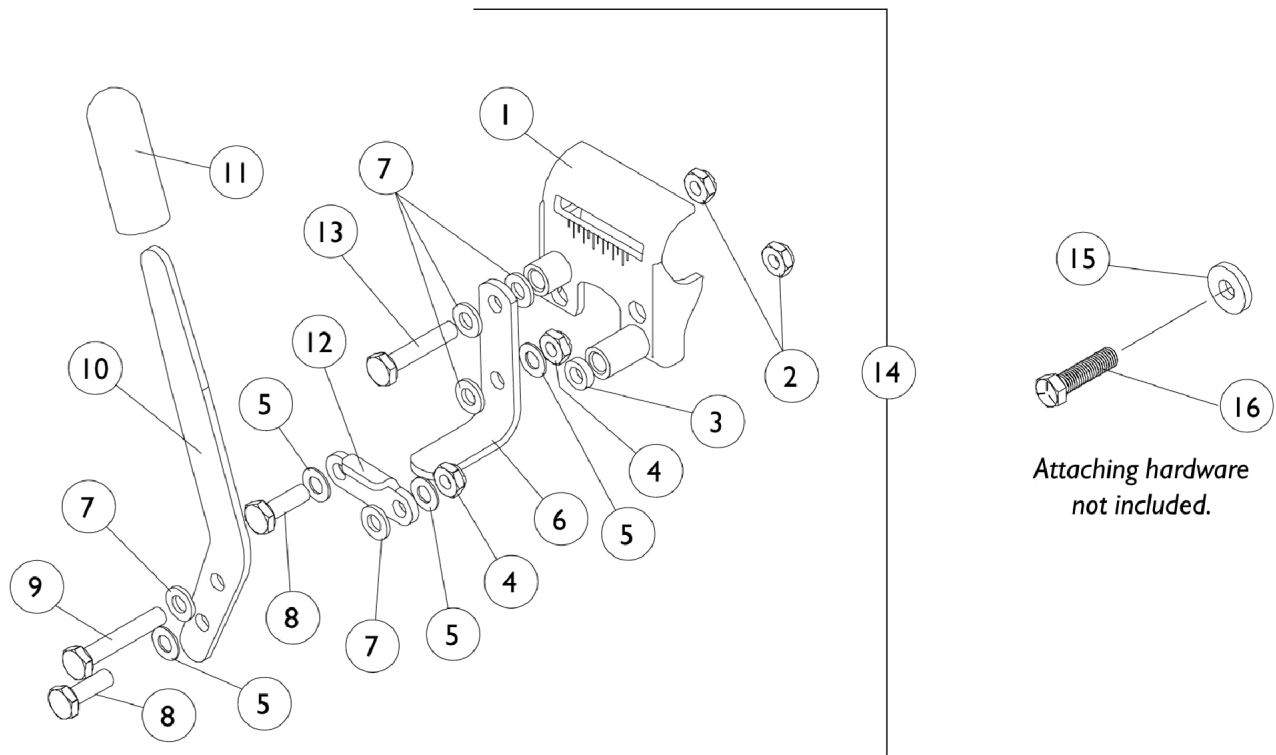

Wheel Locks and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1031837-NLA	NLA - Bracket, Mounting, Black - Right	1	1
1		1031838-NLA	NLA - Bracket, Mounting, Black - Left	1	1
2		7930323-NLA	NLA - Package, Locknut (1/4-20)	25	2
3		1026105-NLA	NLA - Spacer, Nylon (1/4 x 7/16 x 1/8")	1	1
4		7930334-NLA	NLA - Locknut (1/4-28)	25	2
5		1026103	Washer, Nylon (1/4 x 1/2 x 1/32")	1	4
6		1031833-NLA	NLA - Shoe, Wheel Lock	1	1
7		1026033	Washer, Nylon (1/4 x 1/2 x 1/16")	1	5
8		1028687	Screw, Hex Head (1/4-28 x 3/4")	1	2
9		1030949	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
10		1061504-NLA	NLA - Handle, Wheel Lock, Black	1	1
11		47376X024	Tip, Rubber Wheel Lock Handle, Black	1	1
12		1018733-NLA	NLA - Link, Stop, Black	1	1
13		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	1
14		1061612-NLA	NLA - Wheel Lock Assembly - Right	1	1
14		1061613-NLA	NLA - Wheel Lock Assembly - Left	1	1
15		1018739	Washer (1/4 x 5/8 x 3/32")	1	1
16		7930326-NLA	NLA - Package, Screw, Hex Head (1/4-20 x 3/4")	25	1



Motor Gearbox Assembly

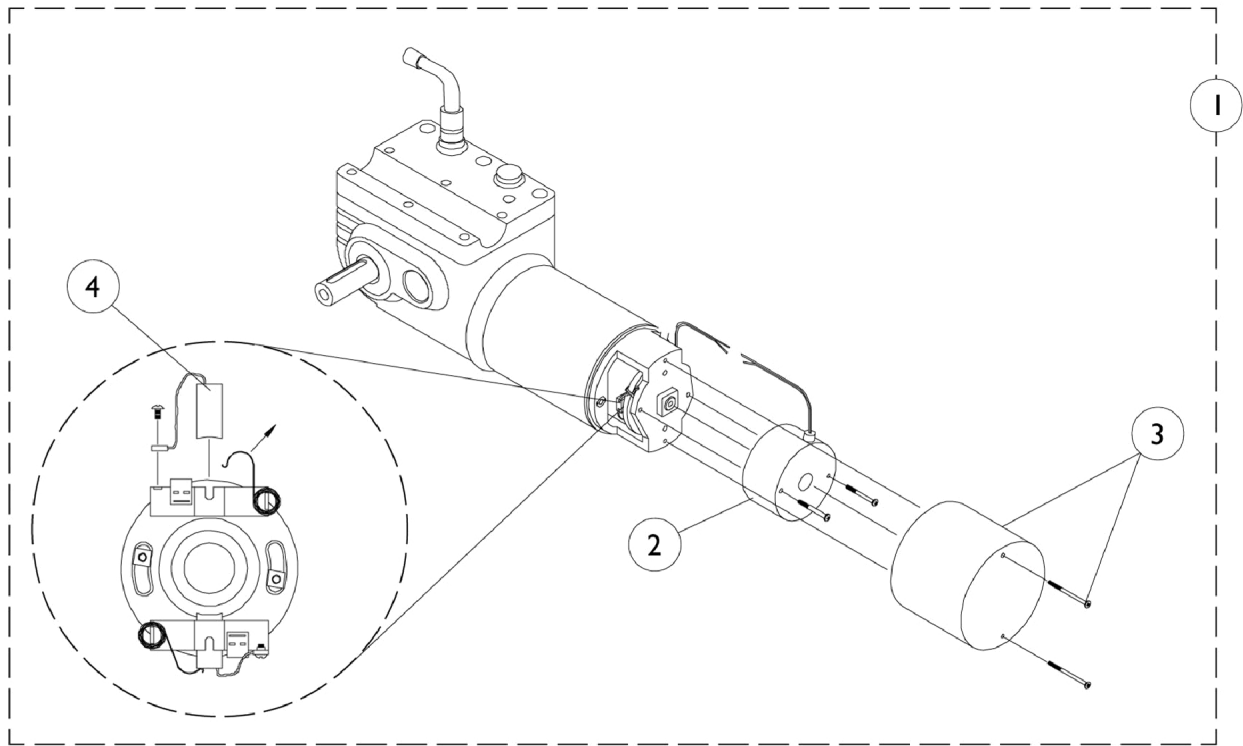


Motor Gearbox Assembly

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1037543-NLA	NLA - Motor/Gearbox Assembly with Brake	2	1
1	B	1057902	Motor/Gearbox Assembly, Long Cap - Right	1	1
1	B	1057903	Motor/Gearbox Assembly, Long Cap - Left	1	1
2		1055872-NLA	NLA - Brake, Spring Set with Screws	1	1
3		1038361-NLA	NLA - Kit, Motor/Brake Covers with Screws	2	1
4		561-005-NLA	NLA - Kit, Motor Brush w/o Cap - Internal, Long Cap (Pair)	2	1
NOTE: A - Before 3/31/96 B - After 4/1/96					


Battery Boxes - Chargers - Harnesses - Trays

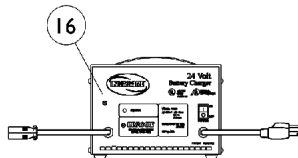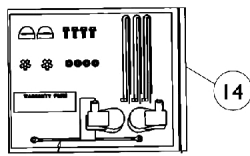


Lester 4 Amp. Battery Charger

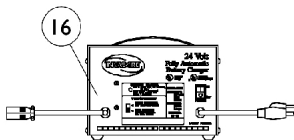


Lester 8 Amp. Battery Charger

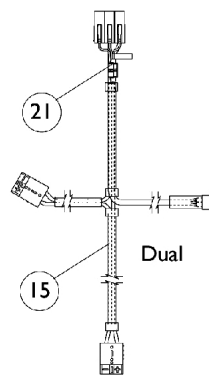





Dual

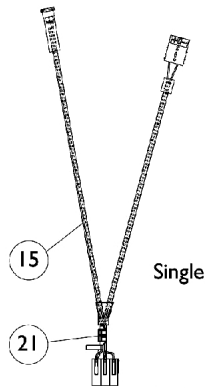

Single

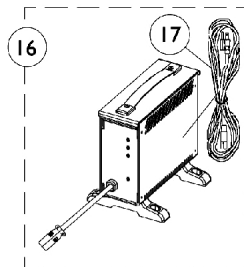

Dynamic or CTE
Battery Charger

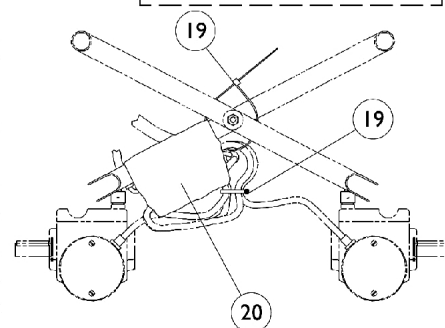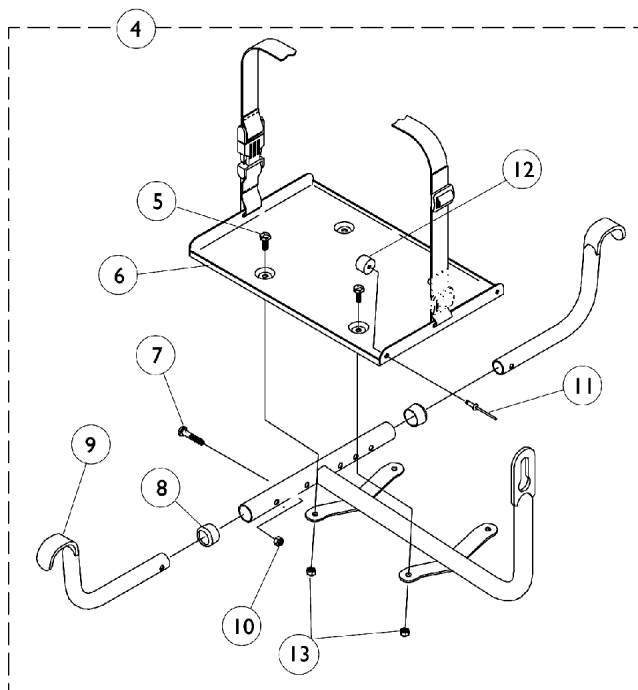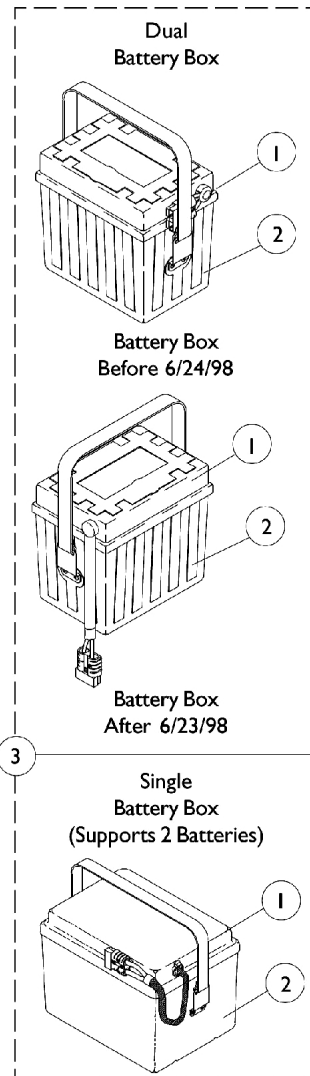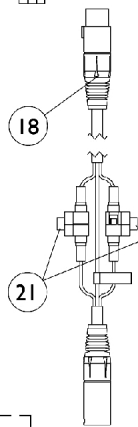



Battery Boxes - Chargers - Harnesses - Trays

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B,B1,E	1060462-NLA	NLA - Battery Box Top Assembly, Single	1	1
1	B,B1,E, F	1037275-NLA	NLA - Battery Box Top Assembly, Dual w/ Single Fixed Connector	1	2
1	B,B1,E, G	1057941-NLA	NLA - Battery Box Top Assembly, Dual w/ Single Extended Connector, UI	1	2
2		1060460-NLA	NLA - Battery Box Bottom, Single	1	1
2		1009268-NLA	NLA - Battery Box Bottom Assembly, U1	1	2
3		1061509-NLA	NLA - Battery Box Assembly, Single, Supports 2 Batteries	1	1
3	F	1061510-NLA	NLA - Battery Box Assembly, Dual w/ Single Fixed Connector	1	2
3	G	1057942-NLA	NLA - Battery Box Assembly, Dual w/ Single Extended Connector, UI	1	2
4		1060451	Battery Tray Assembly, Single	1	1
4		1061158	Battery Tray Assembly, Dual	1	1
5		1025752-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/2")	1	4
6		1037270-NLA	NLA - Tray with Buckle {Single and Dual}	1	1
7		1021406	Screw, Hex Head (1/4-20 x 1-1/4")	1	2
8		15680X012	Collar, Anti-Rattle (7/8")	1	2
9		1038220-NLA	NLA - Bracket, Tray Hanger, Black	25	2
10		7930333	Locknut (1/4-20)	1	1
11		1007094-NLA	NLA - Rivet, Pop (3/16") (UI Battery Boxes Only)	1	2
12		1016025-NLA	NLA - Bumper, Rubber (UI Battery Boxes Only)	25	2
13		7930323-NLA	NLA - Package, Locknut (1/4-20)	1	1
14		1097005-NLA	NLA - Jumper Cable Hardware Kit	1	1
15		1060464-NLA	NLA - Harness, Wire, Single	1	1
15		1067404	Harness, Wire, Dual	1	1
16	C,O	1053161-NLA	NLA - Charger, Battery, 8 Amp, 115vac, Off Board Dual Mode Round Plug, Lester	1	1
16		1085678-NLA	NLA - Charger, Battery, 8 Amp. 115vac, Off Board, Dual Mode, Round Plug w/ AC Power Line Cord, Dynamic	1	1
16	L	1061411-NLA	NLA - Charger, Battery, 4 Amp. 115vac, Off Board, Dual Mode, Round Plug, Lester	1	1
16	M,M1	1123249	Charger, Battery, 8 Amp. 115vac (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
16	M,M1	1123249E-NLA	NLA - WARR RPLMT - Charger, Battery, 8 Amp, 115VAC (Off Board) Single Mode Round Plug w/ AC Power Line Cord	1	1
17	N	1089627-NLA	NLA - Cord, AC Power, 115VAC, 2 Prong Off-Board (11' 6" L)	1	1
17		1121086	Cord, AC Power, 115VAC, 3 Prong On-Board (6' 6-1/2" L)	1	1
18		1083929-NLA	NLA - Cable, Charger Extension, Round Plug (24" L)	1	1
19		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	1

Battery Boxes - Chargers - Harnesses - Trays

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
20		1058491-NLA	NLA - Connector Boot, Wire	1	1
21	J	1098892-NLA	NLA - Fuse - Buss ATC-15 (15 AMP)	1	1
21	K	1090117-NLA	NLA - Fuse - Buss Mini ATM-15 (15 AMP)	1	2
	A,B,B1,H	50000X166-NLA	NLA - Fuse - Buss 2-1/4" L (60 AMP)	1	2
	A,I	1089385	Fuse, MIDI (Non-G/B & G/B) (80 AMP)	1	1
	A,D	1060454-NLA	NLA - Hardware, Terminal, U1 Batteries	2	1
NOTE: A - Not Shown B - Battery harness fuse changed from a Bussman 60 Amp. to a MIDI 80 Amp. fuse as of 11/5/99 for Dual Battery Boxes and 11/10/00 for the Single Battery Box. B1 - The current battery box top assemblies are compatible for all chairs manufactured before the dates specified and must be ordered when a replacement Bussman 60 amp. fuse is requested. C - On 7/18/01 Lester 8 battery charger was reinstated. D - Hardware includes small battery terminal caps (gray and orange), 1/4-20 x 7/8" hex flange screws and instruction sheet for two batteries. E - It is recommended that new terminal hardware be ordered when a new battery box top is ordered. F - Before 6/24/98 G - After 6/23/98 H - Before 11/5/99 for dual battery box and Before 11/10/00 for Single battery box I - After 11/4/99 for dual battery box and After 11/9/00 for single battery box J - Before 4/1/03. Charger extension cable takes 1 fuse and the wire harness takes 2 fuses K - After 3/31/03. Charger extension cable takes 2 fuses and the wire harness takes 2 fuses L - The 4 amp. Lester charger was obsoleted and replaced with the 8 amp. CTE charger. For wet cell batteries, use the 8 amp. dual mode Lester charger. M - CTE charger introduced 4/1/04. Used on Gel Cell or AGM (Absorbed Glass Mat) batteries. For wet cell batteries refer to the 8 amp. Lester charger. M1 - CTE charger looks exactly the same as Dynamic's charger except for the company name that is silk screened on the narrow side panel, where the charger plug is (hard wired) attached. Serial number starts with 04A00000001. N - No longer available as a service part. Please use 1121086 as its replacement. O - Use Battery Charger part number 1123249 as a replacement.					

P93 Footrest and Hardware

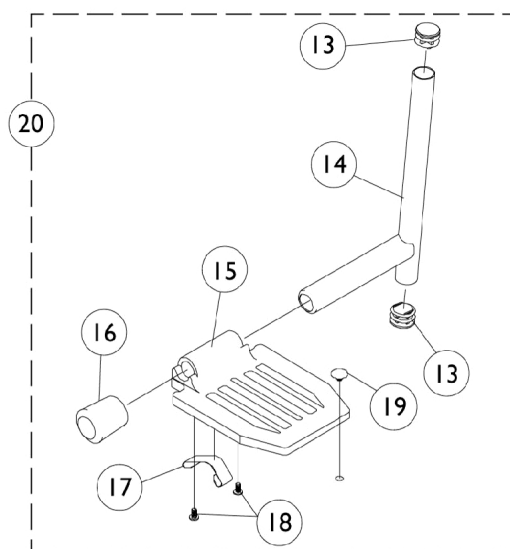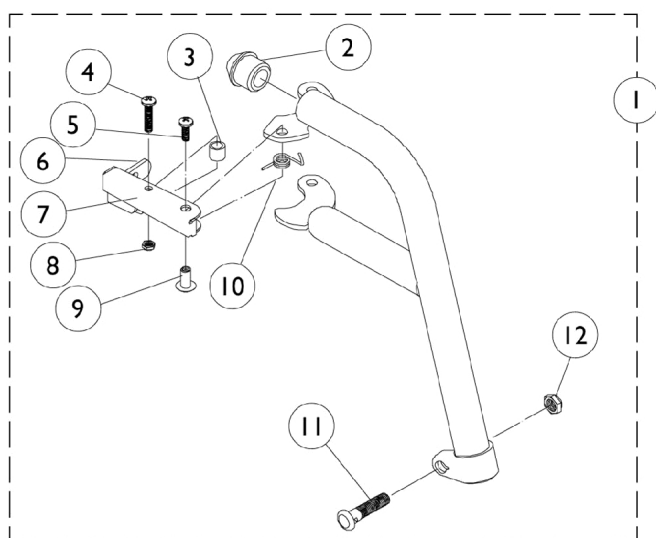

P93 Footrest and Hardware




Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881035779	Footrest Support Assembly, Adult, Color - Right	1	1
1	A	8881035780	Footrest Support Assembly, Adult, Color - Left	1	1
2		1020719	Anti-Rattle, Black	1	1
3		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
4		7930328-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 11/16")	25	1
5		7930335-NLA	NLA - Package, Screw, Phillips Pan Head (#10-32 x 1/2")	25	1
6		1006490	Tip, Plastic, Black	1	1
7		1035539	Handle, Release, Black - Right	1	1
7		1035540	Handle, Release, Black - Left	1	1
8		7930322-NLA	NLA - Package, Screw, Hex Head (M8 x 1.25 x 20mm)	25	1
9		1028846-NLA	NLA - Rivnut (#10-32)	1	1
10		1028707	Spring, Release - Right	1	1
10		1028708	Spring, Release - Left	1	1
11		1026030	Bolt, Lug (5/16-24 x 1-3/8")	1	1
12		1107321	Nut, Low Cap (5/16"-24)	1	1
13		1027097	Plug Button, Black (3/4")	1	1
14		1041527	Pivot/Slide Tube w/ Plug Button, Black	1	1
14		1086474-NLA	NLA - Pivot Slide Tube w/ Plug Buttons, Chrome (2" Longer)	1	1
14		1086475-NLA	NLA - Pivot/Slide Tube w/ Plug Buttons, Chrome (4" Longer)	1	1
15		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
16		1016346	Cap, End, Tube, Black (7/8")	1	1
17		1006981	Spring, Tension	1	2
18		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
19		1070168	Plug, Push, Footplate	1	1
20		See Chart	#20 Pivot/Slide Tube and Footplate	1	1

NOTE: A - Specify frame color from Frame Finishes, Touch-Up Paints and Upholstery Color Page

#20 Pivot/Slide Tube and Footplate

Side	Standard	2" Longer	4" Longer
Right	1057969-NLA	1086503-NLA	1086504-NLA
Left	1057970-NLA	1086505-NLA	1086506-NLA

PW93 Footrest and Hardware

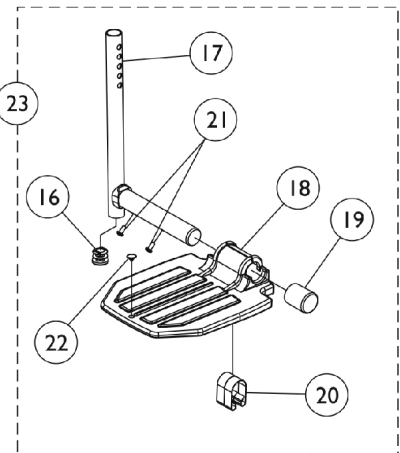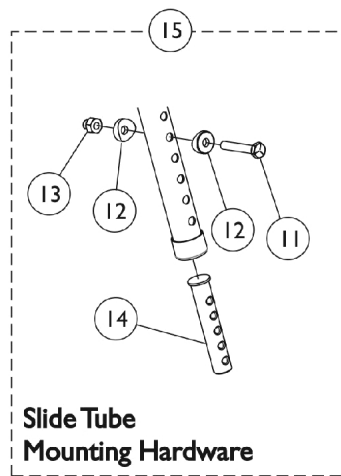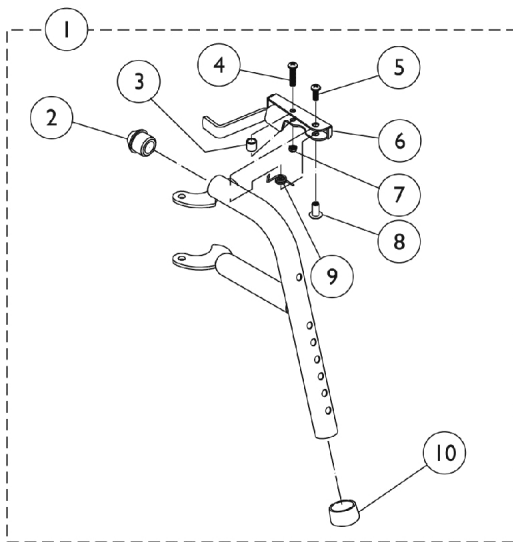

PW93 Footrest and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	8881070776	Footrest Support Assembly, Adult, Color - Right	1	1
1	A	8881070777	Footrest Support Assembly, Adult, Color - Left	1	1
2		1020719	Anti-Rattle, Black	1	1
3		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
4		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	1
5		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
6		1035539	Handle, Release, Black - Right	1	1
6		1035540	Handle, Release, Black - Left	1	1
7		1025735	Locknut (#10-32)	1	1
8		1028846-NLA	NLA - Rivnut (#10-32)	1	1
9		1028707	Spring, Release - Right	1	1
9		1028708	Spring, Release - Left	1	1
10		15680X012	Collar, Anti-Rattle (7/8")	1	1
11		1070759	Screw, Hex Head (1/4-20 x 1-1/2")	1	1
12		1026099	Spacer, Coved (1/4 x 11/16 x 1/2")	1	2
13		1044214	Nut, Hex with Nylon Cap (1/4-20)	1	1
14	B	1020534	Insert, Threaded	1	1
15		1074143	Kit, Slide Tube Mounting Hardware	1	1
16		1027097	Plug Button, Black (3/4")	1	1
17		1048834	Pivot/Slide Tube w/ Plug Button, Black	1	1
17		1098906	Pivot/Slide Tube w/ Plug Button, Black (2" Longer)	1	1
17		1086482	Pivot/Slide Tube w/ Plug Button, Black (4" Longer)	1	1
18		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
19		1016346	Cap, End, Tube, Black (7/8")	1	1
20		1006981	Spring, Tension	1	2
21		40120X001	Screw, Phillips Pan Head (#8-32 x 3/8")	1	2
22		1070168	Plug, Push, Footplate	1	1
23		See Chart	#23 Pivot/Slide Tube with Footplate	1	1


NOTE: A - Specify frame color from Frame Finishes, Touch-Up Paints and Upholstery Color Page
 B - As of 8/15/03, the Threaded Insert was revised from 3 to 5 Holes. Threaded Insert is Backward Compatible and does not require ordering in Pairs.

#23 Pivot/Slide Tube with Footplate

Side	Standard	2" Longer	4" Longer
Right	1074149	1086494-NLA	1086495-NLA
Left	1074150	1086497-NLA	1086498-NLA

P904/T934A Legrest and Hardware

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.

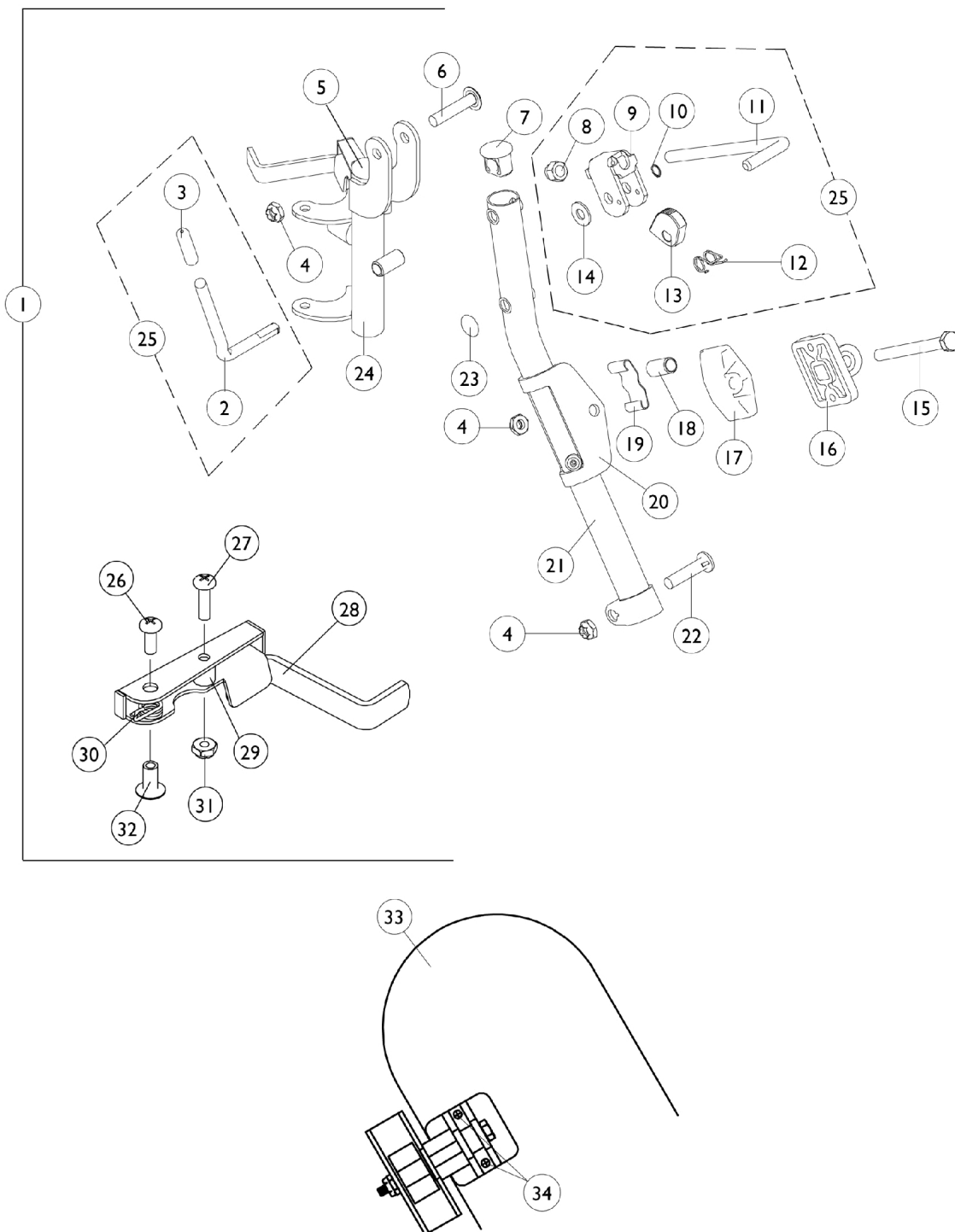

P904/T934A Legrest and Hardware

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	C,I	1031579-NLA	NLA - Legrest Support Assembly, Adult, Silver Vein - Right	1	1
1	C,I	1031580-NLA	NLA - Legrest Support Assembly, Adult, Chrome - Left	1	1
2		1037126	Handle, Release	1	1
3		1040060	Tip, Plastic, Black	1	1
4		7930363-NLA	NLA - Package, Locknut (5/16"-24)	25	3
5		1027095	Plug Button, Black (7/8")	1	1
6		1030948	Screw, Slotted Truss Head (5/16-24 x 1-1/2")	1	1
7		1095433	Plug Button, Black (7/8")	1	1
8		7930336-NLA	NLA - Nut, Acorn (5/16-24)	25	1
9		04528X002	Bracket, Release, Black	1	1
10		1040831	Ring, Snap	1	1
11		42326X001	Bar, Ratchet	1	1
12		1037485	Spring, Release	1	1
13		1037125	Cam, Legrest	1	1
14		7930310-NLA	NLA - Washer, Nylon (5/16 x 11/16 x 1/16")	25	1
15		1014807	Screw, Hex Head (5/16-24 x 3-1/4")	1	1
16		04526X019	Plate, Legrest Pad	1	1
17		1091220	Spacer, Legrest	1	1
18		47328X003	Bushing, Pivot (5/16 x 7/16 x 7/8")	1	1
19		48823X008	Spring, Leaf	1	1
20		1025104	Bracket, Legrest, Chrome	1	1
21	A,G	1031358-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Right	1	1
21	A,G	1031359-NLA	NLA - Tube, Support, Legrest, Elevating, Chrome - Left	1	1
21	H	1149473-NLA	NLA - Tube, Support, Legrest, Elevating, Silver Vein - Right	1	1
21	H	1149474-NLA	NLA - Tube, Support, Legrest, Elevating, Silver Vein - Left	1	1
22		11232X009	Bolt, Lug (5/16-24 x 1-3/8")	1	1
23		1096773	Nut, Push Dome (5/16")	1	1
24		1031581-NLA	NLA - Legrest Support Tube Assembly, Adult, Black - Right	1	1
24		1031582	Legrest Support Tube Assembly, Adult, Black - Left	1	1
25	D	1046328-NLA	NLA - Kit, Ratchet	2	1
26		1025716	Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
27		1026114	Screw, Phillips Pan Head (#10-32 x 1/2")	1	1
28		1035539	Handle, Release, Black - Right	1	1
28		1035540	Handle, Release, Black - Left	1	1
29		47565X057	Bushing (7/32 x 3/8 x 11/32")	1	1
30		1028707	Spring, Release - Right	1	1
30		1028708	Spring, Release - Left	1	1

P904/T934A Legrest and Hardware

Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.


Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
31		1025735	Locknut (#10-32)	1	1
32		1028846-NLA	NLA - Rivnut (#10-32)	1	1
33	B,E,K	1091767-NLA	NLA - Pad with Screws, Legrest (Pair)	2	1
33	B,F,K	8881091764	Pad, Legrest, Vinyl - Right/Left	1	1
34	J	1066591	Hardware, Legrest Pad	1	1
			Screw, Phillips Pan Head (#10-32 x 11/16")	1	2
			Screw, Phillips Pan Head (#10-32 x 1")	1	2
			Instruction Sheet, Legrest Pad Mtg.	1	1
NOTE: A - Due to product improvement, item #5 must also be ordered. B - Specify U67 Black vinyl upholstery when ordering. C - Includes items 2-24 D - Includes items 2, 3 and 8-14 E - Before 9/5/00 F - After 9/4/00 G - Before 7/6/07 H - After 7/5/07 I - After 7/5/07 Color changed from chrome to silver vein. It is recommended to order in pairs to match J - Includes Screws and Instruction Sheet. K - Due to a change to the Legrest Calf Pad size made on 4/23/12, you must also order the hardware kit (1066591) when ordering replacement Calf Pads.					

Pivot/Slide Tube & Footplate P904 and T934A

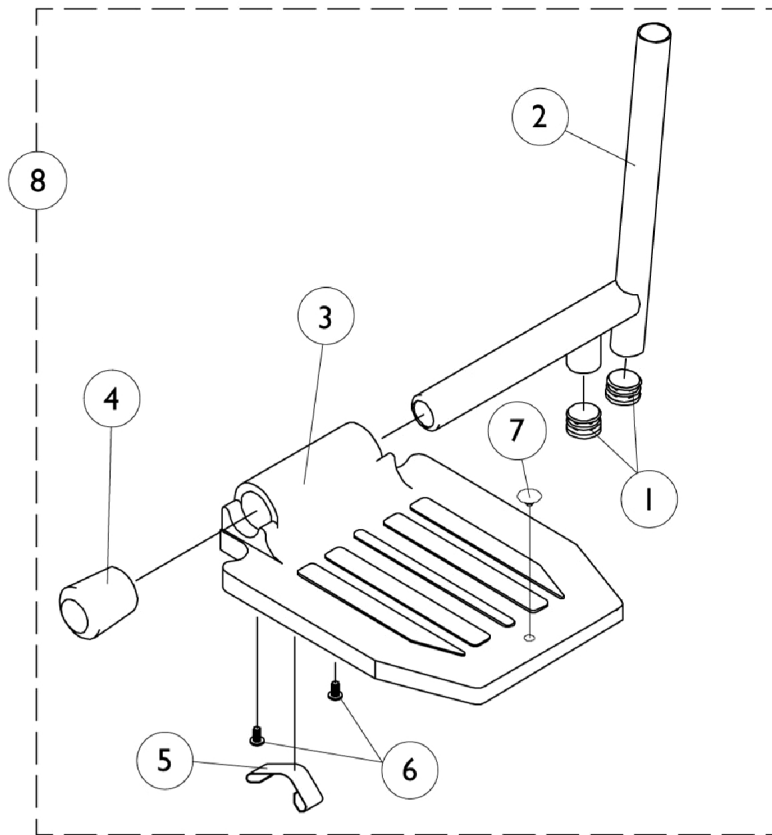

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027097	Plug Button, Black (3/4")	1	2
2		04470X013-NLA	NLA - Pivot/Slide Tube, Chrome	1	1
3		44200X027	Footplate, Aluminum, Medium, Black (7-3/4" W x 6" L)	1	1
4		1016346	Cap, End, Tube, Black (7/8")	1	1
5		1006981	Spring, Tension	1	1
6		7930354-NLA	NLA - Package, Phillips Head Screw (#8-32 x 3/8")	25	1
7		1070168	Plug, Push, Footplate	1	1
8		1017237-NLA	NLA - Pivot/Slide Tube w/ Aluminium Footplate Assembly, Chrome - Right	1	1
8		1017238-NLA	NLA - Pivot/Slide Tube w/ Aluminium Footplate Assembly, Chrome - Left	1	1

MKIV RII Joystick, Controller and Hardware

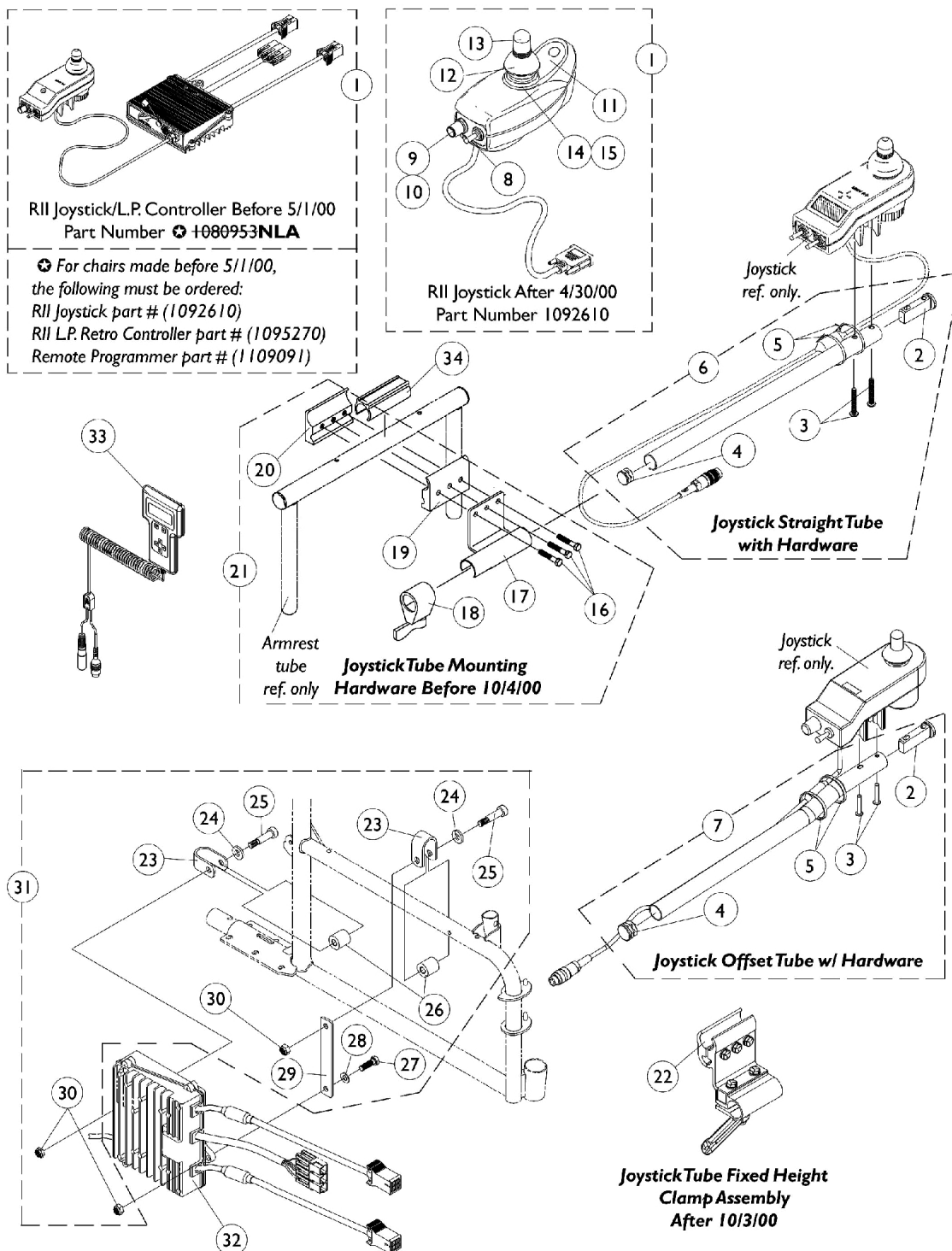


MKIV RII Joystick, Controller and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	B,F	1080953-NLA	NLA - Joystick/Controller, MKIV RII L.P	1	1
1	A,G	1092610-NLA	NLA - Joystick, RII	1	1
2		1093063	Spacer, Joystick Tube, Black	1	1
3		1025714	Screw, Phillips Pan Head (#10-32 x 1-1/8")	1	1
4		1027095	Plug Button, Black (7/8")	1	1
5		1095502-NLA	NLA - Package, Tie Wrap (11-1/2" L)	10	2
6	I	1043568	Kit, Straight Joystick Tube with Hardware	1	1
7	D,D1,P	1093069	Kit, 1/2" Offset Joystick Tube with Hardware	1	1
8		1010042-NLA	NLA - Switch, Toggle SPDT	1	1
9		1025516-NLA	NLA - Knob, Speed Control	1	1
10		1040225-NLA	NLA - Potentiometer, Speed	1	1
11	F	1069204	Decal, Joystick Overlay - RII	1	1
12		1040218	Skirt, Joystick Rubber, Black	1	1
13		1040217	Knob, Joystick Rubber, Black	1	1
14		1036764-NLA	NLA - Inductive, Joystick	1	1
15		50000X364-NLA	NLA - Boot, Joystick	1	1
16		7930349-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/8")	1	1
17		1049582-NLA	NLA - Bracket, Joystick Mounting, Black	1	1
18		1027274	Adjustment Lock Assemnbly, Black	1	1
19		1027266	Half Clamp, Non-Threaded Holes	1	1
20		1027267-NLA	NLA - Half Clamp, Threaded Holes	1	1
21	I	1049591-NLA	NLA - Hardware, Joystick Tube Mounting	1	1
22	D,D1,J, P	1093071	Clamp, Fixed Height Joystick Tube	1	1
23		1069658-NLA	NLA - Bracket, Pivot, Black	1	2
24		1069548-NLA	NLA - Washer, Steel (7/16 x 13/16 x 1/8")	1	2
25		1062604	Screw, Shoulder Hex Socket (3/8 x 1-1/4", 5/16-18)	1	2
26		1069659-NLA	NLA - Bushing	1	1
27		1050161-NLA	NLA - Screw, Hex Head w/ Patch (5/16-18 x 1")	1	1
28		1025704	Washer, Flat (5/16 x 5/8 x 1/16")	1	1
29		1069657-NLA	NLA - Strap, Black	1	1
30		1028769	Locknut (5/16-18)	1	3
31		1069671-NLA	NLA - Hardware, P7E Controller Mounting	1	1
32	A,G	1095270-NLA	NLA - Controller, MKIV RII, Retro L.P.	1	1
33	C	1109091-NLA	NLA - Programmer, Remote	1	1
34	N	1061983-NLA	NLA - Spacer, Plastic Snap Clamp (3/4" I.D. x 2-1/2")	1	1
	K,O	1064404-NLA	NLA - Virtual Service Modem Kit	1	1
	L,O	1091631-NLA	NLA - Virtual Service Software Kit	1	1
	O	1095204-NLA	NLA - Cable, Virtual Service, Charger/Modem	1	1
	O	1091980-NLA	NLA - Disk, 3.5 H.D. Floppy with V.S. Software	1	1
	O	1091637-NLA	NLA - Adaptor, Null Modem	1	1
	O	1091636-NLA	NLA - Service Manual - V.S. Software Kit	1	1
	O	1095204-NLA	NLA - Cable, Virtual Service, Charger/Modem	1	1

MKIV RII Joystick, Controller and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
	M,O	1046117-NLA	NLA - Printer and Interface	1	1
	O	1026572-NLA	NLA - Paper, Printer	1	1
	O	1027280-NLA	NLA - Harness, Printer Wiring	1	1
	O	1058491-NLA	NLA - Connector Boot, Wire	1	1
NOTE: A - After 4/30/00 retro L.P. controller part # (1095270) is sold separately from the joystick. B - Joystick and retro L.P. controller after 4/30/00 serve as direct replacements for chairs made before 5/1/00. C - Remote programmer part #(1109091) with the dual connectors (3-Pin and 5-Pin) is the only programmer functionable with the new RII joystick part #(1092610). D - The joystick straight tube changed to an offset joystick tube on 10/4/00. The joystick tube mounting hardware also changed to a new fixed height clamp assembly to allow an individual to use an D1 - adjustment lever to securely hold the joystick tube in place. The fixed height joystick tube clamp can be used as a direct replacement in the field for the joystick tube mounting hardware part number 1049591 manufactured for chairs before 10/4/00. F - Before 5/1/00 G - After 4/30/00 I - After 10/3/00 J - Includes items 16-20 K - Part number 1095204 is available for individual sale L - Part numbers 1091980, 1091637, 1091636 and 1095204 are available for individual sale. M - Part numbers 1026572 and 1027280 are available for individual sale. N - Spacer allows the half clamps to grip securely around the 3/4" arm tube O - Not Shown P - After 10/3/00					

INT. Controller and Hardware

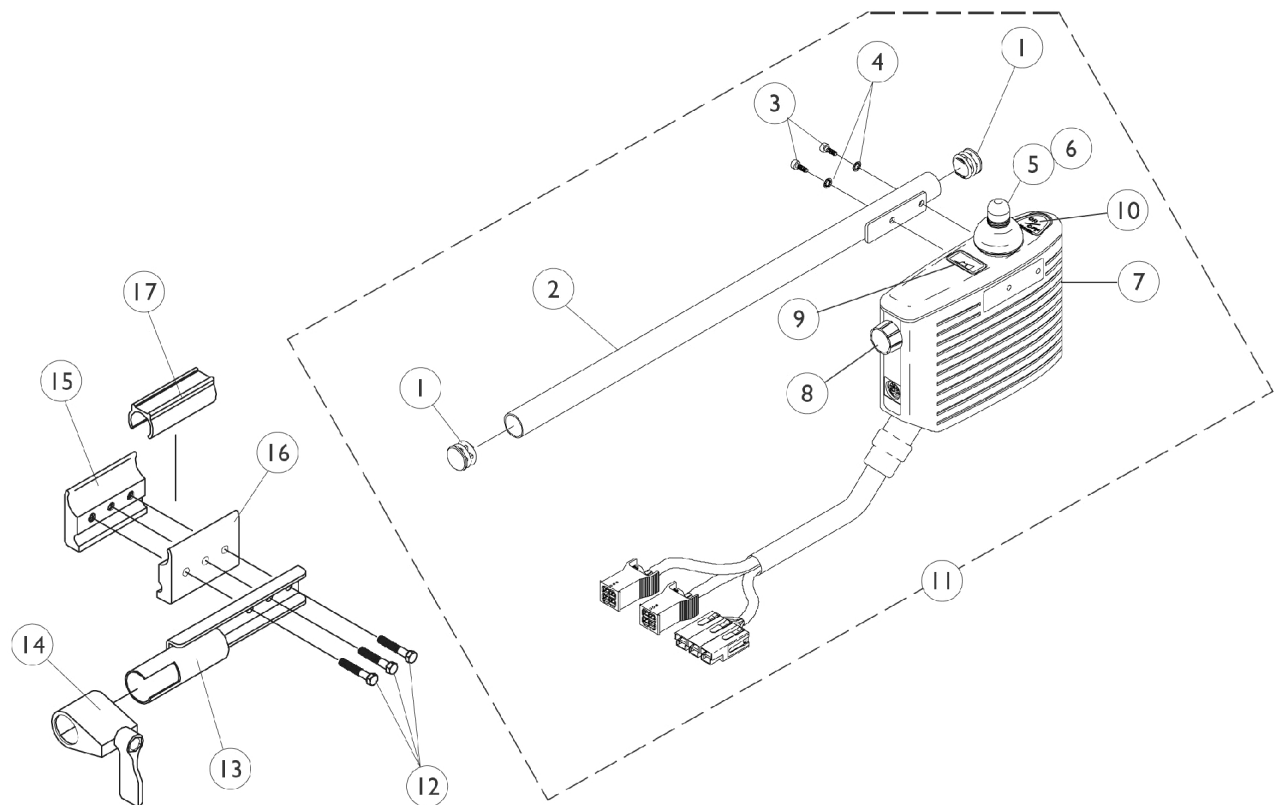

INT. Controller and Hardware

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1027095	Plug Button, Black (7/8")	1	2
2		1056086-NLA	NLA - Tube, Controller Mounting	1	1
3		1056087-NLA	NLA - Screw, Socket Head (#10-32 x 1/2")	1	2
4		1027263-NLA	NLA - Washer, Lock Internal Tooth (#10)	1	2
5		1040217	Knob, Joystick Rubber, Black	1	1
6		1040218	Skirt, Joystick Rubber, Black	1	1
7	B	1060086-NLA	NLA - INT., Non Programmable MKIV	1	1
8		1056559-NLA	NLA - Knob, Speed Control	1	1
9		GLA35137-NLA	NLA - Decal, Battery	1	1
10		GLA35136-NLA	NLA - Decal, On-Off	1	1
11		1056088-NLA	NLA - Controller with Tube	1	1
12		7930349-NLA	NLA - Screw, Hex Head (1/4-20 x 1-1/8")	25	1
13		1035414-NLA	NLA - Tube, Controller Bracket	1	1
14		1027274	Adjustment Lock Assemnbly, Black	1	1
15		1027267-NLA	NLA - Half Clamp, Threaded Holes	1	1
16		1027266	Half Clamp, Non-Threaded Holes	1	1
	A	1058491-NLA	NLA - Connector Boot, Wire	1	1
17	C	1061983-NLA	NLA - Spacer, Plastic Snap Clamp (3/4" I.D. x 2-1/2")	1	1
NOTE: A - Not Shown B - Includes items 5, 6, and 8. C - Spacer allows the half clamps to grip securely around the 3/4" arm tube					

Accessories - Miscellaneous

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
		TAGTS1034751-NLA	NLA - Strap, Positioning, Seat/Chest w/ Airline Style Buckle (Eyelet to Eyelet Length 46")	1	1
		1004310	Strap, Positioning, Seat/Chest w/ Auto Style Push Button Buckle (Eyelet to Eyelet Length 48")	1	1
		1034744	Strap, Positioning, Seat/Chest w/ Hook and Loop Style Buckle (Eyelet to Eyelet Length 45")	1	1
		40176X010-NLA	NLA - Pump, Tire	1	1
	A	8881047959	Pocket, Carrying, Nylon (16 - 20" W Chair)	1	1
		1063287-NLA	NLA - Extension Handle Assembly, Clutch (Pair)	2	1
		1058899	Wheel Lock Extension Handle Assembly, Black (6")	1	2
		1058900	Wheel Lock Extension Handle Assembly, Black (8")	1	2
	F	1143050	Non-Removable Device, Footrest/Legrest (1820)	1	1
			NLA - Non-Removable Device, Footrest-Legrest Carton	1	1
		1032002	Strap, "H" Heel	1	1
		1035902	Strap, Calf w/ Hook and Loop (31" L)	1	1
	A,B	8881006725-NL A	NLA - Panel, Legrest	1	1
	A,C	8881006726	Panel, Legrest	1	1
	A,B	8881035816-NL A	NLA - Panel, Legrest	1	1
	A,C	8881035817-NL A	NLA - Panel, Legrest	1	1
		1035917-NLA	NLA - Heel Loop (Pair)	2	1
		1017621	Heel Loop - Right/Left	1	1
	D	1035907	Kit, Heel Loop Attaching Hardware (Pair)	2	1
	D	04572X025	Toe Loop (Single)	1	1
	D	1037087	Toe Loops without Hardware (Pair)	2	1
	E	1037089	Hardware, Toe Loop (Pair)	2	1
NOTE: A - Specify upholstery color when ordering. See page titled Frame Finishes, Touch-Up Paints and Upholstery Color page. B - For 16" wide chairs C - For 18" wide chairs D - Fits all sizes E - Includes 4 ea. Screw, Phillips Pan Head (10-32 x 7/8"); 4 ea. Washer (1/4 x 3/4 x 3/32"); 4 ea. Locknut (10-32) F - Includes Device, Non-Removable and Carton.					

Power Wheelchair Tool Kit

I



**Multimeter
with Leads**



**Grounding
Wire**



**1" Test
Leads**



**Protractor/Angle
Locator**



**Invacare
Pen Light**



**Invacare
Tape Measure**



**Invacare
Phillips/Flat Head
Screwdriver**



MKIV Programmer



**Virtual Service
Charger Cable**

Power Wheelchair Tool Kit

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1125222-NLA	NLA - Kit, Power Wheelchair Tool Includes: 1 ea. Multimeter with Leads 1 ea. Locator, Protractor/Angle Magnetic Base 1 ea. Instructions, MKIV Programmer 1 ea. Wire, Grounding with Hand Strap and Clip 1 ea. Leads, 1" Test 1 ea. Light, Pen with Invacare Logo 1 ea. Tape Measure, Compact with Invacare Logo 1 ea. Screwdriver, Pen-Size Phillips/Flat Head 1 ea. Cable, Virtual Service Charger/Modem	1	1

Warranty Replacement Parts

Part Number	Description	QTY per Assembly
1123249E-NLA See Note(s) A	WARR RPLMT - Charger, Battery, 8 Amp. 115vac (Off Board) Single Mode Round Plug w/ AC Power Line Cord, CTE Corp.	1
NOTE: A - See "Battery Boxes - Chargers - Harnesses - Trays" Category for more information.		

Invacare Corporation

www.invacare.com



Yes, you can.

One Invacare Way
Elyria, Oh USA
44036-2125
800-333-6900

Technical Support
800-832-4707
Customer Service
800-333-6900

Customer Service
800-333-6900

© 2022 Invacare Corporation. All rights reserved. Republication, duplication or modification in whole or in part is prohibited without prior written permission from Invacare. Trademarks are identified by ™ and ®. All trademarks are owned by or licensed to Invacare Corporation or its subsidiaries unless otherwise noted. (Revised 05/18/22)

Form No. 95-167