
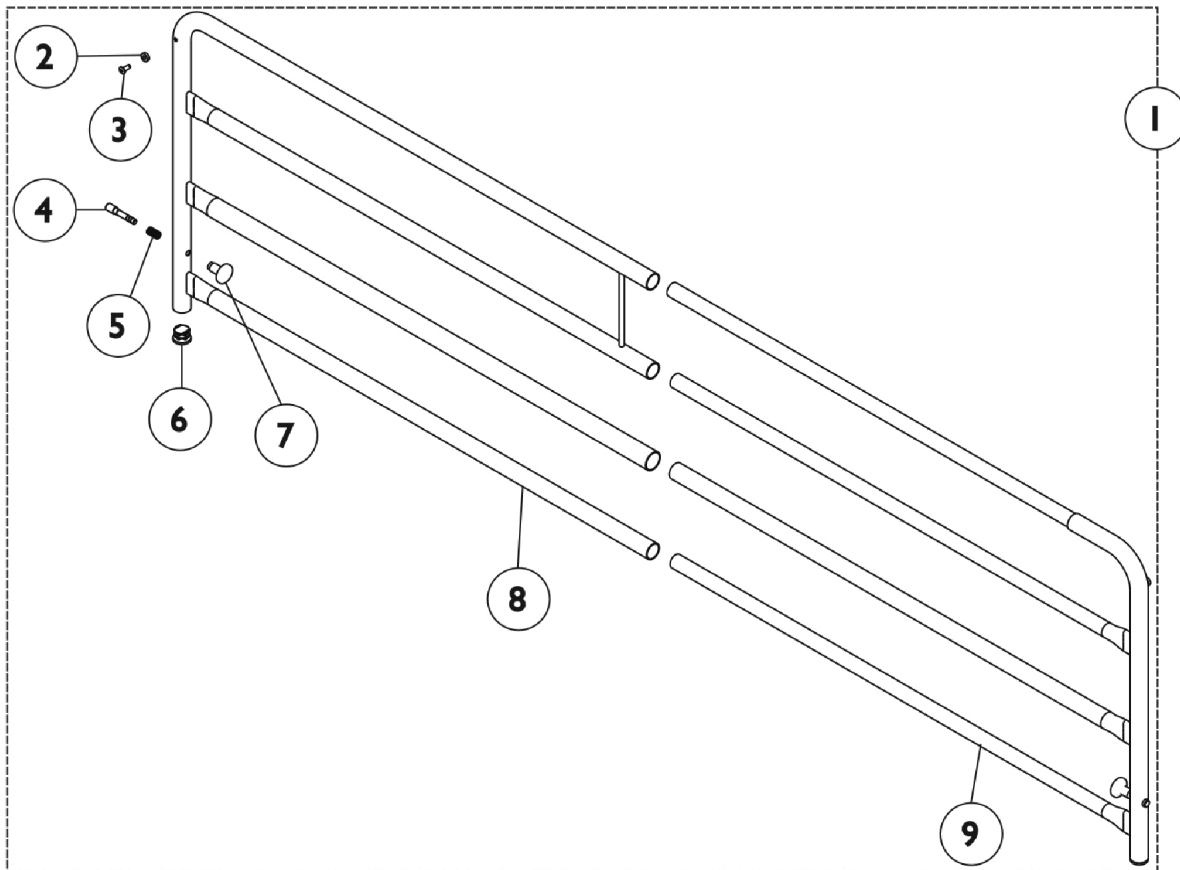
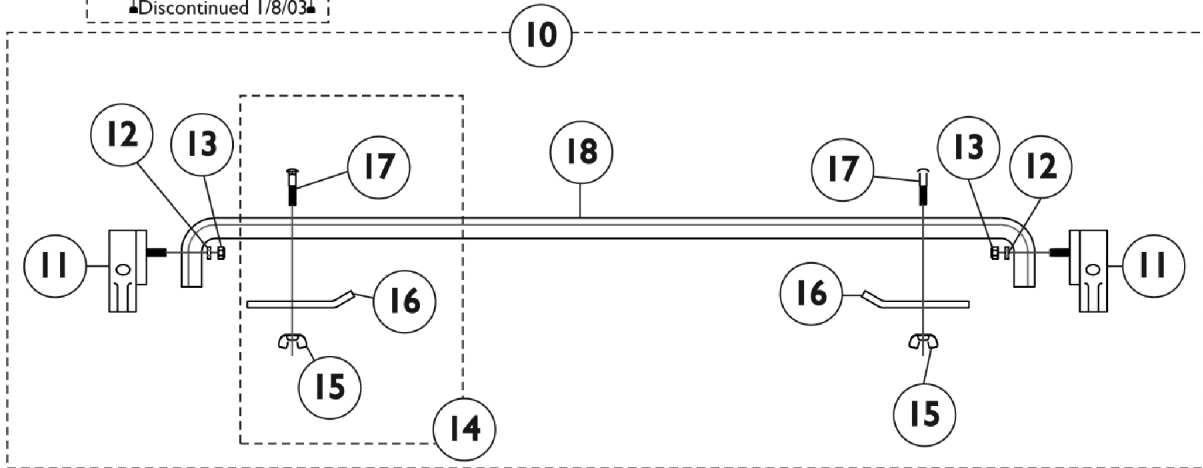


Standard Telescope Bed Rails 6623

This Model was discontinued 1/8/03

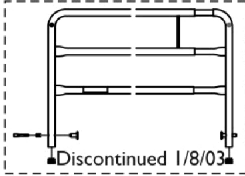

Standard Telescope Bed Rails 6623

This Model was discontinued 1/8/03

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1135777-NLA	NLA - Kit, Telescoping Bed Rail	1	2
2			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	2
3			NLA - Rivet (3/16 x 7/16")	1	2
4			NLA - Pin, Plunger	1	2
5			Spring, Plunger (23/64 x 5/8")	1	2
6			Plug Button, Grey (7/8")	1	2
7			Knob, Plunger, Grey	1	2
8			NLA - Rail, Telescoping - Outside	1	1
9			NLA - Rail, Telescoping - Inside	1	1
			Label, Entrapment Warning	1	1
10	B	1021834-NLA	NLA - Kit, Crossbrace Assembly	1	2
11			Barrel, Right	1	1
11			Barrel, Left	1	1
12			Washer, Flat (5/16 x 11/16 x 1/16")	1	2
13			Locknut (5/16-18)	1	2
15			Nut, Wing (1/4-20)	1	2
16			NLA - Clamp Bar	1	2
17			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
18			Channel Only, Granite	1	1
14	C	1028789-NLA	NLA - Kit, Frame Bar (Pair)	1	1
15			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
16			NLA - Clamp Bar	1	2
17			Nut, Wing (1/4-20)	1	2
	D,E	1012658-NLA	NLA - Spacer, Tube	1	2

NOTE: A - Includes: Items 2-9 and the Warning Label
 B - Includes: Items 11-18
 C - Includes: Items 15-17
 D - Before 8/11/97 only.
 E - Not Shown

2/1/04 the rails will have 4 rungs instead of 3 so an inner and an outer Rail needs to be ordered when replacing.