


Standard Quick Clip-On Bed Rails 6627

This item was discontinued 2/16/96

Model 6627 (Discontinued)

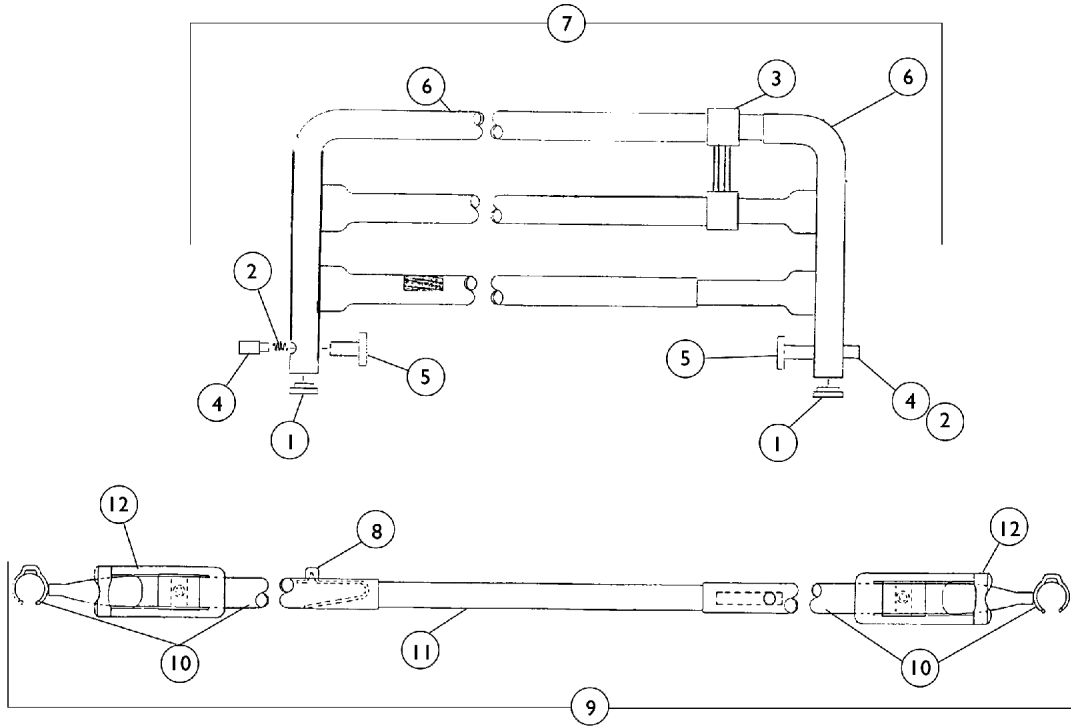

Standard Quick Clip-On Bed Rails 6627

This item was discontinued 2/16/96

| Item Number | Note | Part Number | Description | Quantity Per | |
|-------------|------|-------------|--|--------------|----------|
| | | | | Package | Assembly |
| 1 | | 1027096 | Plug Button, Grey (7/8") | 1 | 4 |
| 2 | | 19000X023 | Spring, Plunger (23/64 x 5/8") | 1 | 4 |
| 3 | | 1012658-NLA | NLA - Spacer, Tube | 1 | 2 |
| 4 | | 1012660-NLA | NLA - Pin, Plunger | 1 | 4 |
| 5 | | 1012656 | Knob, Plunger, Grey | 1 | 4 |
| 6 | | 1012669 | Rail, Telescoping - Outside | 1 | 2 |
| 6 | | 1012670 | Rail, Telescoping - Inside | 1 | 2 |
| 7 | A | 1013305-NLA | NLA - Telescoping Rail Assembly | 1 | 2 |
| 8 | | 40188X009 | Button, Single Snap | 1 | 4 |
| 9 | B | 1023490-NLA | NLA - Crossbrace Assembly | 1 | 2 |
| 10 | | 1022789-NLA | NLA - Tube, Channel with Barrel - Left Head End / Right Foot End | 1 | 2 |
| 10 | | 1022790-NLA | NLA - Tube, Channel with Barrel - Left Foot End / Right Head End | 1 | 2 |
| 11 | | 1022842-NLA | NLA - Tube, Channel - Center | 1 | 2 |
| 12 | | 1022652-NLA | NLA - Clip, Spring | 1 | 2 |

NOTE: A - Includes items 1-6
B - Crossbrace Assembly is no longer available. May be replaced with crossbrace from model 6629.