
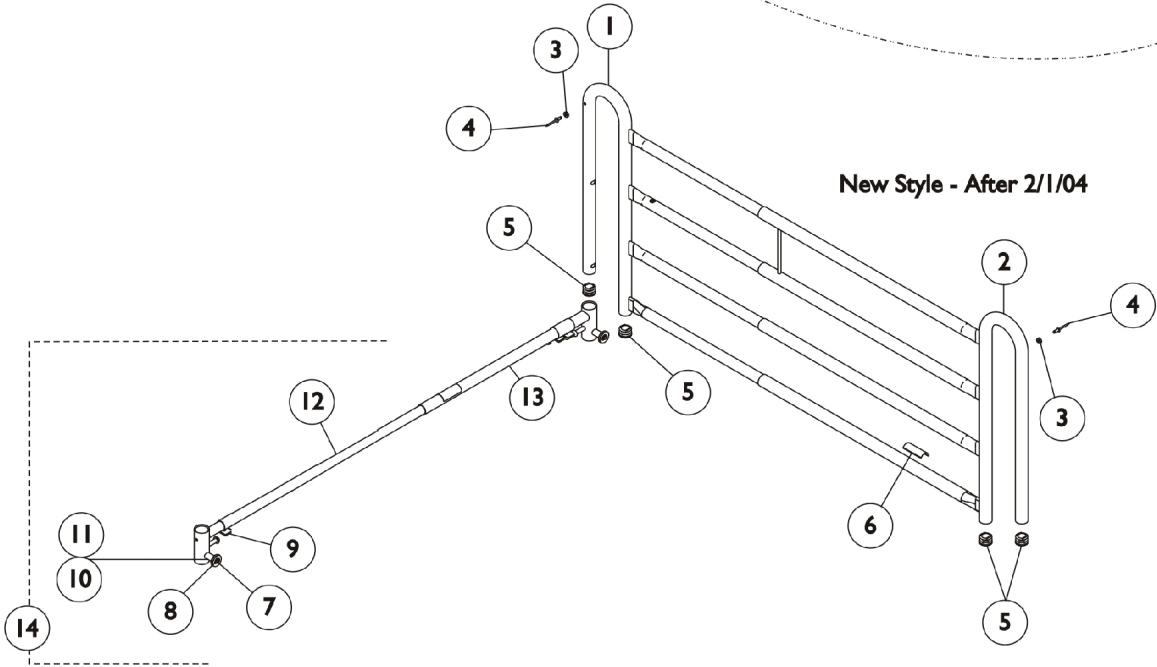


Deluxe Spring Loaded Bed Rails 6628 & 6628-TSS

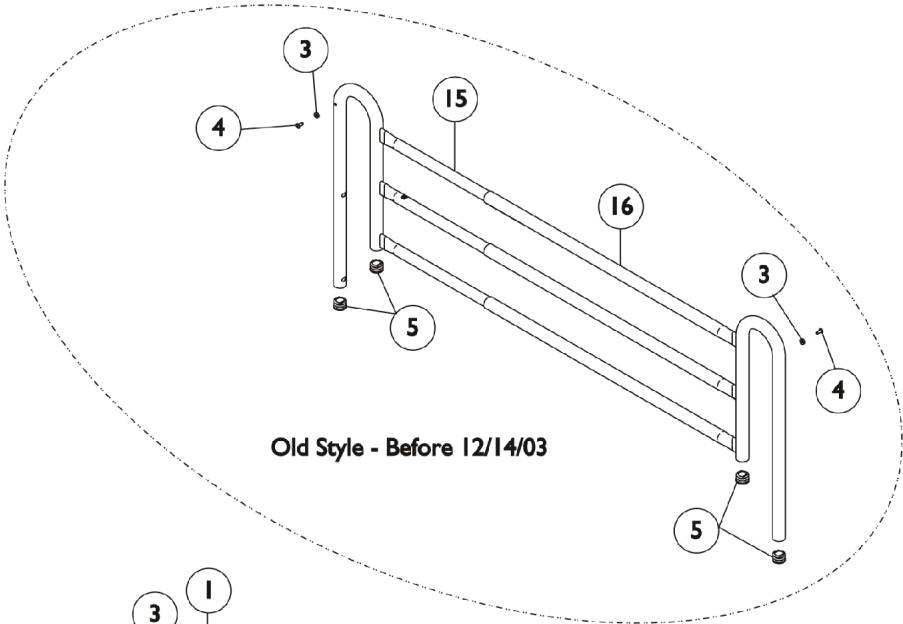

Deluxe Spring Loaded Bed Rails 6628 & 6628-TSS

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	F	1121932-NLA	NLA - Rail, Inside	1	1
2	F	1121933-NLA	NLA - Rail, Outside	1	1
3		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	2
4		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	2
5		1057287	Plug Button, Grey (1")	1	8
6		1122149-NLA	NLA - Label, Warning - Reduced Gap	1	1
7		40314X001-NLA	NLA - Ring, Retaining	1	4
8		40014X020-NLA	NLA - Knob, Plunger	1	4
9		1110764-NLA	NLA - Clip, Rail - Small	1	2
9		1110765-NLA	NLA - Clip, Rail - Large	1	2
12		1106232-NLA	NLA - Tube, Channel with Barrel - Inside	1	1
12	G	1110951-NLA	NLA - Tube, Channel with Barrel, Chrome - Inner Deluxe	1	1
13		1106234-NLA	NLA - Tube, Channel with Barrel, Outside	1	1
13		1110954-NLA	NLA - Tube, Channel with Barrel, Chrome - Outer Deluxe	1	1
14	A,D	1106235-NLA	NLA - Crossbrace Assembly, Chrome	1	2
14	A,C	1110955-NLA	NLA - Crossbrace Assembly, Deluxe, Wide	1	2
15		16320X007-NLA	NLA - Rail, Inside	1	1
16		16320X006-NLA	NLA - Rail, Outside	1	1
	B,E	1110957-NLA	NLA - Kit, Bed Rail Hardware	1	1

NOTE: A - Includes Items 3-12 and the Bed Rail Hardware Kit.
 B - Not Shown
 C - To be used with a TSS Mattress Only. Crossbraces are extra wide to accommodate the mattress.
 D - Both old style and new style models use this crossbrace.
 E - Includes the Wing Screws (4) to hold the X-Brace in place on the Bed Frame.
 F - 2/1/04 the rails will have 4 rungs instead of 3 so an inner and an outer Rail needs to be ordered when replacing.
 G - No longer available as a service part individually. Order item 14 part #1110955 as a replacement.