
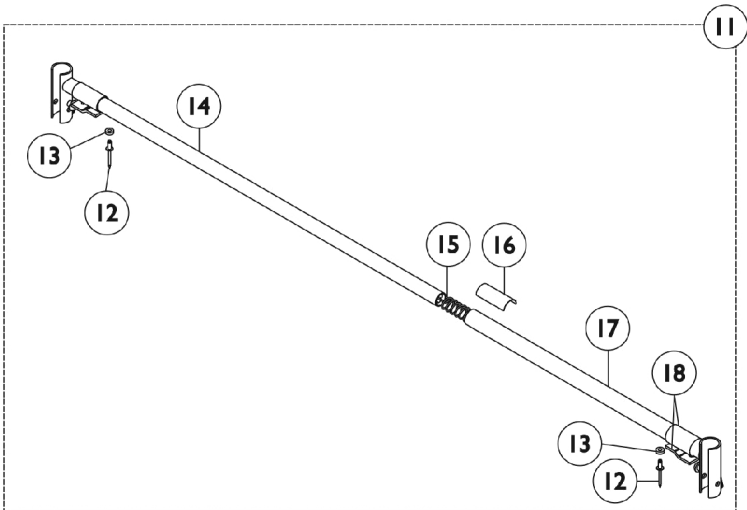
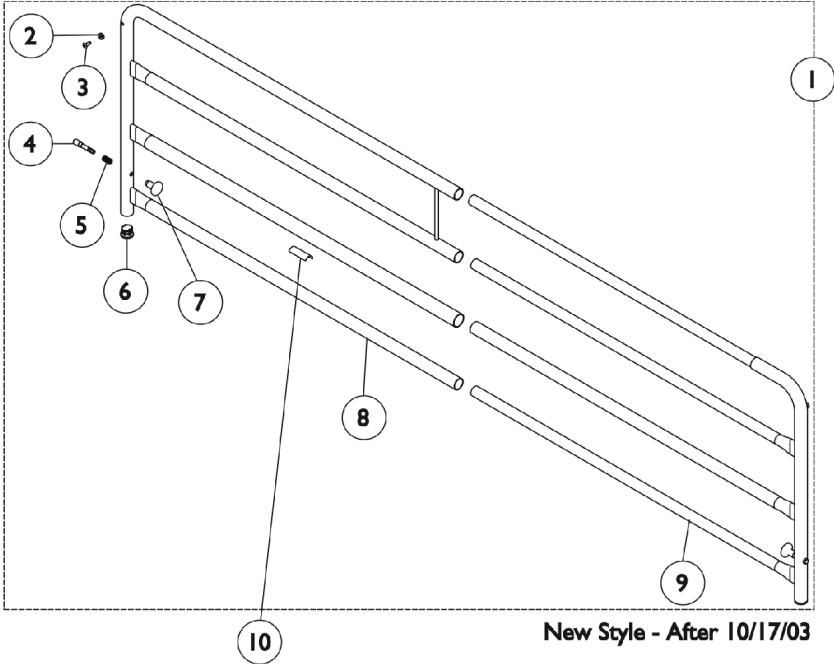


# Standard Spring Loaded Bed Rails 6629

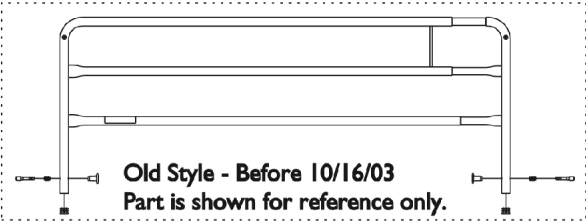

## Standard Spring Loaded Bed Rails 6629

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A,C	1135777-NLA	NLA - Kit, Telescoping Bed Rail	1	2
2			NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	2
3			NLA - Rivet (3/16 x 7/16")	1	2
4			NLA - Pin, Plunger	1	2
5			Spring, Plunger (23/64 x 5/8")	1	2
6			Plug Button, Grey (7/8")	1	2
7			Knob, Plunger, Grey	1	2
8			NLA - Rail, Telescoping - Outside	1	1
9			NLA - Rail, Telescoping - Inside	1	1
			Label, Entrapment Warning	1	1
10		1122149-NLA	NLA - Label, Warning - Reduced Gap	1	1
11	B	1108836-NLA	NLA - Crossbrace Assembly, Chrome	1	2
12		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	4
13		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	4
14	E	1108832-NLA	NLA - Tube, Channel w/ Barrel - Inside	1	2
15		1047101-NLA	NLA - Spring, Compression with Hook End	1	2
16		1011239-NLA	NLA - Label, Caution	1	2
17		1108835-NLA	NLA - Tube, Channel with Barrel - Outside	1	2
18		1047096-NLA	NLA - Clip, Crossbrace	1	4
19	D	1110957-NLA	NLA - Kit, Bed Rail Hardware	1	1

NOTE: A - Includes: Items 2-9 and the Warning Label  
 B - Includes: Items 12-18  
 C - As of 10/17/03 the rails have 4 rungs instead of 3 so an inner and an outer Rail needs to be ordered when replacing.  
 D - Not Shown.  
 E - No longer available as an individual service part, use Item 11 (1108836) as its replacement.