


Half Length Bed Rails - 6630

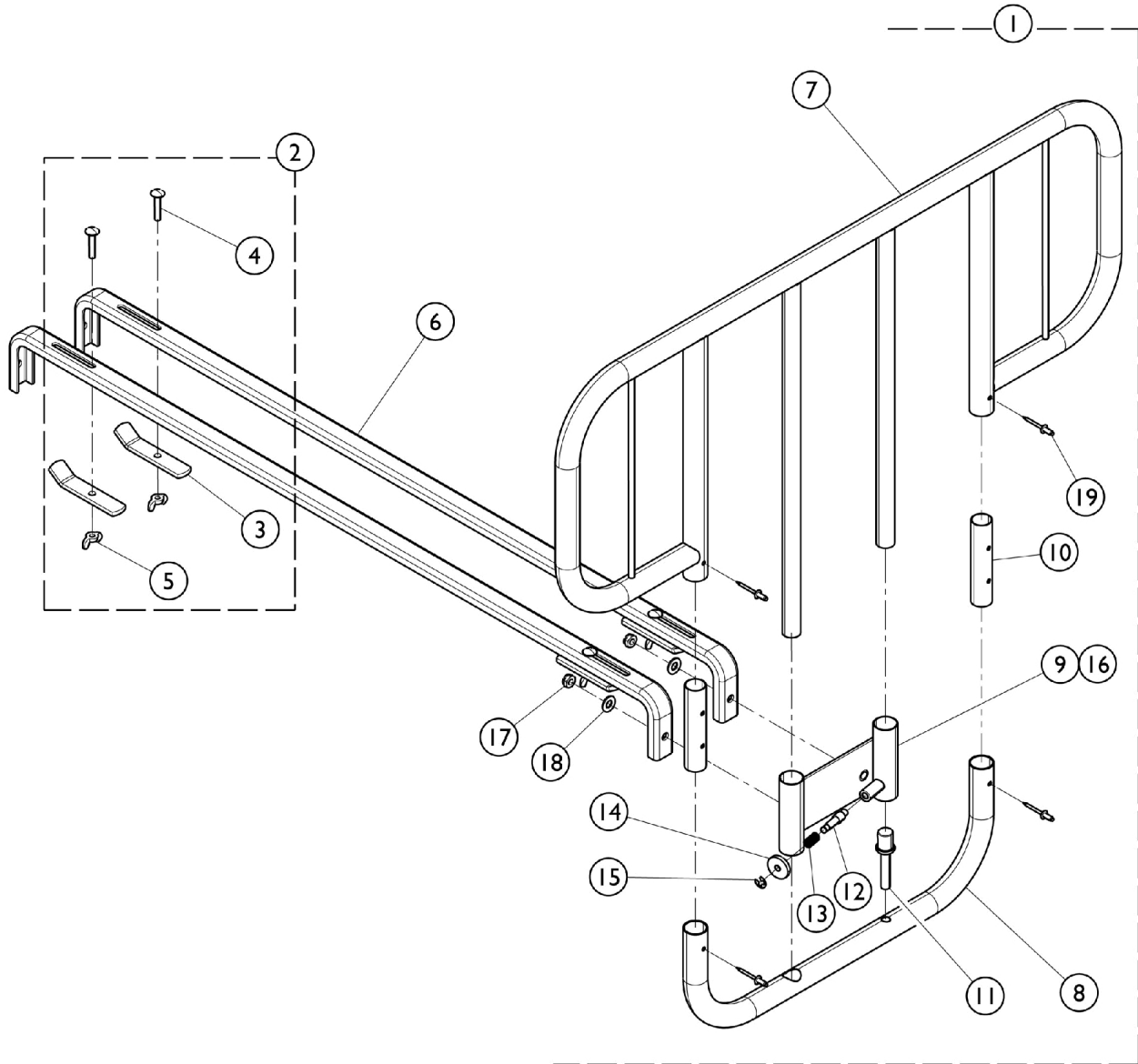

Half Length Bed Rails - 6630

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	16364X000-NLA	NLA - Half Rail Assembly	1	2
2	B	1028789-NLA	NLA - Kit, Frame Bar (Pair)	1	1
3			Bolt, Carriage (1/4-20 x 1-1/4")	1	2
4			NLA - Clamp Bar	1	2
5			Nut, Wing (1/4-20)	1	2
6		1007727	Channel Only, Granite	1	2
7		16357X000-NLA	NLA - Rail, Top	1	2
8		16362X000-NLA	NLA - Rail, Bottom	1	2
9		16346X000-NLA	NLA - Bracket, Slide, Chrome	1	2
10		16363X000-NLA	NLA - Bushing (4")	1	2
11		16335X000	Insert	1	2
12		19000X012	Pin, Plunger	1	2
13		19000X023	Spring, Plunger (23/64 x 5/8")	1	2
14		40014X020-NLA	NLA - Knob, Plunger	1	1
15		40314X001-NLA	NLA - Ring, Retaining	1	1
16		16336X000	Bushing, Composite, Nylon (25/32 x 29/32 x 3/4")	1	6
17	D	60290X002	Locknut (5/16-18)	1	4
18	D	16308X000	Washer, Flat (5/16 x 11/16 x 1/16")	1	4
19		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	8
	C	1016386-NLA	NLA - Instruction Sheet, 6630 Bedrail	1	1

NOTE: A - Includes items 7 - 19.
 B - Includes items 3 - 5.
 C - Not shown in illustration.
 D - Used to mount Item 6 to Item 9, mounted to studs already welded to Item 9 on the opposite side from the view in the illustration. Refer to Instruction Sheet for more detail.