


Economy Telescoping Bed Rail 6650

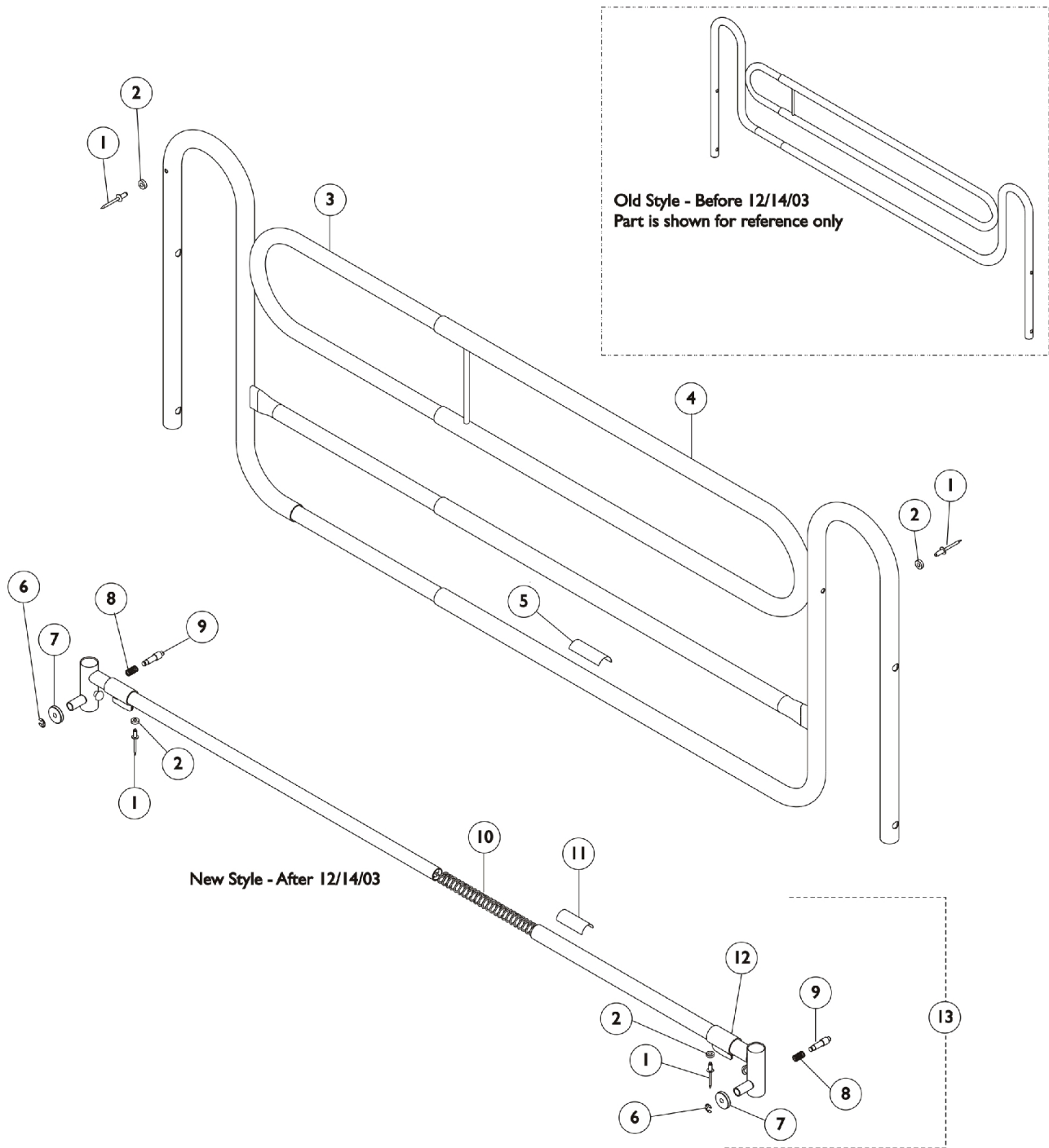

Economy Telescoping Bed Rail 6650

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		16348X000-NLA	NLA - Rivet (3/16 x 7/16")	1	2
2		16347X000-NLA	NLA - Washer, Nylon (3/16 x 7/16 x 1/8")	1	2
3	A	1121941-NLA	NLA - Rail, Telescoping - Inside	1	1
4	A	1121940-NLA	Rail, Telescoping - Outside	1	1
5		1122149-NLA	NLA - Label, Warning - Reduced Gap	1	1
6		40314X001-NLA	NLA - Ring, Retaining	1	2
7		40014X020-NLA	NLA - Knob, Plunger	1	2
8		19000X023	Spring, Plunger (23/64 x 5/8")	1	2
9		19000X012	Pin, Plunger	1	2
10		1047101-NLA	NLA - Spring, Compression with Hook End	1	1
11		1011239-NLA	NLA - Label, Caution	1	1
12		1047096-NLA	NLA - Clip, Crossbrace	1	2
13		1106244-NLA	NLA - Crossbrace Assembly	1	2

NOTE: A - After 2/1/04 the rails will have 4 rungs instead of 3 so an inner and an outer Rail needs to be ordered when replacing.