Reduced Gap Heavy Duty Half-Length Bed Rails

Assembly, Installation and Operating Instructions

Model No. 6640

Yes, you can:

SAVE THESE INSTRUCTIONS

NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact your Dealer/Carrier for further instruction.

NOTE: A brochure entitled "A Guide to Bed Safety; Bed Rails in Hospitals, Nursing Homes and Home Health Care: The Facts" can be found on the FDA's website at <http://www.fda.gov/cdrh/beds/>.

SAFETY SUMMARY

In order to ensure the safe installation, assembly and operation of the Bed Rails, these instructions **MUST** be followed:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

Invacare recommends the use of two (2) sets of half length bed rails: one (1) set at the **HEAD END** of the bed, and one (1) set at the **FOOT END** of the bed where permitted.

When used with a manual/electrical bed, the bed rails DO NOT fall under any weight limitations. Bed rails can be deformed or broken if excessive side pressure is exerted on the bed rails. These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are NOT intended nor may be used for restraint purposes. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.

Replacement mattresses and bed rails with dimensions **DIFFERENT** than the original equipment supplied or specified by the bed manufacturer are **NOT** interchangeable. Variations in bed rail design, width and thickness or firmness of the mattress could cause/contribute to entrapment. Use **ONLY** authorized Invacare replacement parts.

DO NOT use the bed rails as push handles when moving the bed.

This bed rail is **NOT** a handhold for getting into or out of bed.

INSTALLATION AND OPERATION WARNINGS

After **ANY** adjustments, repair or service and **BEFORE** use, make sure all attaching hardware is tightened securely.

Always test to make sure that the bed rails are properly and securely in place **BEFORE** using.

On **ALL** beds **EXCEPT** the **Bariatric Bed**, the **HEAD-END** and **FOOT-END** crossbraces **MUST** be secured **BETWEEN THE FOURTH AND FIFTH** springs on the bed frame for installation stability. On the **Bariatric Bed**, the **SIDE ADJUSTMENT CLAMPS** **MUST** be secured in the bed rail mounting holes in the **HEAD** and **FOOT** ends of the pan deck for installation stability.

Invacare recommends that the mattress be centered on the bed frame. Otherwise, entrapment may occur between the bed rail and the bed frame.

INSTALLATION AND OPERATION WARNINGS (CONTINUED)

After raising/lowering the head/foot end of the manual/electric bed, check the distance between the bottom of the bed rail and the mattress. If there is excessive distance between the bottom of the bed rail and the mattress in which entrapment may occur, adjust the height of the bed rail (if applicable), OR provide alternative means of patient protection.

Unless the bed rail is in the lowest position, ensure that the plunger pins fully protrude through the adjustment hole of each rail tube. This ensures that the bed rail is securely locked in position.

Because of the heavier weight limitations associated with the **Bariatric Bed**, use the mounting brackets attached to the bed frame to assemble the 6640 half bed rail side adjustment clamps. These side adjustment clamps are designed to support heavier weight limitations and **MUST** be used when installing half length bed rails onto the **Bariatric Bed**. **DO NOT** use the chrome colored side adjustment clamps supplied with the half length bed rail kits, as these will **NOT** support heavier weight limitations and injury or damage may result.

INSTALLING THE CROSSBRACES

NOTE: Refer to INSTALLATION AND OPERATION WARNINGS in the SAFETY SUMMARY of this instruction sheet.

ALL BEDS EXCEPT THE BARIATRIC BED (FIGURE 1)

1. Remove the mattress from the bed.
2. Raise the HEAD END of the bed.
3. Remove hardware kit labeled HOME CARE BED HARDWARE KIT located in the 6640 bed rail box and set aside for use in installing the crossbraces.
4. Remove hardware kit labeled BARIATRIC BED HARDWARE KIT BAR600 located in the 6640 bed rail box and discard.
5. Position the bed rail bracket under the bed frame lip, between the FOURTH AND FIFTH springs counting from the HEAD END of the bed (DETAIL "A").
6. Position the side adjustment clamp assembly on the outside of the bed frame, between the FOURTH AND FIFTH springs counting from the HEAD END of the bed as shown in (DETAIL "A").
7. Secure the side adjustment clamp assembly to the bed rail bracket by doing the following:
 - A. Install two (2) LONG bolts, washers and locknuts, making sure the LONG bolts pass through the holes on the SIDE of the side adjustment clamp assembly and the bed rail bracket (DETAIL "A").
 - B. Install one (1) LONG bolt, washer and locknut, making sure the LONG bolt passes through the holes in the TOP of the side adjustment clamp assembly and bed rail bracket (DETAIL "A").
8. Repeat STEPS 5-7 for the opposite side of the bed frame.
9. Secure the bed rail brace to the two (2) side adjustment clamp assemblies using the SHORT bolts, washers, and locknuts (DETAIL "B").

NOTE: When positioning half length bed rails at the foot end of the bed, position the rails between the **FOURTH AND FIFTH** springs of the bed frame counting from the **FOOT END** of the bed.

10. When placing half length bed rails on the **FOOT END** of the bed, raise the **FOOT END** of the bed.
11. Position the bed rail bracket under the bed frame lip, between the **FOURTH AND FIFTH** springs counting from the **FOOT END** of the bed (DETAIL "A").
12. Position the side adjustment clamp assembly on the outside of the bed frame, between the **FOURTH AND FIFTH** springs counting from the **FOOT END** of the bed as shown in (DETAIL "A").
13. Secure the side adjustment clamp assembly to the bed rail bracket by doing the following:
 - A. Install two (2) **LONG** bolts, washers and locknuts, making sure the **LONG** bolts pass through the holes on the **SIDE** of the side adjustment clamp assembly and the bed rail bracket (DETAIL "A").
 - B. Install one (1) **LONG** bolt, washer and locknut, making sure the **LONG** bolt passes through the holes in the **TOP** of the side adjustment clamp assembly and bed rail bracket (DETAIL "A").
14. Repeat **STEPS 11-13** for the opposite side of the bed frame.
15. Secure the bed rail brace to the two (2) side adjustment clamp assemblies using the **SHORT** bolts, washers, and locknuts (DETAIL "B").
16. Position the bed in the flat position and place the mattress on the bed frame.
17. Install the half length bed rails. Refer to **INSTALLING/RAISING/LOWERING THE SIDE RAILS** in this instruction sheet.

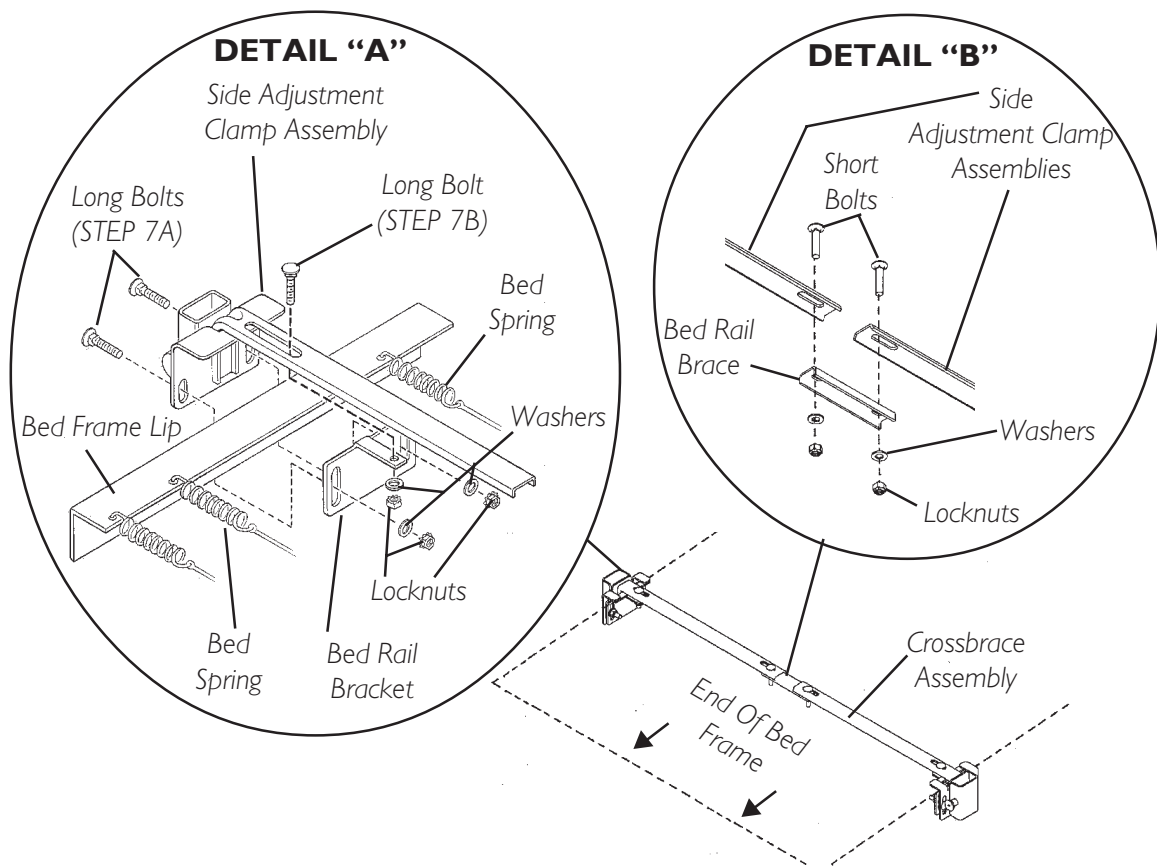
FIGURE I - INSTALLING THE CROSSBRACE ASSEMBLY - ALL BEDS EXCEPT THE BARIATRIC BED

BARIATRIC BED (FIGURE 2)

NOTE: The Bariatric Bed uses half length bed rails (model number 6640) ONLY.

WARNING

Because of the heavier weight limitations associated with the Bariatric Bed, use the mounting brackets attached to the bed frame to assemble the 6640 half bed rail side adjustment clamps. These side adjustment clamps are designed to support heavier weight limitations and **MUST be used when installing half length bed rails onto the Bariatric Bed. **DO NOT** use the chrome colored side adjustment clamps supplied with the half length bed rail kits, as these will **NOT** support heavier weight limitations and injury or damage may result.**

1. Remove the mattress from the bed.
2. Raise the HEAD END of the bed.
3. Remove hardware kit labeled BARIATRIC BED HARDWARE KIT BAR600 located in the 6640 bed rail box and set aside for use in installing the crossbraces.
4. Remove hardware kit labeled HOME CARE BED HARDWARE KIT located in the 6640 bed rail box and discard.
5. Locate the two (2) mounting holes on the TOP of the pan deck by counting eight (8) rows of holes from the head end of the bed AND two (2) holes in from each side of the bed.
6. Locate the two (2) mounting holes in each of the two (2) bed rail mounting brackets on the sides of the bed frame.
7. Align one (1) mounting hole on the TOP of one (1) side adjustment clamp with one (1) of the mounting holes in STEP 5 (DETAIL "A").
8. Secure the side adjustment clamp to the TOP of the pan deck using one (1) LONG bolt and one (1) locknut. Hand tighten ONLY (DETAIL "A").
9. Align the two (2) mounting holes on the side of the side adjustment clamp with two (2) of the mounting holes of the bed rail mounting bracket (DETAIL "B").
10. Secure the side adjustment clamp to the bed rail mounting bracket using two (2) SHORT bolts, spacers and locknut (DETAIL "B"). Hand tighten ONLY.
11. Securely tighten all hardware.
12. Repeat STEPS 7-11 for the opposite side of the bed.

NOTE: When positioning half length bed rails at the foot end of the bed, begin counting from the FOOT end of the bed.

13. When placing half length bed rails on the FOOT END of the bed, raise the FOOT END of the bed and repeat STEPS 5-12.
14. Position the bed in the flat position and place the mattress on the bed frame.
15. Install the half length bed rails. Refer to INSTALLING/RAISING/LOWERING THE SIDE RAILS in this instruction sheet.

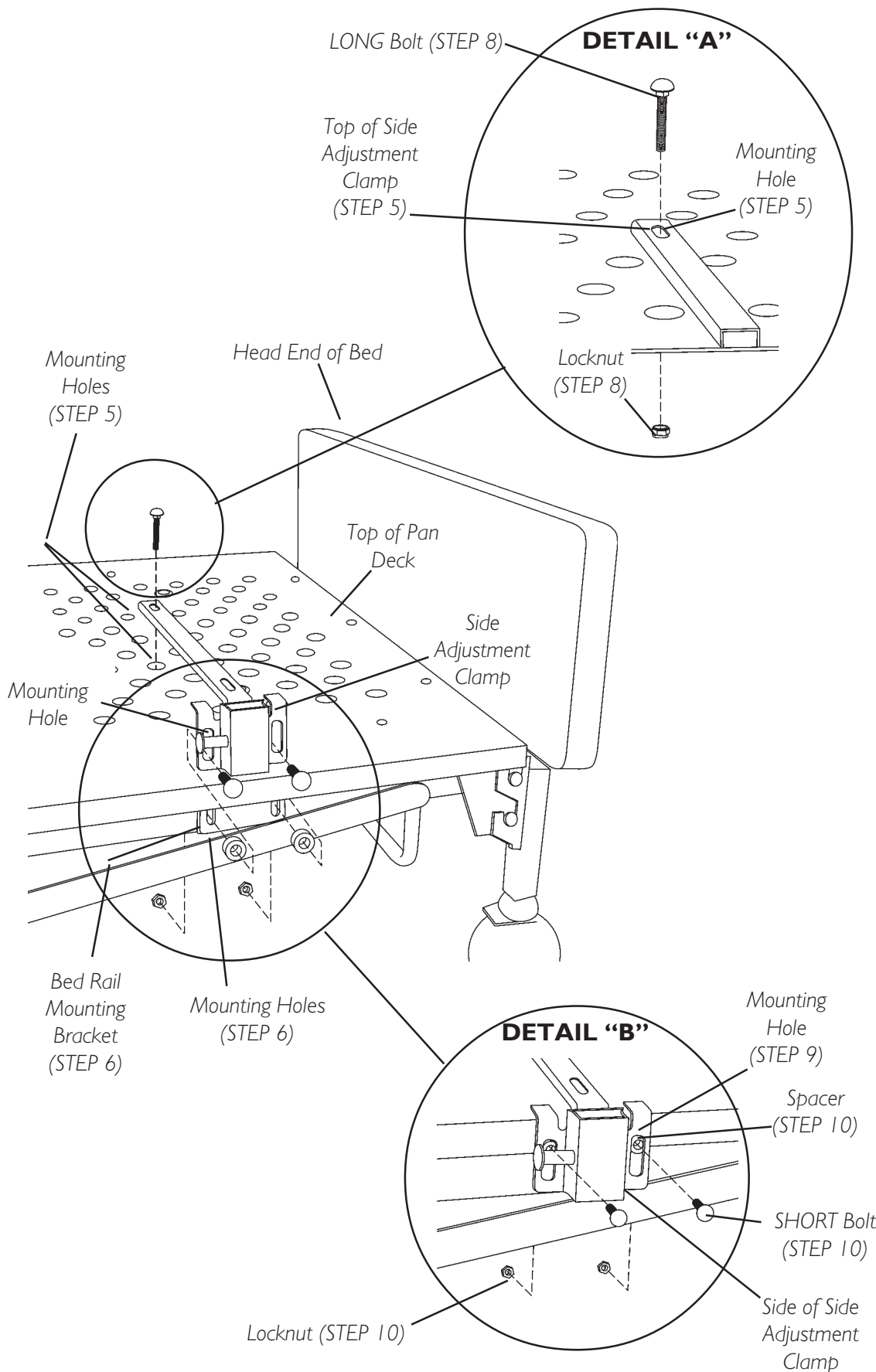
FIGURE 2 - INSTALLING THE CROSSBRACE ASSEMBLY - BARIATRIC BED

INSTALLING/RAISING/LOWERING THE SIDE RAILS (FIGURE 3)

WARNING

Invacare recommends that the mattress be centered on the bed frame. Otherwise, entrapment may occur between the bed rail and the bed frame.

When used with a manual/electrical bed, the bed rails **DO NOT** fall under any weight limitations. Bed rails can be deformed or broken if excessive side pressure is exerted on the bed rails. These bed rails are used for the purpose of preventing an individual from inadvertently rolling out of bed. The bed rails are **NOT** intended nor may be used for restraint purposes. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.

After raising/lowering the head/foot end of the manual/electric bed, check the distance between the bottom of the bed rail and the mattress. If there is excessive distance between the bottom of the bed rail and the mattress in which entrapment may occur, adjust the height of the bed rail (if applicable), **OR** provide alternative means of patient protection.

1. Pull and hold the plunger pin on the side adjustment clamp assembly.
2. Position the half length bed rail in the half length bed rail barrel.
3. Release the plunger pin on the side adjustment clamp assembly.
4. Raise the half length bed rail until the plunger pin “clicks” securely into the adjustment hole, or lower the half length bed rail until the top support of the half length bed rail rests securely against the top of the half length bed rail barrel.
5. Repeat STEPS 1-5 to install the other half bed rail(s).
6. To raise or lower the half length bed rail, hold the half length bed rail, while pulling the plunger pin and repeat STEP 4 for **BOTH** half length bed rails.

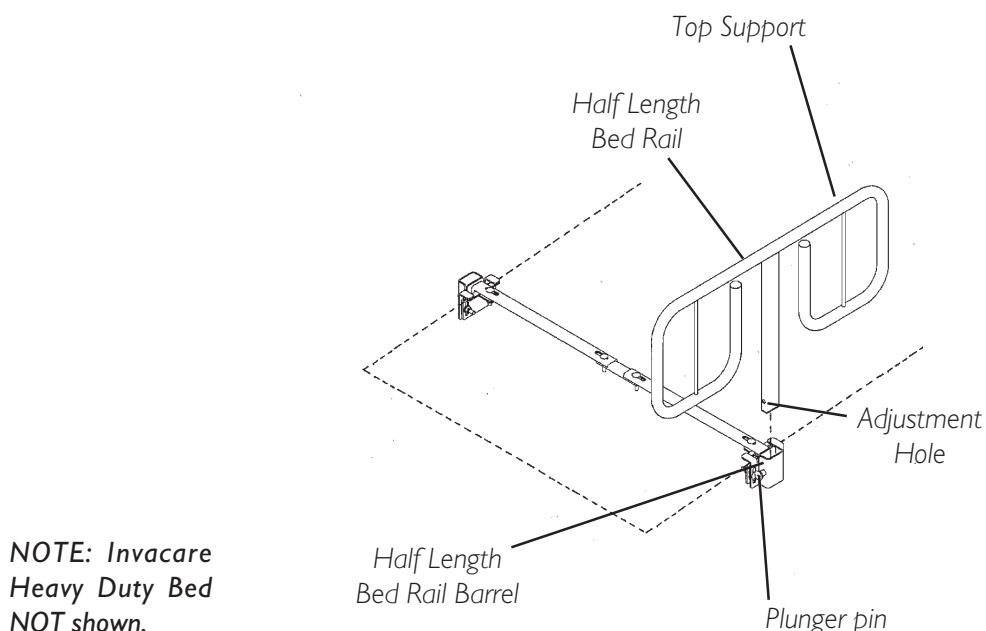
FIGURE 3 - INSTALLING THE HALF LENGTH BED RAILS

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

Invacare warrants its product to be free from defects in materials and workmanship for two (2) years for the original purchaser. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address below. Provide dealer's name, address, model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number.

Invacare Corporation will issue a return authorization. The defective unit or parts must be returned for warranty inspection using the serial number, when applicable, as identification within thirty (30) days of return authorization date. **DO NOT** return products to our factory without our prior consent. C.O.D. shipments will be refused; please prepay shipping charges.

LIMITATIONS AND EXCLUSIONS: THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THE ENCLOSED INSTRUCTIONS. IN ADDITION, THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED; PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE; OR PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO: MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE; PRODUCTS DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL; PRODUCTS REPAIRED BY ANYONE OTHER THAN AN AUTHORIZED INVACARE DEALER, SUCH EVALUATION SHALL BE SOLELY DETERMINED BY INVACARE.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

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© 2005 Invacare Corporation
Invacare is a registered trademark of
Yes, you can. is a trademark of Invacare
Invacare Corporation www.invacare.com