


MNS 600 Lateral Rotation System

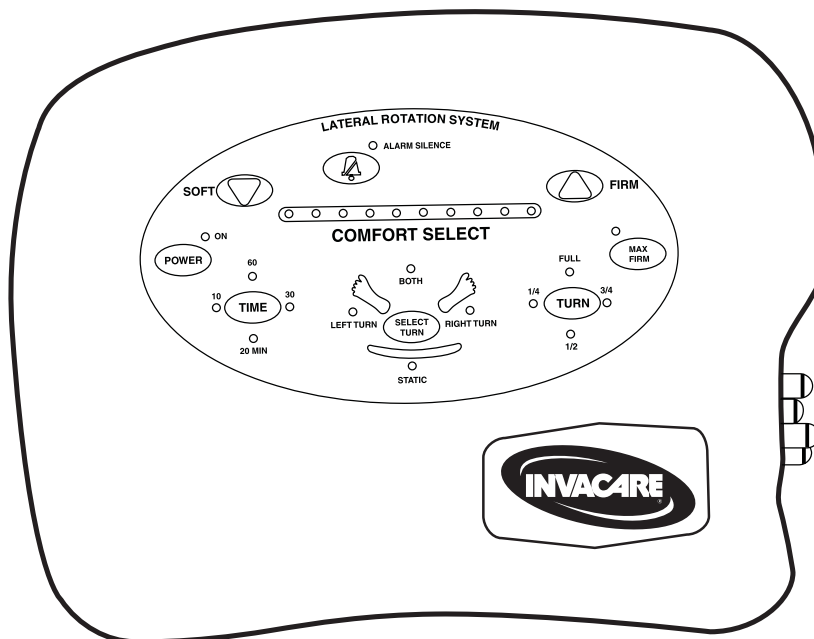

DEALER: This manual **MUST** be given to the user of the product.

USER: BEFORE using this product, read this manual and save for future reference.

For more information regarding Invacare products, parts, and services, please visit www.invacare.com



Yes, you can.

⚠ WARNING

DO NOT OPERATE THIS EQUIPMENT WITHOUT FIRST READING AND UNDERSTANDING THIS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS, AND INSTRUCTIONS, CONTACT A HEALTHCARE PROFESSIONAL, DEALER OR TECHNICAL PERSONNEL IF APPLICABLE BEFORE ATTEMPTING TO USE THIS EQUIPMENT - OTHERWISE INJURY OR DAMAGE MAY RESULT.

PROCEDURES OTHER THAN THOSE DESCRIBED IN THIS MANUAL MUST BE PERFORMED BY A QUALIFIED TECHNICIAN.

SPECIAL NOTES

Signal words are used in this manual and apply to hazards or unsafe practices which could result in personal injury or property damage. Refer to the table below for definitions of the signal words.

SIGNAL WORD	MEANING
DANGER	Danger indicates an imminently hazardous situation which, if not avoided, will result in death or serious injury.
WARNING	Warning indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.
CAUTION	Caution indicates a potentially hazardous situation which, if not avoided, may result in property damage.

NOTICE

THE INFORMATION CONTAINED IN THIS DOCUMENT IS SUBJECT TO CHANGE WITHOUT NOTICE.

Check all parts for shipping damage and test before using. In case of damage, DO NOT use. Contact Invacare Customer Service Department for further instruction.

⚠ WARNING

Invacare products are specifically designed and manufactured for use in conjunction with Invacare accessories. Accessories designed by other manufacturers have not been tested by Invacare and are not recommended for use with Invacare products.

NOTE: Updated versions of this manual are available on www.invacare.com.

TABLE OF CONTENTS

SPECIAL NOTES	2
REGISTER YOUR PRODUCT	4
TYPICAL PRODUCT PARAMETERS	7
LABEL LOCATION	8
SECTION 1—GENERAL GUIDELINES	9
Classification	9
Installation	9
Entrapment May Occur.....	9
Fire Hazard.....	10
Smoking.....	10
Anesthesia Equipment.....	10
Oxygen	10
Electrical	10
Servicing.....	11
SECTION 2—FEATURES	12
MNS 600 Lateral Rotation System	12
Front Panel	12
Right Side Panel	13
Left Side Panel.....	14
Rear Panel.....	15
SECTION 3—OPERATION	16
Preparing the Bed.....	16
Installing the Mattress.....	16
Installing the Power Unit	16
Connecting the Air Hose.....	17
Connecting the Power Cord	17
Using the System	18
Patient Placement	18
Checking the Mattress Setting.....	19
Patient Transfer From/To a Gurney	19
Patient Transfer From/To a Wheelchair	20
Installing the Side Rails	20
SECTION 4—MAINTENANCE AND TROUBLESHOOTING	21
Setting Up the MNS Systems	21
Recommended Cleaning Materials	21
Cleaning the MNS System	22
Storing the MNS System	23
Troubleshooting.....	23
Inspection, Cleaning, Maintenance and Test Intervals	24
APPENDIX	25

Inspection and Test Reports.....25
Inspection and Test Reports for Hospital Installations.....26
LIMITED WARRANTY 28

REGISTER YOUR PRODUCT

The benefits of registering:

- 1. Safeguard your investment.
- 2. Ensure long term maintenance and servicing of your purchase.
- 3. Receive updates with product information, maintenance tips, and industry news.
- 4. Invacare can contact you or your provider, if servicing is needed on your product.
- 5. It will enable Invacare to improve product designs based on your input and needs.

Register ONLINE at www.invacare.com

- or -

Complete and mail the form on the next page

Any registration information you submit will be used by Invacare Corporation only, and protected as required by applicable laws and regulations.



PRODUCT REGISTRATION FORM

Register **ONLINE** at www.invacare.com - or -
Complete and mail this form

Name _____

Address _____

City _____ State/Province _____

Zip/Postal Code _____

Email _____ Phone No. _____

Fold
here

Invacare Model No. _____ Serial No. _____

Purchased From _____ Date of Purchase: _____

1. Method of purchase: (check all that apply)

Medicare Insurance Medicaid Other _____

2. This product was purchased for use by: (check one)

Self Parent Spouse Other

3. Product was purchased for use at:

Home Facility Other

4. I purchased an Invacare product because:

Price Features (list features) _____

5. Who referred you to Invacare products? (check all that apply)

Doctor Therapist Friend Relative Dealer/Provider Other _____

Advertisement (circle one): TV, Radio, Magazine, Newspaper No Referral _____

6. What additional features, if any, would you like to see on this product?

Fold
here

7. Would you like information sent to you about Invacare products that may be available for a particular medical condition? Yes No

If yes, please list any condition(s) here and we will send you information by email and/or mail about any available Invacare products that may help treat, care for or manage such condition(s):

8. Would you like to receive updated information via email or regular mail about the Invacare home medical products sold by Invacare's dealers? Yes No

9. What would you like to see on the Invacare website?

10. Would you like to be part of future online surveys for Invacare products? Yes No

11. User's Year of birth: _____


If at any time you wish not to receive future mailings from us, please contact us at Invacare Corporation, CRM Department, 39400 Taylor Parkway, Elyria, OH 44035, or fax to 877-619-7996 and we will remove you from our mailing list.

To find more information about our products, visit www.invacare.com.

Cut Along Line




Fold
here

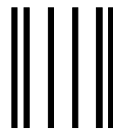

BUSINESS REPLY MAIL
FIRST-CLASS MAIL PERMIT NO. 196 ELYRIA OH

POSTAGE WILL BE PAID BY ADDRESSEE

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NO POSTAGE
NECESSARY
IF MAILED
IN THE
UNITED STATES



INVACARE CORPORATION
CRM DEPARTMENT
39400 TAYLOR PARKWAY
ELYRIA OH 44035-9836



Fold
here

Invacare Product Registration Form

Please Seal with
Tape Before Mailing

Cut Along Line




TYPICAL PRODUCT PARAMETERS

	MNS 600
AIR TUBES WASHABLE: WATERPROOF: NUMBER OF CELLS: PRESSURE SECTIONS: TUBE SEAM BONDING:	Yes Yes Sixteen Mattress Bladders One Lower Mattress Three RF Welded
PATIENT MATTRESS INTERFACE VAPOR PERMEABLE:	Yes
AIR FILTERS:	No
WEIGHT OF PRODUCT CONTROL UNIT: MATTRESS MODEL: S B	11 lbs (5.5 kgs) 23 lbs (10.5 kgs) 28 lbs (13 kgs)
WEIGHT CAPACITY MAXIMUM PATIENT WEIGHT - MODEL: S B	500 lbs (227 kgs) 750 lbs (341 kgs)
AIR BLOWER - ESTIMATED SERVICE LIFE:	20,000 Hours
DIMENSIONS (Width x Length x Height) MATTRESS MODEL: S B CONTROL UNIT (Width x Height x Depth):	35" x 80" x 8" (89cm x 203cm x 20cm) 42" x 80" x 8" (107cm x 203cm x 20cm) 15" x 11.5" x 6" (28cm x 32cm x 13cm)
ELECTRICAL OPERATING POWER: U.S.: E.U.: FUSES:	108-132 V, 60 Hz, 3 A Normal, 5 A Max. 108-264 V, 60 Hz, 3 A Normal, 5 A Max. 750mA/250 V (UL listed fuses)
LISTING AGENCY:	Entela, CSA IEC 60601
MATTRESS SANITIZATION FLUID ENTRAPMENT: DETERGENT RESIST:	No (Seamless) Yes
ENVIRONMENTAL CONDITIONS OPERATING CONDITIONS: Ambient Temperature: Relative Humidity: STORAGE AND SHIPPING CONDITIONS*: Ambient Temperature: Relative Humidity:	18°C to 34°C 65°F to 94°F 10% to 95% -18°C to +43°C 0°F to 110°F 10% to 95%

*NOTE: Product may be shipped on all conventional transport. Pack system in its original boxes to assure best protection. DO NOT drop or expose to impact greater than 25 G's. Doing so can damage internal components and void warranty.

LABEL LOCATION

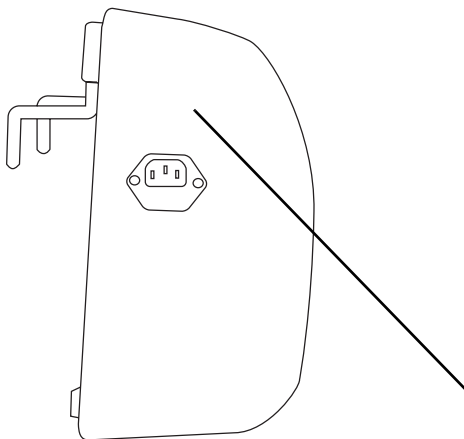

⚠ DANGER

Explosion risk if used with flammable anesthetics.


Possible fire hazard when used with oxygen administering equipment Other than nasal mask or 1/2 bed tent type. Oxygen tent should not extend below mattress support level.

Electrical shock hazard. **DO NOT** remove cover. Refer to qualified service personnel.

⚠ WARNING

Grounding reliability can only be achieved when the equipment is connected to a universal receptacle.

1141497 Rev A - 05/04/06

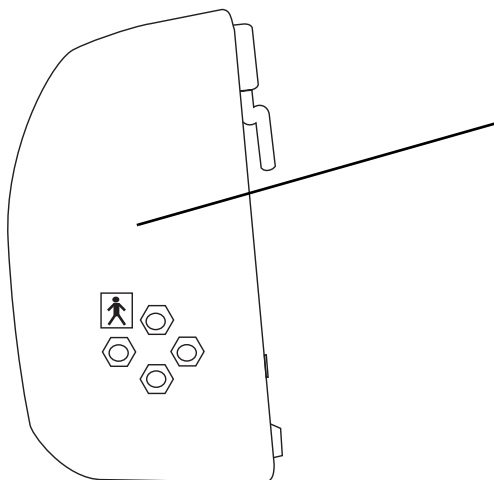

⚠ WARNING

Disconnect air hose(s) before administering CPR.

1141498 Rev A - 05/04/06

SECTION I—GENERAL GUIDELINES

⚠ WARNING

SECTION I - GENERAL GUIDELINES contains important information for the safe operation and use of this product. **DO NOT** use this product or any available optional equipment without first completely reading and understanding these instructions and any additional instructional material such as **Owner's Manuals, Service Manuals or Instruction Sheets** supplied with this product or optional equipment. If you are unable to understand the **Warnings, Cautions or Instructions**, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment - otherwise, injury or damage may occur.

Classification

The MNS 600 Lateral Rotation System is Class II Medical Devices and type BF (not defibrillation proof) applied parts.

NOTE: ALWAYS consult the patient's physician before using the MNS 600 Lateral Rotation System.

Installation

The MNS 600 system **MUST** be installed on standard medical bed frames only, such as hospital, nursing home and homecare beds.

Some bed frames have controls on the footboard that may be obstructed by the power unit. It will be necessary to relocate the power unit.

Before placing a patient on the bed, ensure that the air hoses and power cord are clear of the moving bed components.

Operate all motorized functions through their full range of motion to be certain that there is no pulling, interference or pinching.

Entrapment May Occur

⚠ WARNING

Proper patient assessment and monitoring, and proper maintenance and use of equipment is required to reduce the risk of entrapment. Variations in bed rail dimensions, and mattress thickness, size or density could increase the risk of entrapment. Visit the FDA website at <http://www.fda.gov> to learn about the risks of entrapment. Review "A Guide to Bed Safety", published by the Hospital Bed Safety Workgroup, located at www.invacare.com. Use the link located under each bed rail product entry to access this bed safety guide.

⚠ WARNING

Refer to the owners manuals for beds and rails for additional product and safety information.

After any adjustments, repair or service and before use, make sure all attaching hardware is tightened securely. Assist rails with dimensions different from the original equipment supplied or specified by the bed manufacturer may not be interchangeable and may result in entrapment or other injury.

Mattress **MUST** fit bed frame and assist rails snugly to reduce the risk of entrapment.

Fire Hazard

⚠ DANGER

Smoking

Users **MUST NOT SMOKE** while using this device. **MNS 600** system uses room air for circulation through the mattress. A cigarette can burn a hole in the bed surface and cause damage to the mattress. Also, patient clothing, bed sheets, etc. may be combustible and cause a fire. Failure to observe this warning can result in severe fire, property damage and cause physical injury or **DEATH**.

Smoking by visitors in the room will contaminate the system. Therefore, visitor smoking is **NOT** permitted.

Anesthesia Equipment

There is an explosion risk if used with flammable anesthetics.

Oxygen

There is a possible fire hazard when used with oxygen administering equipment other than nasal mask or half bed tent type. The oxygen tent should **NOT** extend below mattress support level.

Electrical

Before performing any maintenance to the power unit, disconnect the power cord from the wall outlet.

Refer servicing to qualified personnel only. Grounding reliability depends upon a properly grounded wall outlet.

Servicing

The MNS 600 contains no user-serviceable parts. DO NOT attempt to open the power unit—refer all repair questions to the Dealer. To deliver consistent performance, the MNS 600 MUST be maintained, inspected and tested according to the schedule on page 24.

If your system requires service, contact Customer Service for a return shipment authorization number.

Package systems in the original shipping box. Clean system components before shipment. **Unsanitary units will be returned without servicing.**

SECTION 2—FEATURES

MNS 600 Lateral Rotation System

Front Panel

NOTE: For this procedure, refer to FIGURE 2.1.

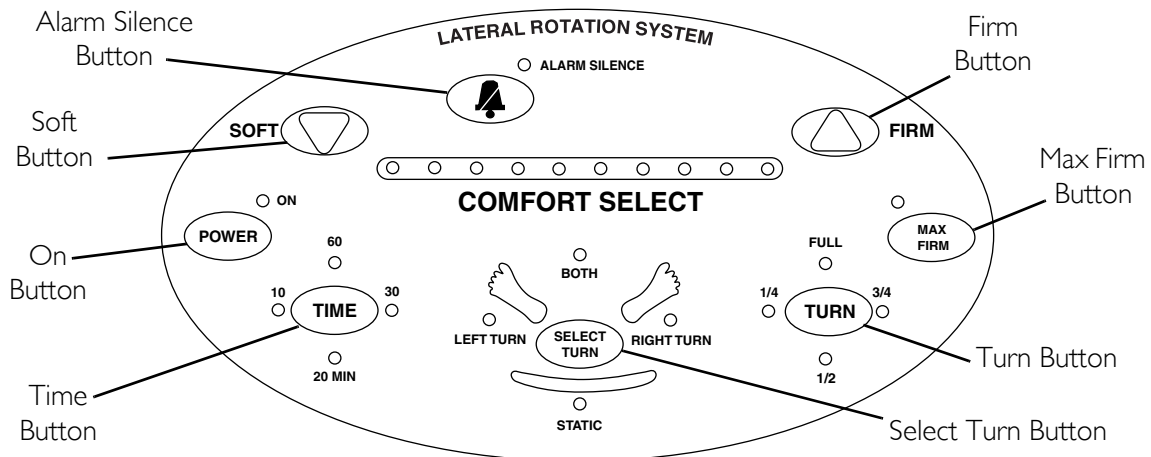


FIGURE 2.1 MNS 600 Lateral Rotation System - Front Panel

Power Button

1. Press and hold the Power button in order to check that all the lights on the front panel are working.
2. To turn the power unit on, press and release the Power button.

Firm/Soft Buttons

1. Select comfort setting by pressing Firm or Soft button.

NOTE: There are ten levels of comfort setting indicated by the ten lights on the display. Softer settings generally result in reduced pressure areas on the body, however, a too soft setting depress the patient's body in the center and possibly bottom out on foam cushion at the bottom of the mattress. Try to achieve a level body position with the lowest comfort setting.

Max Firm Button

NOTE: The Max Firm mode will be active for 30 minutes. The controller will return the mattress pressure to the previous setting level.

1. Press the Max Firm button -
 - to transfer the patient to a chair;
 - to move the patient to a new position; or
 - to examine a patient.

Select Turn Button

1. Press the Select Turn button to select one of the following:
 - Static mode - When the mattress is in this mode, the patient's weight is evenly distributed over the mattress.
 - Left - When the mattress is in this mode, the patient is turned to the left.
 - Right - When the mattress is in this mode, the patient is turned to the right.
 - Both - When the mattress is in this mode, the patient is turned to both the right and the left.

Turn Button

1. Press the Turn button to select one of the four turn positions:

NOTE: A patient's original position is when they are lying flat on their back.

- ¼ Turn - Turns the patient 15° to one side from the original position.
- ½ Turn - Turns the patient 25° to one side from the original position.
- ¾ Turn - Turns the patient 30° to one side from the original position.
- Full Turn - Turns the patient 45°, to the patient's side.

Time Button

1. Press the Time button to select one of the four time intervals in the turning mode:
 - 10 Mode - The mattress will turn the patient to each side for 10 minutes.
 - 20 Mode - The mattress will turn the patient to each side for 20 minutes.
 - 30 Mode - The mattress will turn the patient to each side for 30 minutes.
 - 60 Mode - The mattress will turn the patient to each side for an hour.

Alarm Silence Button

1. Press the Alarm Silence button to acknowledge and silence the alarm when there is low air pressure detected or when power to the pump has failed.

NOTE: The alarm light will remain on until the fault has been corrected.

Right Side Panel

NOTE: For this procedure, refer to FIGURE 2.2 on page 14.

Air Hose Connection

NOTE: The MNS 600 has a single connection for four air hoses.

1. Release the air hose connection to rapidly deflate the mattress to perform CPR.

CPR

NOTE: The hose connection at the control unit is marked CPR.

1. Disconnect the hose from the control unit.
2. Place a cardiac board under the patient and initiate CPR.
3. To reinflate the cushions, reattach the hose to the control unit.

NOTE: It is advisable to place the patient on a hard surface like the floor or a table for effective chest compressions rather than on the bed because the bed frame may have a suspension system that will absorb compression efforts.

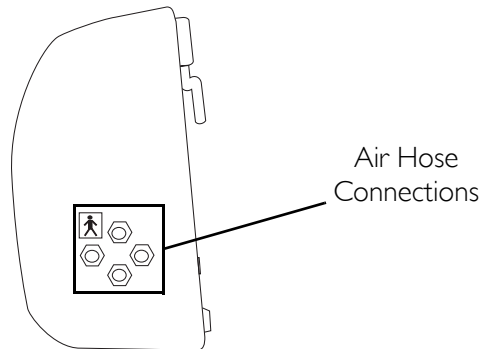


FIGURE 2.2 MNS 600 Lateral Rotation System - Right Side Panel

Left Side Panel

NOTE: For this procedure, refer to FIGURE 2.3.

Power Outlet

1. Plug one end of the power cord into the power outlet.

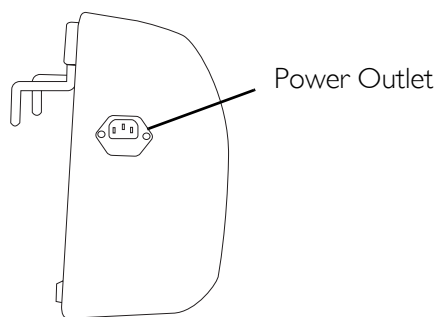


FIGURE 2.3 MNS 600 Lateral Rotation System - Left Side Panel

Rear Panel

NOTE: For this procedure, refer to FIGURE 2.4.

Bed Hooks

NOTE: The standard power unit installs on the upper edge of the footboard.

Bed hooks are used to hang the unit on the footboard of the bed.

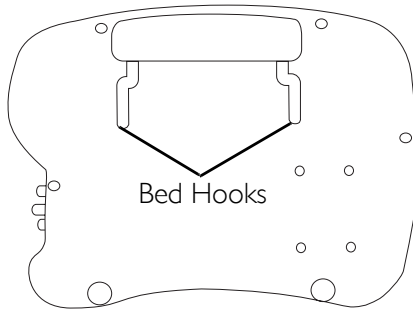


FIGURE 2.4 MNS 600 Lateral Rotation System - Rear Panel

SECTION 3—OPERATION

Preparing the Bed

NOTE: The bed MUST be a standard hospital bed or medical grade home care bed.

1. Remove the original bed mattress and place it in storage.

Installing the Mattress

1. If necessary, lower the side rails to facilitate installation of the mattress.
2. Roll out the mattress onto the bed frame with the hose connection at the foot end.
3. Secure the six tie-down straps to the ends and sides of the bed frame at the convenient locations by performing the following:
 - A. Make certain that the side rails can move freely.
 - B. Thread the strap ends through the buckles provided and take up the excess slack.
 - C. To release strap tension, lift the tab on the buckle.

Installing the Power Unit

NOTE: For this procedure, refer to FIGURE 3.1.

NOTE: The standard power unit installs on the upper edge of the footboard.

1. Swing out the bed hooks and place the power unit on the footboard.

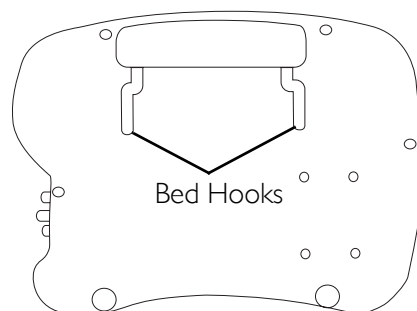


FIGURE 3.1 Installing the Power Unit

Connecting the Air Hose

CAUTION

The mattresses and matching pumps on **MNS 400**, **MNS 500** and **MNS 600** are designed to be used as a system. **DO NOT** replace mattresses or pumps with other models or other brands. Otherwise, damage to the system may occur. Contact your supplier to get the correct replacement if needed.

NOTE: For this procedure, refer to FIGURE 3.2.

1. Attach the hose connector from the mattress to the mating connectors on the side of the pump.

NOTE: Ensure that the colors on the air hose connectors match the colors on the connectors on the pump.

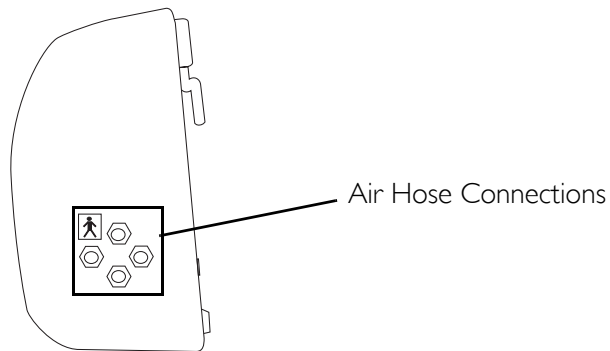


FIGURE 3.2 Connecting the Air Hose

Connecting the Power Cord

⚠ WARNING

DO NOT alter plug to fit a non-conforming outlet. Instead, have an electrician install a properly grounded outlet. Failure to use the correct plug and outlet can result in a potential safety hazard.

NOTE: For this procedure, refer to FIGURE 3.3 on page 18.

1. Plug the rectangular end of the supplied power cord into the power entry module on the side of the pump.

NOTE: Use the hospital grade power supply cord that came with the pump.

2. Route the power cord on the floor under the center of the bed.

3. Plug the round end of the plug into a properly grounded outlet on the wall.
4. Carefully route the plug cord so that it cannot be pulled out of the outlet, damaged from nearby furniture, cleaning devices or people.

NOTE: If the plug is damaged, call your supplier for a replacement hospital grade cord.

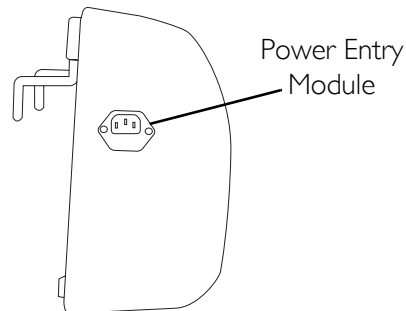


FIGURE 3.3 Connecting the Power Cord

Using the System

NOTE: For this procedure, refer to FIGURE 2.1 on page 12.

1. Turn on the power to the system by pressing the Power button on the control panel.

NOTE: While the button is pressed and held, all the lights will activate.

NOTE: Once the button is released, normal operation of the pump will begin. The GREEN light illuminates when the unit is on.

NOTE: Allow up to 20 minutes for the mattresses on the MNS 600 systems to fully inflate.

2. Once the mattress is fully inflated, place the patient on the mattress. Refer to [Patient Placement](#) on page 18.

Patient Placement

1. Place the patient on the bed.
2. Ensure that the patient's feet are towards the end of the mattress with the connector tube.
3. Center the patient on the bed from side-to-side and head-to-foot.

NOTE: Special positioning may be required with contracted patients to provide comfortable positions.

4. Adjust the comfort range to the desired firmness to avoid bottoming out.
5. If the patient feels that the bed is too soft/hard, press the Soft or Firm button to adjust the comfort settings.

NOTE: Ensure that the patient does not bottom out.

6. Press the Select Turn button to select the desired operation mode.

7. After placing the patient, make certain no objects will fall under the patient, such as feeding tubes, IV's etc.
8. Use a regular pillow to help support and stabilize the patient's head.

Checking the Mattress Setting

⚠ WARNING

The head of the bed should be maintained at the lowest degree of elevation consistent with medical conditions; otherwise injury may occur.

The lowest skin pressure is maintained with 0° to 20° head raise angle. Angles above 30° place more body weight over the seat area, reducing pressure relief.

Instruct the caregiver to avoid head angles over 30° for extended periods, unless medically necessary (U.S. Department of Health and Human Services publication No. 92-0048).

1. Before elevating the head of the bed, elevate the knee section first.

NOTE: Head elevation without knee elevation may cause the patient to slide toward the foot of the bed.

2. Elevate the head of the bed to 30°- 45°.
3. Using your hand, check under the patient's buttocks and between two air tubes to check if the patient has bottomed out.
4. If the patient has bottomed out, increase the comfort adjust one level (LED light) at a time, until the patient no longer bottoms out.

Patient Transfer From/To a Gurney

⚠ WARNING

ALWAYS engage the wheel locks of the bed before transferring the patient from the bed to the gurney.

1. Engage the wheel locks of the bed. Refer to the owner's manual provided with the bed.
2. Engage the wheel locks of the gurney. Refer to the owner's manual provided with the gurney.
3. Press the Max Firm button to achieve maximum mattress pressure.
4. Raise or lower the bed to match the gurney height. Refer to the owner's manual provided with the bed.
5. When the mattress has reached maximum firmness, perform one of the following:
 - Bed to Gurney Transfer - Slide the patient onto the gurney.
 - Gurney to Bed Transfer - Slide the patient onto the bed.

Patient Transfer From/To a Wheelchair

⚠ WARNING

ALWAYS engage the wheel locks of the bed before transferring the patient from the bed to the wheelchair.

1. Engage the wheel locks of the bed. Refer to the owner's manual provided with the bed.
2. Engage the wheel locks of the wheelchair, if applicable. Refer to the owner's manual provided the wheelchair.
3. Press the Max Firm button to achieve maximum mattress pressure.
4. Raise or lower the bed to match the wheelchair height. Refer to the owner's manual provided with the bed.
5. When the mattress has reached maximum firmness, perform one of the following:
 - Bed to Wheelchair Transfer - Slide the patient onto the wheelchair.
 - Wheelchair to Bed Transfer - Slide the patient onto the bed.

Installing the Side Rails

⚠ WARNING

Mattress MUST fit bed frame and assist rails snugly to reduce the risk of entrapment.

NOTE: Refer to the instructions provided with the side rails for the installation procedure.

SECTION 4—MAINTENANCE AND TROUBLESHOOTING

Setting Up the MNS Systems

The MNS 600 systems are made patient-ready at the dealership before delivery to the installation site. The mattress is pre-assembled and the power unit has been tested and serviced.

Recommended Cleaning Materials

CAUTION

DO NOT use chlorine based cleaners; otherwise, product damage may occur.

Disinfectant chemicals should be registered, hospital-grade, and have effectiveness claims for destroying or inactivating harmful microorganisms. Use only in the manufacturer's recommended dilution. NEVER mix different chemicals together in the same container.

Phenolic disinfectants are supplied in highly concentrated form. They are tuberculocidal when used at the recommended dilution. They provide a high level of disinfection. They are also effective in decontaminating blood spills.

Quaternary cleaner disinfectants are supplied in highly concentrated form. They are usually capable of cleaning, deodorizing and disinfecting in a single operation. They are especially suitable for cleaning equipment at the patient site where low level disinfecting or odor control is indicated.

Laundry disinfectant presoaks can be either phenol or quaternary chemicals. These products are suitable for adding to water as a presoak.

For lighter soil, the cover can be cleaned by topical wipe down with a PH balanced soap and water solution normally used to wash the patient. Rinse with water.

Cleaning the MNS System

⚠ WARNING

Before cleaning or disassembling the MNS systems, check the underside of the mattress folds for sharp objects such as scissors, needles, etc. These objects should be removed and discarded before proceeding with further cleaning or disassembly; otherwise injury or damage to the product may occur.

Because of potential risk of infectious exposure, cleaning with the patient on the bed is not recommended.

All equipment should be inspected. Any item that is visibly soiled with the patient's blood or other body fluids should be properly cleaned or removed.

NOTE: Refer to Recommended Cleaning Materials on page 21.

1. Remove the bedding.
2. If necessary, inflate the mattress.
3. Ensure that the power unit is off.
4. Unplug the power cord from the wall outlet.
5. Ensure that the underside of the mattress is clear of all sharp objects.
6. Examine the surface of the power unit and mattress assembly components for visible blood or body fluids.
7. Perform one of the following:
 - If blood is present, decontaminate the product.
 - i. Remove all visible soil with disposable paper towels.
 - ii. Scrub the area with freshly prepared effective phenolic detergent disinfectant solution.
 - If blood is not present, remove any soil from the cover with paper towels.

NOTE: If grossly soiled, the cover should be removed, cleaned and decontaminated.

8. Using a clean sponge or paper towel, wipe down the cover surface with a dilute detergent solution of quaternary cleaner disinfectant or other germicidal detergent solution.
9. Remove the cover and launder it.
10. Perform the following steps to clean the power unit and hose fittings:
 - A. Wipe all controls and chassis and hose fittings with a quaternary disinfectant solution.
 - B. Using a nylon brush, gently clean all crevices as they can harbor microorganisms.
 - C. Air dry all treated surfaces.

Storing the MNS System

1. Ensure that the power unit is off and disconnect the power cord from the wall outlet.
2. Clean the MNS system. Refer to [Cleaning the MNS System](#) on page 22.
3. Disconnect the air hose connector(s) from the power unit and allow air to vent from the mattress.
4. Gently roll up the mattress with minimal handling and agitation.
5. Ensure the cover surface is inside the roll.
6. Move the entire system from the patients room to storage.

Troubleshooting

SYMPTOM	SOLUTION
Control Unit does not turn on.	Check the power cord. Be sure it is plugged in to both the wall outlet and into the side of the pump housing. Check that outlet has power by plugging in a device that you know is working.
The mattress is too hard.	Check Max Firm mode and turn it off if it is on. Press the Soft button to create a softer mattress. Listen to the pump; if it is running continuously, call your supplier.
One side is softer than the other.	In the Turn mode, this is normal; depending on the cycle, it will change over time. If the mattress stays soft on one side for more than two hours, contact your supplier.
The mattress is deflating or has low pressure even though the pump is working.	Check the mattress connections on the side of the pump housings; check the connector tubes for kinks, crimps or damage; listen for air leaks around the mattress.
Pump is noisy and/or causing vibrations to the bed.	The MNS 600 has a powerful pump that sometimes causes the footboard of the bed to vibrate. Place a fabric over the footboard to reduce the vibrations. Tightly secure the parts of the bed that vibrate excessively.

Inspection, Cleaning, Maintenance and Test Intervals

TIME INTERVAL	DESCRIPTION OF THE REQUIRED ACTIVITY
When Soiled	Launder the cover.
168 hours (One week of continuous operation)	Wipe down the cover, air hoses and power unit.
670 hours (Four weeks of continuous operation)	Clean the cover. Inspect the air hose connections and mattress assembly function Inspect the power unit and power cord.
Between Users	Launder the cover. Perform general physical inspection of the mattress, air hoses, power unit and power cord. Perform electrical line leakage test. Perform electrical ground integrity test.
Annually or after 8000 hours of operation (whichever comes first)	Launder the cover. Perform general physical inspection of the mattress, air hoses, power unit and power cord. Perform electrical line leakage test. Perform electrical ground integrity test. Check for loose hardware or physical damage which might affect operation or safety.

APPENDIX

Inspection and Test Reports

SHEET 1 OF 2

System Serial #: _____

Date: ___/___/___

Hour Meter Reading: _____

Service Person: _____

Print Name

Service Performed:

 1000 Hours Repair Annual or 8000 Post Decontamination

PHYSICAL INSPECTION CHECKLIST

ITEM	OK	BAD	COMMENTS
Mattress			
Air Tubes			
CPR Label on Air Hose Connector			
Air Hoses			
Power Unit			
Power Cord			
Mounting Hardware			
Marking Labels			
Other			

Inspection and Test Reports for Hospital Installations

SHEET 2 OF 2

ITEM	PARAMETER	REQUIREMENTS
Electrical Leakage: Normal reverse ground connected.	_____ microamps	No reading shall exceed 100 microamps
Ground resistance from plug to Power Unit Chasis	_____ milli-ohms	Resistance shall not exceed 100 milli-ohms

This unit has ____passed/____failed the above inspections. If failed, the following repairs and corrections are required:

I certify that this unit conforms to all the above test and inspection criteria and that scheduled maintenance has been performed as indicated.

I certify that this unit is suitable for service.

Signature

Print Name of Service Person

____ / ____ / ____
Date

NOTES

LIMITED WARRANTY

PLEASE NOTE: THE WARRANTY BELOW HAS BEEN DRAFTED TO COMPLY WITH FEDERAL LAW APPLICABLE TO PRODUCTS MANUFACTURED AFTER JULY 4, 1975.

This warranty is extended only to the original purchaser/user of our products.

This warranty gives you specific legal rights and you may also have other legal rights which vary from state to state.

With regard to the original purchaser/user only, Invacare warrants the **MNS 600 Lateral Rotation System** for one year on the pump and six months on the mattress. If within such warranty period any such product shall be proven to be defective, such product shall be repaired or replaced, at Invacare's option. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of any such product. Invacare's sole obligation and your exclusive remedy under this warranty shall be limited to such repair and/or replacement.

For warranty service, please contact the dealer from whom you purchased your Invacare product. In the event you do not receive satisfactory warranty service, please write directly to Invacare at the address at the bottom of this page. Provide dealer's name, address, the product model number, date of purchase, indicate nature of the defect and, if the product is serialized, indicate the serial number. Do not return products to our factory without our prior consent.

LIMITATIONS AND EXCLUSIONS: THE FOREGOING WARRANTY SHALL NOT APPLY TO SERIAL NUMBERED PRODUCTS IF THE SERIAL NUMBER HAS BEEN REMOVED OR DEFACED, PRODUCTS SUBJECTED TO NEGLIGENCE, ACCIDENT, IMPROPER OPERATION, MAINTENANCE OR STORAGE, PRODUCTS MODIFIED WITHOUT INVACARE'S EXPRESS WRITTEN CONSENT INCLUDING, BUT NOT LIMITED TO, MODIFICATION THROUGH THE USE OF UNAUTHORIZED PARTS OR ATTACHMENTS; PRODUCTS DAMAGED BY REASON OF REPAIRS MADE TO ANY COMPONENT WITHOUT THE SPECIFIC CONSENT OF INVACARE, OR TO A PRODUCT DAMAGED BY CIRCUMSTANCES BEYOND INVACARE'S CONTROL, AND SUCH EVALUATION WILL BE SOLELY DETERMINED BY INVACARE. THE WARRANTY SHALL NOT APPLY TO PROBLEMS ARISING FROM NORMAL WEAR OR FAILURE TO ADHERE TO THESE INSTRUCTIONS.

THE FOREGOING EXPRESS WARRANTY IS EXCLUSIVE AND IN LIEU OF ANY OTHER WARRANTIES WHATSOEVER, WHETHER EXPRESS OR IMPLIED, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, AND THE SOLE REMEDY FOR VIOLATIONS OF ANY WARRANTY WHATSOEVER, SHALL BE LIMITED TO REPAIR OR REPLACEMENT OF THE DEFECTIVE PRODUCT PURSUANT TO THE TERMS CONTAINED HEREIN. THE APPLICATION OF ANY IMPLIED WARRANTY WHATSOEVER SHALL NOT EXTEND BEYOND THE DURATION OF THE EXPRESS WARRANTY PROVIDED HEREIN. INVACARE SHALL NOT BE LIABLE FOR ANY CONSEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

THIS WARRANTY SHALL BE EXTENDED TO COMPLY WITH STATE/PROVINCIAL LAWS AND REQUIREMENTS.

Invacare Corporation www.invacare.com

2101 E. Lake Mary Blvd.
Sanford, Florida
32773

Invacare, Yes, you can., and the Medallion Design are registered trademarks of Invacare Corporation.

© 2006 Invacare Corporation

Part No 1141507

Rev A - 06/13/06



Yes, you can.