


Hydraulic Patient Lift - 9805P

Includes 9800AP Mast & 9884P Base

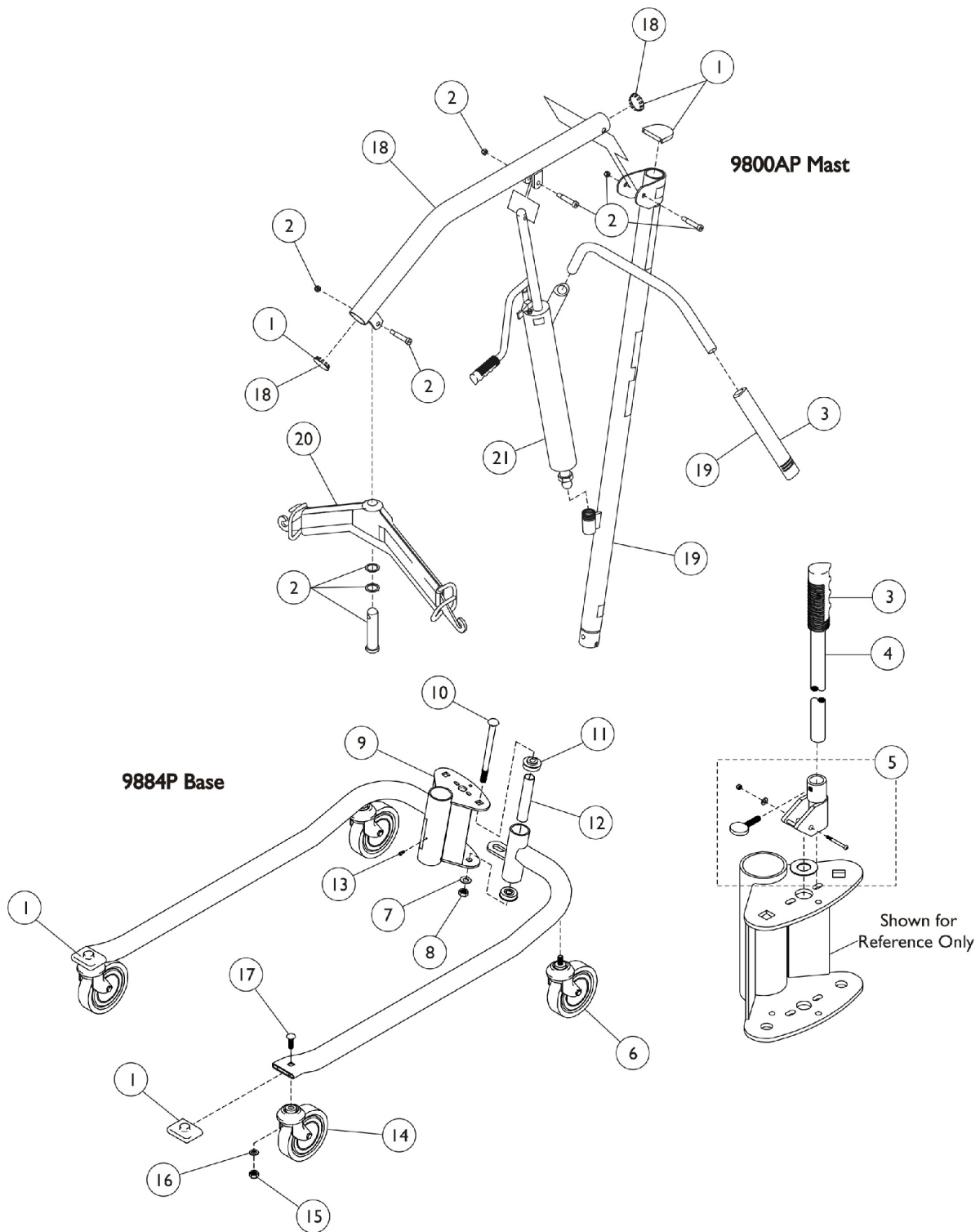

Hydraulic Patient Lift - 9805P

Includes 9800AP Mast & 9884P Base

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1		1123442	Kit, Plugs and Caps	1	1
			Plug Button, Metal, Chrome (1-3/4")	2	1
			Cap, Mast	1	1
			Cap, End, Grey (2 x 2-1/2")	2	1
2		1123443	Kit, Hardware, Boom	1	1
			Washer, Nylon (1-1/64 x 1-11/32 x 1/16")	2	1
			Pin (1-1/4" x 4-1/4")	1	1
			Screw, Shoulder Hex Socket Head (3/8 x 1-1/4", 5/16-18)	1	1
			Screw, Shoulder Hex Socket Head (3/8" x 1-1/2", 5/16-18)	1	1
			Screw, Shoulder Hex Socket Head (3/8" x 2-1/4", 5/16-18)	1	1
			NLA - Locknut (5/16-18)	3	1
3		1123444	Kit, Handgrip	1	1
			Grip, Hand - 9805	2	1
			Grip, Handle - 9805	1	1
4		1140730	Kit, Shifter	1	1
			Grip, Handle - 9805	1	1
			Handle, Shifter - 9805	1	1
5	A	1140731	Kit, Camlock w/ Hardware - 9805P	1	1
			Cam, Lock - 9805P	1	1
			NLA - Bushing (5/8 x 1-1/8 x 5/16")	1	1
			NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
			Washer, Flat (1/4 x 1/2 x 1/16")	1	1
			Locknut (1/4-20)	1	1
			NLA - Screw, Thumb (1/4-20 x 1")	1	1
			NLA - Instruction Sheet	1	1
6		1123537	Caster Assembly, Swivel (5")	1	2
7	E	90044X000-NLA	NLA - Washer, Lock (1/2 x 7/8 x 1/32")	1	1
8	E	90070X000-NLA	NLA - Nut (1/2-13)	1	1
8		1123448	Kit, Base Bolt	1	1
			NLA - Washer, Lock (1/2 x 7/8 x 1/32")	1	1
			NLA - Nut (1/2-13)	1	1
			NLA - Bolt, Carriage (1/2-13 x 6-3/4")	2	1
9		1140721-NLA	NLA - Socket - Beige	1	1
10	E	90069X000-NLA	NLA - Bolt, Carriage (1/2-13 x 6-3/4")	1	2
11		90068X000	Bearing, Flange	1	4
12		90071X000-NLA	NLA - Spacing, Bearing	1	2
13		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	1
14		1123536	Caster Assembly, Swivel (5")	1	2
14		1123449	Kit, Caster, Front	1	1
			Caster Assembly (5")	2	1
			Locknut (7/16-14)	2	1
			Washer, Lock (7/16 x 25/32 x 1/64")	2	1
			NLA - Bolt, Carriage (7/16-14 x 1-3/4")	2	1

Hydraulic Patient Lift - 9805P

Includes 9800AP Mast & 9884P Base

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
15		1024297	Locknut (7/16-14)	1	2
16		1014808	Washer, Lock (7/16 x 25/32 x 1/64")	1	2
17		1026474-NLA	NLA - Bolt, Carriage (7/16-14 x 1-3/4")	1	2
18		1140728	Kit, Boom - 9805P	1	1
			Bar, Boom - 9805P	1	1
			Button - 9805P (1-3/4")	2	1
19		1140729	Kit, Mast - 9805P	1	1
			Cap, Mast	1	1
			Grip, Hand - 9805	2	1
			Mast - 9805P	1	1
20	B	1140717	Bar, Hanger - 9805P	1	1
21		9099P	Pump, Hydraulic	1	1
	B,C,D, F	1143629	Kit, Hanger Bar w/ Clips - Retrofit	1	1
			Bar, Hanger	1	1
			Bar, Pad	1	1

NOTE: A - Thumb Screw (1/4-20 x 1") also available as a Package of 25, Part #7930368.
 B - When determining the right side from left side for this product, you must stand in front of the operating controls and face the lift.
 C - Not Shown.
 D - When using the RLS6 Scale with the 9805 Lift, you must use the Hanger Bar Kit Part #1143629 in order to install the Reliant Scale.
 E - Individual item no longer available, refer to Kit Part #1123448 for replacement.
 F - When using Hanger Bar Kit to retrofit with the 9805 Lift, you must also order the Hanger Pin Part #1064339.

NOTE: When using the RLS6 Scale on the 9805/9805P Lift, a Reliant Hanger Bar Retrofit Kit is required. The correct hanger pin is supplied with the RLS6 Scale. When using a Reliant hanger bar on the 9805P Lifts, without the RLS6 scale, the Reliant Hanger Pin is also required. See category titled, "Boom Assembly" for Hanger Pin.