


Hydraulic Patient Lift - 9805P

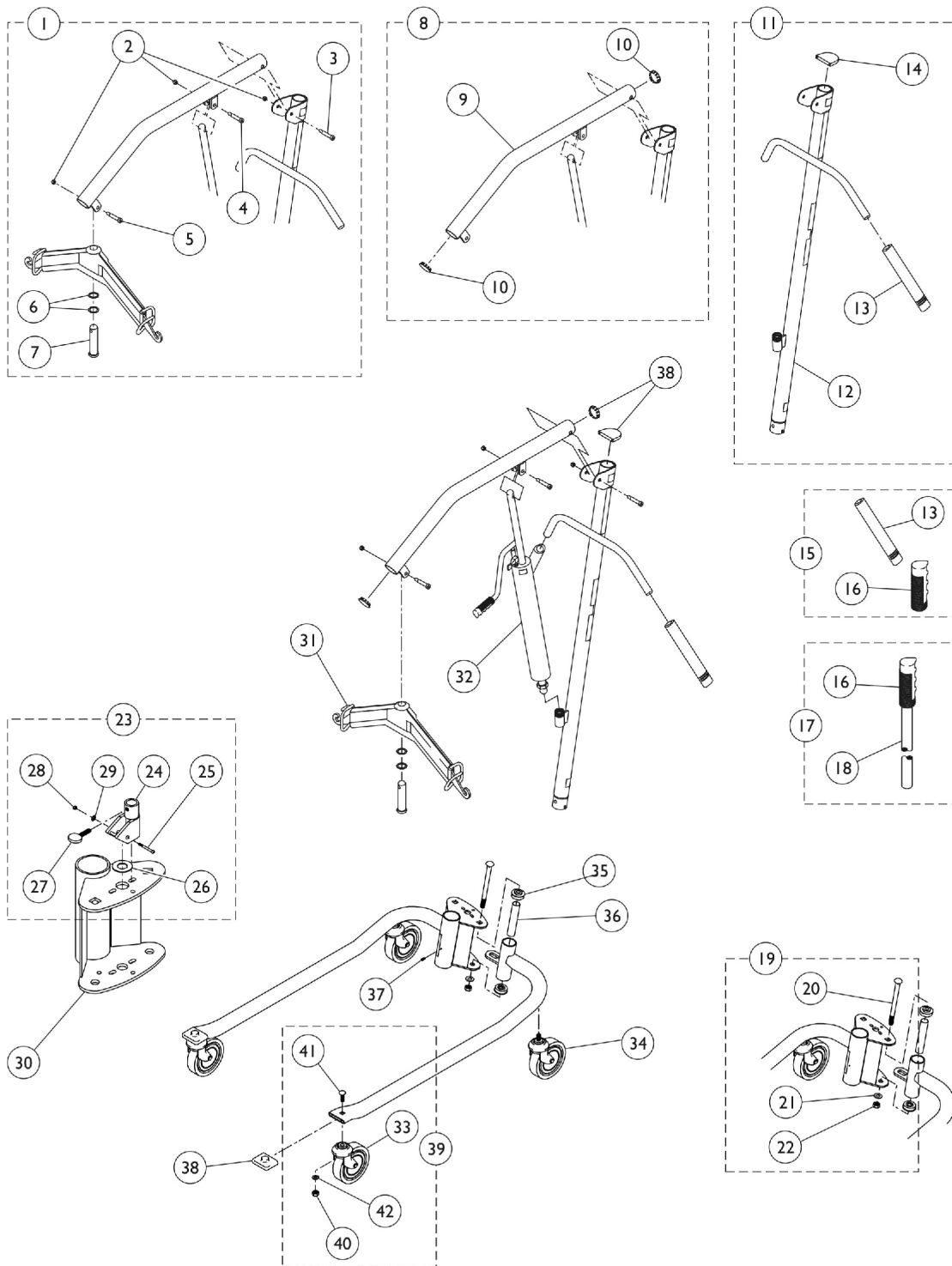

Hydraulic Patient Lift - 9805P

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
1	A	1123443	Kit, Hardware, Boom	1	1
2			NLA - Locknut (5/16-18)	3	1
3			Screw, Shoulder Hex Socket Head (3/8" x 2-1/4", 5/16-18)	1	1
4			Screw, Shoulder Hex Socket Head (3/8" x 1-1/2", 5/16-18)	1	1
5			Screw, Shoulder Hex Socket Head (3/8 x 1-1/4", 5/16-18)	1	1
6			Washer, Nylon (1-1/64 x 1-11/32 x 1/16")	2	1
7			Pin (1-1/4" x 4-1/4")	1	1
8		1140728	Kit, Boom - 9805P	1	1
9			Bar, Boom - 9805P	1	1
10			Button - 9805P (1-3/4")	2	1
11		1140729	Kit, Mast - 9805P	1	1
12			Mast - 9805P	1	1
13			Grip, Hand - 9805	2	1
14			Cap, Mast	1	1
15		1123444	Kit, Handgrip	1	1
13			Grip, Hand - 9805	2	1
16			Grip, Handle - 9805	1	1
17		1140730	Kit, Shifter	1	1
16			Grip, Handle - 9805	1	1
18			Handle, Shifter - 9805	1	1
19		1123448	Kit, Base Bolt	1	1
20			NLA - Bolt, Carriage (1/2-13 x 6-3/4")	1	1
21			NLA - Washer, Lock (1/2 x 7/8 x 1/32")	1	1
22			NLA - Nut (1/2-13)	1	1
23		1140731	Kit, Camlock w/ Hardware - 9805P	1	1
24			Cam, Lock - 9805P	1	1
25			NLA - Screw, Socket Button Head (1/4-20 x 1-7/16")	1	1
26			NLA - Bushing (5/8 x 1-1/8 x 5/16")	1	1
27			NLA - Screw, Thumb (1/4-20 x 1")	1	1
28			Locknut (1/4-20)	1	1
29			Washer, Flat (1/4 x 1/2 x 1/16")	1	1
			NLA - Instruction Sheet	1	1
30		1140721	NLA - Socket - Beige	1	1
31		1140717	Bar, Hanger - 9805P	1	1
32		9099P	Pump, Hydraulic	1	1
34		1123537	Caster Assembly, Swivel (5")	1	2
35		90068X000	Bearing, Flange	1	1
36		90071X000	NLA - Spacing, Bearing	1	1
37		60284X001	Screw, Socket Head (1/4-20 x 1/2")	1	1
38		1123442	Kit, Plugs and Caps	1	1
			Plug Button, Metal, Chrome (1-3/4")	2	1
			Cap, Mast	1	1
			Cap, End, Grey (2 x 2-1/2")	2	1
39		1123449	Kit, Caster, Front	1	1

Hydraulic Patient Lift - 9805P

Item Number	Note	Part Number	Description	Quantity Per	
				Package	Assembly
33			Caster Assembly, Swivel (5")	2	1
40			Locknut (7/16-14)	2	1
41			NLA - Bolt, Carriage (7/16-14 x 1-3/4")	2	1
42			Washer, Lock (7/16 x 25/32 x 1/64")	2	1
NOTE: A - Thumb Screw (1/4-20 x 1") also available as a Package of 25, Part #7930368.					
NOTE: When using the RLS6 Scale on the 9805/9805P Lift, a Reliant Hanger Bar Retrofit Kit is required. The correct hanger pin is supplied with the RLS6 Scale. When using a Reliant hanger bar on the 9805P Lifts, without the RLS6 scale, the Reliant Hanger Pin is also required. See category titled, "Boom Assembly" for Hanger Pin.					