

NOTE: Check all parts for shipping damage before using. In case of damage, DO NOT use the equipment. Contact the Dealer/Carrier for further instructions.

SAFETY SUMMARY

The following recommendations are made for the safe use of the Telescoping Bed Rails:

GENERAL WARNINGS

DO NOT install or use this equipment without first reading and understanding these instructions. If you are unable to understand the Warnings, Cautions or Instructions, contact a healthcare professional, dealer or technical personnel before attempting to install this equipment - otherwise, injury or damage may occur.

The Home Bed Rails when used with a manual stationary deck bed DO NOT fall under any weight limitations. Bed rails can be deformed if excessive side pressure is exerted on the bed rails. The bed rails are used for the purpose of preventing an individual from inadvertently rolling/climbing out of bed. If an individual is capable of injuring himself/herself, a physician or a healthcare professional should be consulted for alternative means of safe restraint.

Replacement mattresses and bed rails with dimensions DIFFERENT than the original equipment supplied or specified by the bed manufacturer may not be interchangeable. Variations in bed side rail design, thickness, size or density of the mattress could cause entrapment. Use ONLY authorized Invacare replacement parts.

DO NOT use the bed rails as push handles when moving the bed.

INSTALLATION AND OPERATION WARNINGS

After ANY adjustments, repair or service and BEFORE use, make sure all attaching hardware is tightened securely.

Always test to make sure that the Home Bed Rails are properly and securely in place BEFORE using.

Make certain that the crossbraces do not exceed width of mattress. Otherwise, individuals may become entangled between bed rail and mattress or bed frame.

To ensure proper bed rail installation, arrows crossbrace decal MUST point to center of bed.

INSTALLATION AND OPERATION WARNINGS

Raise the bed rail to maintain the necessary height of the side rail for patient safety keeping in mind the gap between the bottom of the side rail and the mattress.

Unless the rail is in the lowest position, ensure that the spring loaded knobs fully protrude through the same respective adjustment hole of each rail tube. This ensures that the rail is securely locked in position and an even height adjustment is achieved.

The Home Bed Rail is only for use on beds with stationary decks. DO NOT use on beds with movable decks.

This bed rail will fit on widths of 36-inches to 58-inches. Make certain that the crossbraces do NOT exceed the width of the mattress. Objects may become caught between rail and mattress if the space is too wide.

NOTE: Rail Sections Telescope from 47 to 74-inches along the length of the bed.

INSTALLATION OF HOMEBED RAIL MODEL NO. 6631 (FIGURE 1)

Installing the Crossbraces

1. With the bed in the flat position, remove the mattress from the bed.
2. Adjust the width of the crossbrace assembly tube 1 to 3-inches wider than the mattress by depress the snap buttons and sliding the crossbrace assembly tube to the desired width.
3. Make sure that both snap buttons lock through the holes in the crossbrace assembly tube.

NOTE: The crossbar MUST be installed with the slotted side of the barrel toward the center of the bed (FIGURE 1), and swedged end downward. If the slotted portions are not facing the center of the bed, rotate crossbrace 180°.

4. Position one (1) set of crossbraces at each end of the bed between the bed spring and the mattress.
5. With the outside telescoping portion of the rail (Item 6) toward the head section of the bed, insert the bed rail into the barrels of the crossbraces.
6. Pull the plunger knob (Item 5) allowing the rail to lower down along the side of the bed.
7. Pull rail up until plunger pin (Item 4) snaps into position.

PARTS LIST

Ref. No.	Part No.	Description
1	1027096	7/8-inch Plastic Plug Button
2	19000x023	Spring for Plunger
3	1012658	3/4 to 7/8-inch Tube Spacer
4	1012660	Plunger Pin
5	1012656	Plunger Knob
6	1012669	Outside Telescoping Rail
7	1012670	Inside Telescoping Rail
8	1013305	Telescoping Rail Complete
9	40188x009	1/4-inch Snap Button
10	1022798	Crosstube Assembly
11	1023599	Crossbrace Assembly
12	1022790	Crosstube Assembly
13	1042353	Crosstube Assembly Complete
	1016042	Carton Insert
	1019093	Warranty
	1027708	Invacare Detail
	40131x030	Carton

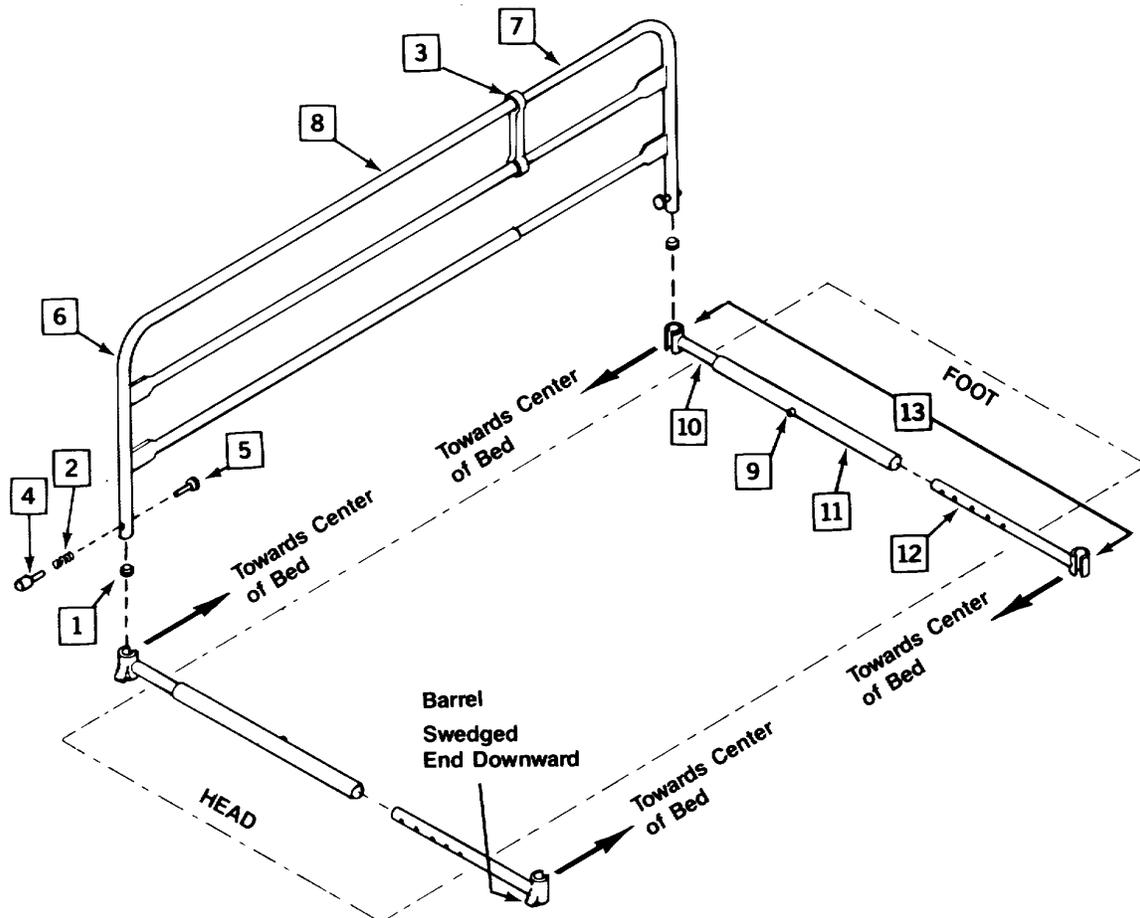


FIGURE 1 - INSTALLATION OF HOMEBED RAIL MODEL NO. 6631

Invacare Corporation www.invacare.com



Yes, you can.™

USA
 One Invacare Way
 Elyria, Ohio USA
 44036-2125
 800-333-6900

Canada
 5970 Chedworth Way
 Mississauga, Ontario
 L5R 3T9, Canada
 905-890-8838

Invacare and "Yes, you can" are trademarks of Invacare Corporation.
 © 2000 Invacare Corporation
 Form No. 90-81 Part No. 1023603 Rev E (1) 7/00